Paramedic CU-ER Series

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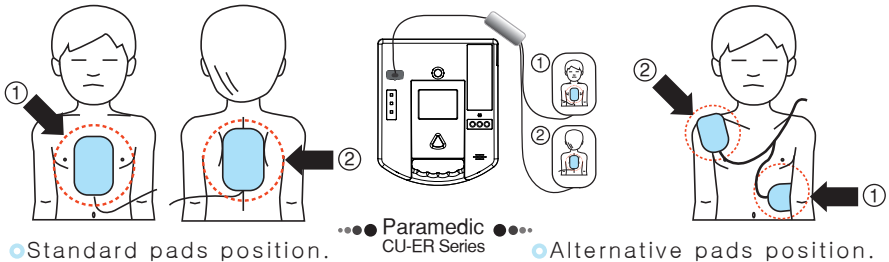
Paramedic CU-ER Series Infant / Child Reduced Energy Defibrillator Pads

These pads are for use with the Paramedic CU-ER Series defibrillators operated in AED mode only. Use on children up to 8 years old or up to 55lbs (25kg).

Do not delay treatment to determine the exact age and weight of the child.

If the child appears older or larger, use the Paramedic CU-ER Series adult defibrillation pads

- The Paramedic CU-ER Series defibrillators are intended for use on victims of sudden cardiac arrest with all of the following symptoms: unresponsiveness, absence of normal breathing, and lack of detectable pulse. If in doubt, apply the pads.
- The Paramedic CU-ER Series defibrillators (AED mode) and the CUA0711P pediatric defibrillation electrode pads are intended to be used by persons who have been trained in their operation. The user should also be qualified by training in Basic Life Support or other physician-authorized emergency medical response program.

- Attach the pads to the patient. See the illustration at the back of the pads for proper placement
- If it is desired to use the alternative pads position:
Attach the pad intended for the back of the patient to the upper right side of the patient's chest.
Attach the pad intended for the chest of the patient to the patient's side torso.

► INSTRUCTIONS FOR USE

- Turn ON the defibrillator.
- Remove clothes from upper body of the patient.
- Ensure that skin site is clean and dry.
- Tear open the electrode pouch.
- Connect the pads connector to the defibrillator.
- Apply to patient as shown on the graphic at the back of the pads.
- press down the electrode pads to the patient's skin.
- Stand back and press the SHOCK button when instructed.

▶ THESE PADS ARE

- For external defibrillation use only. Not for pacing, synchronized cardioversion, or monitoring.
- Disposable and for single patient use only. Do not reuse.
- Self adhesive with a conductive gel made of polymer.
- To be stored in a cool, dry place (-12 °C / 10 °F to 43 °C / 110 °F)
- Non sterile

▶ CAUTIONS

- Do not open package until just prior to using the electrodes.
- Do not apply any substance to the skin surface that will leave a residue e.g. lotion, oils, gel, water, etc. Make sure the skin surface is dry, i.e. wipe dry water, sweat, alcohol, etc.
- Apply a new pair of electrodes when there is indication that the electrodes are not adhering properly.
- For long term use, apply a new set of electrodes every 24 hours.
- The adhesive on the backing foam is mildly irritating to skin.
- The electrodes and its packaging must be disposed of according to national and local regulations.

▶ WARNINGS

- Use only with the Paramedic CU-ER Series running in AED mode (defibrillator output is 150 Joules, pads output is 50 Joules, nominal;)
- Electrode performance may be adversely affected by removing them from their package and exposing them to air for long periods of time which can cause some drying out.
- Do not use if conductive polymer gel is dried out.
- Defibrillators and their accessories contain ferromagnetic materials. These must not be used in the presence of the high magnetic field created by a magnetic resonance imaging (MRI) device. The said high magnetic field will attract the equipment with a force sufficient to cause death or serious personal injury to persons between the equipment and the MRI device.
- Defibrillation may cause implanted electrical devices to malfunction. Place the electrode pads away from implanted devices if possible.
- Due to possible fire and explosion hazard, do not use a defibrillator and these electrode pads in the presence of concentrated oxygen and flammable anesthetics.
- Misapplication of any pad may result in patient burns.

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CU Medical Systems, Inc.
EU Authorized Representative of CU Medical Systems, Inc.
Branch Office of CU Medical Systems, Inc. in Germany