# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**BARELLA A SOLLEVAMENTO AUTOMATICO**  
**AUTOMATIC STRETCHER**  
**BRANCARD À LEVAGE AUTOMATIQUE**  
**KRANKENBETT MIT AUTOMATISCHER HEBEVORRICHTUNG**  
**CAMILLA DE ELEVACIÓN AUTOMÁTICA**  
**AMACA DE RODAS DE ELEVAÇÃO AUTOMÁTICA**  
**ΦΟΡΕΙΟ ΜΕ ΑΥΤΟΜΑΤΗ ΑΝΥΨΩΣΗ**  
**نقالة الرفع الأوتوماتيكية**

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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**Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd**

No.999 Hongqi Road, Changyinsha  
Modern Agricultural Demonstration Area,  
Zhangjiagang City, Jiangsu, 215623, China  
Made In China

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### II. How to use

- The stretcher surface is made of soft sponge mattress, which make patient feel comfortable. One medical staff can push the trolley on to the ambulance or send to ward.
- When need low position, pull out the safety handle & then press it, the stretcher's height will automatically lower down.
- When destination arrived, pulling out the safety handle, the stretcher's legs will unfold automatically.
- There are the return elastic lock device at both side of the stretcher & the leg parts. Lock tightly when turn up for patient safety, turn down for patients out.
- Pressing the button under the stretcher, the head part will be raised or lowered.

### III. Applied for

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, etc. to carry the wounded & patients. It also can be used in general operation.

### IV. Cautions

- When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
- In general operation, be careful not to out torn the stretcher surface.

### V. Maintenance

- Keep clean as routine (incl. sterilization).
- Often check if any parts loose or not.

### VI. Storage & transportation

- Store this product in the place of damp proof & non corrosion.
- Common transporting vehicle can carry this product.
- Upside down is prohibited during transportation & storage. Stack the goods as per the instructions printed on the package.

 REF	Product code		Manufacturer		Consult instructions for use
 LOT	Lot number		Medical Device compliant with Regulation (EU) 2017/745		Date of manufacture
	Keep in a cool, dry place		Caution: read instructions (warnings) carefully		Authorized representative in the European community
	Keep away from sunlight		Medical Device		Unique identifier
	Imported by				

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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