# GIMA

PROFESSIONAL MEDICAL PRODUCTS

## BARELLA CON RUOTE WHEEL STRETCHER

ITALIANO

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

## II. Technical Features

1. Product size (L x W x H): 188 x 53 x 21 cm
2. Folded size (L x W x H): 92 x 50 x 11 cm
3. Packing size (2pcs/carton): 96 x 54 x 21 cm
4. Net Weight: 8 kg Gross Weight: 9 kg
5. Load bearing: less than 159 kg

## II. How to use

This separable first-aid folding stretcher is mainly used for transporting patients with fractures. Both ends of the stretcher have hinge-type clutches to separate the stretcher into a right and left part. This allows for the patient to be quickly and gently placed onto the stretcher, operation table or bed.

## III. Applied for

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

## IV. Cautions

1. When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
2. In general operation, be careful not to out torn the stretcher surface.

## V. Maintenance

1. Keep clean as routine (incl. sterilization).
2. Often check if any parts loose or not.

## VI. Storage & transportation

1. Store this product in the place of damp proof & non-corrosion.
2. Common transporting vehicle can carry this product.

## VII. Package

1. Packed in high durable carton with wooden base.
2. Packing: certificate of inspection & user's manual & packing list.

	Manufacturer		Keep in a cool, dry place		Medical Device compliant with Regulation (EU) 2017/745
	Product code		Keep away from sunlight		Caution: read instructions (warnings) carefully
	Lot number		Medical Device		Authorized representative in the European community
	Consult instructions for use		Date of manufacture		Unique device identifier
	Imported by				

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

### GIMA 34077

Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd  
No.999 Hongqi Road, Changyinsha  
Modern Agricultural Demonstration Area,  
Zhangjiagang City, Jiangsu, 215623, China  
Made in China

**TD010112**

SUNGO Europe B.V.  
Fascinatio Boulevard 522, Unit 1.7, 2909VA  
Capelle aan den IJssel, The Netherlands

**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)