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ICE SPRAY - INDICATIONS FOR USE: Used for relieving pain caused by injuries, scrapes, falls or sprains, and in cases requiring cryotherapy and application of cold. Thanks to its anaesthetic effect induced by cold on the injured area, sports activities can be resumed quickly.

INSTRUCTIONS: Shake before use. Spray product on the affected area from a distance of at least 20 cm for 3-4 seconds. Repeat application if necessary. The mechanical anaesthesia induced by cold eliminates pain temporarily.

WARNINGS: Highly flammable aerosol. Keep away from sunlight and do not expose to temperatures exceeding 50°C/122°F. Container under pressure. Pressurized container may explode if heated. Do not puncture or burn contents, not even when empty. Do not spray onto an open flame or other source of ignition. Keep away from heat sources, hot surfaces, sparks, open flames and other source of ignition. Do not smoke while using. Use in a well ventilated environment. Avoid static electricity build-up. To prevent possible accumulation of flammable gas, do not use product in an excessive or improper way. Do not inhale product directly or spray it into the eyes. Do not spray on broken skin. Before spraying, make sure the injured person is not allergic to the contents (CONTAINS). Avoid contact with the eyes. If unintentionally sprayed in the eyes, rinse with water. Seek medical advice before using this product on people with cardiovascular or circulatory ailments, diabetes and/or insensitive skin (Raynaud's syndrome). Keep out of the reach of children. Do not use on babies and children under the age of 3, on pregnant and lactating women. Seek medical advice before using on children and the elderly. Seek medical advice if you are in doubt about this method of treatment or if symptoms persist. For external use, only. In the event of any serious safety incident occurring by using the medical device, please report it to the Manufacturer and to the Competent Authority of your Country. **CONTAINS:** Isobutane, propane, butane, methylal, isopropyl myristate, menthol.

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Dispatech Srl - Via al piano 20, 23020 Gordona (SO) Italy