ITALIANO

ENGLISH

ESPAÑOL

Autoclave
GIMA H100

- I** ISTRUZIONI PER L'USO
- GB** INSTRUCTIONS FOR USE
- E** INSTRUCCIONES PARA EL USO

GIMA

Distributed By **GIMA S.p.a.**

Via G. Marconi, 1 - 20060 GESSATE (Mi) - ITALY

Italian Country: ☎ 02 9538541 📠 02 95380056
✉ gima@gimaitaly.com - 🌐 www.gimaitaly.com

Other Country: ☎ +39 02 953854209 / 221 / 225
📠 02 95380056
✉ export@gimaitaly.com - 🌐 www.gimaitaly.com

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Autoclave GIMA H100 automatic

Dear Customer,

We thank you for having chosen a quality unit distributed by **GIMA S.p.a.**

This unit has been planned according to the international Safety Regulations. There is no risk for the operator if this unit is used as instructed.

IMPORTANT :

- read the instructions carefully and look at the pictures, then switch the unit on.
- keep this handbook always at hand.
- only use distilled water to avoid damages of the autoclave and of the instruments to be sterilized.
- service carefully what is said in this handbook.

Warning:

this sign shows the necessity of reading the instructions carefully.

GIMA S.p.a. reminds as well that pictures and any other documents given together with the unit is of their property and they reserve all the rights. This material cannot be placed at anybody's disposal. The handbook and pictures reproduction, even if partial, is forbidden.

INDEX

- 1) GENERAL DIRECTIONS PAGE 46

- 2) EQUIPMENTS AND EXTRA SUPPLIES PAGE 47

- 3) STERILIZATION WITH SATURATED STEAM:
GENERAL CONCEPTS PAGE 48

- 4) DESCRIPTION PAGE 50

- 5) SAFETY PAGE 51

- 6) TECHNICAL DATA PAGE 55

- 7) SIZES AND PACKING WEIGHTS PAGE 56

- 8) INSTALLATION PAGE 57

- 9) ELECTRIC CONNECTION PAGE 59

- 10) DIRECTIONS FOR USE PAGE 61

- 11) WORKING ANOMALIES AND THEIR REMEDIES PAGE 74

- 12) MAINTENANCE AND CLEANING PAGE 77

- 13) ELECTRIC AND HYDRAULIC PATTERNS PAGE 80

- 14) SPARE PARTS PAGE 84

Autoclave GIMA H100 automatic

1

GENERAL DIRECTIONS

This handbook gives information about:

- the right assembly
- the safe and efficient working of the unit
- the continuous and regular maintenance

This unit must be used only as instructed.

The operator is legally responsible for installation and working of the unit itself.

Warning:

If the operator does not make the unit correctly or does not service appropriately, the manufacturer can not be considered responsible for any breakings, damages or wrong-workings.

GENERAL INFORMATION ON DELIVERY

On receiving the unit, check if packing is not broken!

Open the box and examine that:

- the supply is in conformity with the order (see shipping document)
- there are any damages

In case of damages or missing parts, please inform the carrier, **GIMA S.p.a.** or their local distributors immediately and give details.

Never throw the packing because it will be useful to send the autoclave back for any repairs and/or maintenance.

2


EQUIPMENTS AND EXTRA SUPPLIES

EQUIPMENTS (see picture no. 1)

- | | | |
|---|---|--|
| 1 | 1 | perforated stainless steel basket (ø mm. 210, 215 mm h.) |
| 2 | 1 | litre of distilled water |
| 3 | 1 | cable with earthed plug |
| 4 | - | operator's handbook; warranty certificate; CE conformity statement |

EXTRA SUPPLIES :

- stainless steel instruments holder
- sterilization indicating tape
- biological indicator for sterilization control

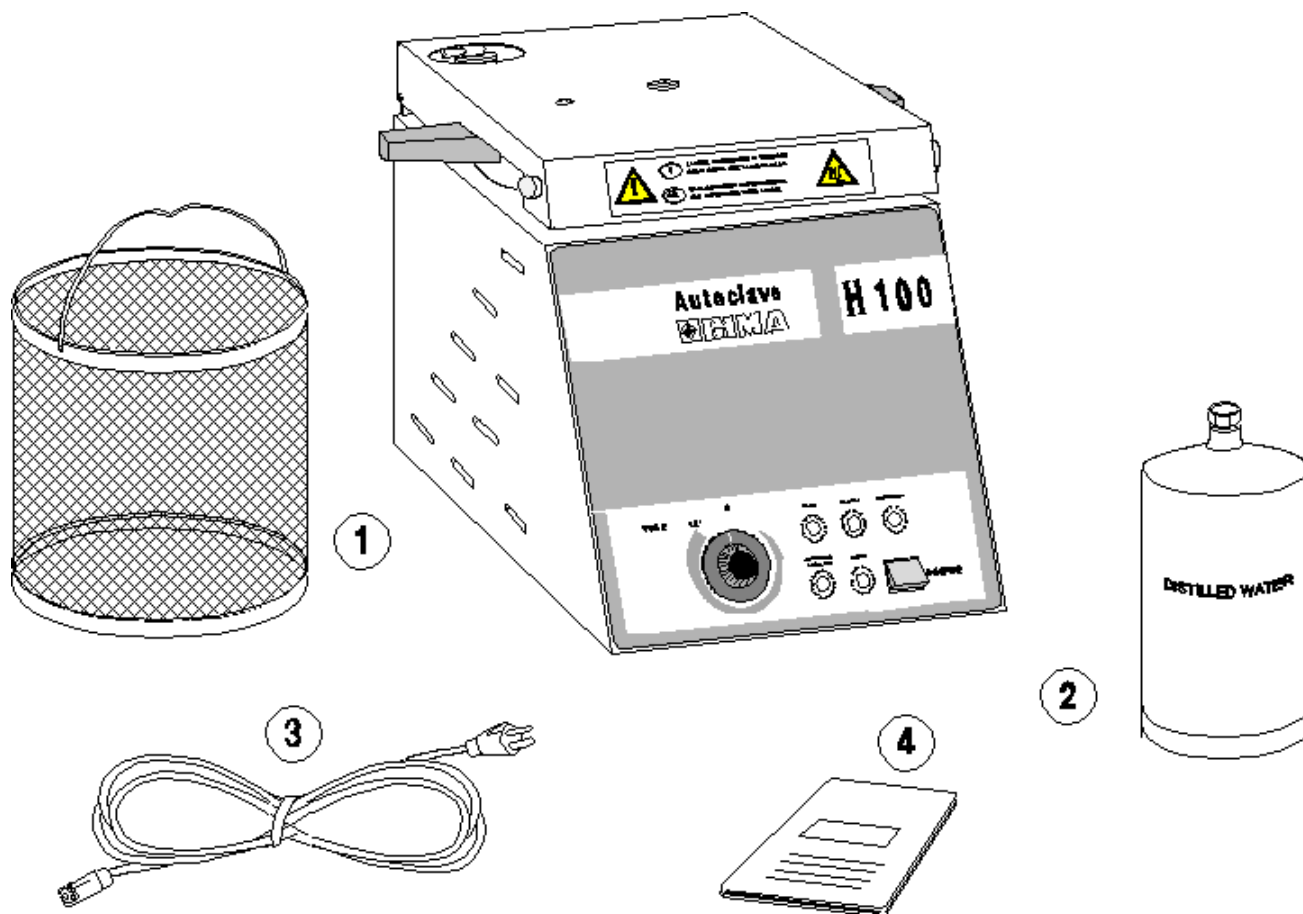Picture no. 1

STERILIZATION WITH SATURATED STEAM : GENERAL CONCEPTS

After many years of application, experience has shown that all the different kinds of sterilization are reliable. But the most effective and safest way to obtain a complete asepsis is certainly the treatment in autoclave with water saturated steam (this is the common way used in hospital).

The working of the autoclave is regulated by the Test of Boyle-Mariotte which binds pressure, temperature and volume.

The autoclaves with distilled water easily sterilize thanks to the perfect distribution of heat, the action of saturated steam, which brings about the heat exchange with the material to be sterilized (latent heat of water evaporation is extremely high: 539 at 100°C, 526 at 120°C, 519 at 130°C).

The essential reason for sterilizing through water saturated steam in pressure is that all the instruments, either liquid or solid, must be treated at the requested temperature for the right time. Pressure itself does not contribute to lethal process, but it just increases latent heat of water evaporation.

When steam comes into contact with any other colder material or body, it yields its quantity of heat by condensing and thus increasing the object temperature. In steam condensing on microorganisms (owing to the difference in temperature) this heat is integrally given with devastating effects on microbic cells. This action goes on till there is a thermal equilibrium, then there is no more heat exchange and further condensing. We can thus say that sterilization takes place in an aqueous fluid. From this introductory statement two important factors come: first, superheated steam must be avoided; second, all air must be excluded.

Superheated steam or steam heated above its usual temperature in connection with its pressure must be avoided for the following reason: although condensing takes place during the initial phase of heating, then it vaporizes again and the sterilizing process becomes a process of dry heat, whose conditions are different and sterilizing temperature is higher and time is longer.

Air must be completely excluded, so that air pockets or layers are avoided, which compromise the effective steam penetration and the right heat exchange. In fact a content of 50% air increases seven times the necessary time to kill spores and time increases 11 times in presence of 100% air.

With water steam heating process is not immediate in any case, and needed time to reach thermal equilibrium varies according to the size of the autoclave, the quantity and quality of the material to be sterilized.

With liquids or materials for surgical dressing and glass articles, where penetration is difficult or heat capacity is high, time is important and you must consider it in the treatment cycle.

Scientific European and American Community have fixed and defined three temperatures, 115°C, 120°C and 134°C and relative exposures for different materials and their resistance to temperature. Treatment at 105°C has been recognized effective with appropriate exposure, to be used for disinfection of thermolabile materials which have obviously no direct contact with patients.

Cycle at 125°C for 15 minutes of sterilization is a middle cycle for sterilization of any metal instruments.

THERMODYNAMIC VACUUM

When you push START the sterilizing chamber walls begins to heat, water becomes steam, rises and compresses cold air towards the bottom part of the chamber, as it is lighter than water. Through the steam valve (see picture no. **10** on page **64**) air goes outside. When there is no more water inside, the valve rises and the autoclave begins to raise pressure.

RECOMMENDED OBJECTS FOR STERILIZING PROCESS

- Stainless steel instruments
- Stainless steel surgical instruments
- Carbon steel instruments
- Dynamic instruments (metallic hand-pieces and turbines)
- Material in thermoresistant glass

PRESSURE TABLE / SATURATED STEAM TEMPERATURE

Temperature (°C)	Pressure (bar)	Temperature (°C)	Pressure (bar)	Temperature (°C)	Pressure (bar)
100	0,00				
101	0,05	116	0,75	131	1,79
102	0,10	117	0,80	132	1,87
103	0,14	118	0,88	133	1,95
104	0,17	119	0,92	134	2,05
105	0,21	120	0,98	135	2,14
106	0,25	121	1,06	136	2,23
107	0,30	122	1,11	137	2,32
108	0,35	123	1,17	138	2,41
109	0,39	124	1,25	139	2,51
110	0,44	125	1,32	140	2,72
111	0,48	126	1,40	141	2,81
112	0,52	127	1,46	142	2,83
113	0,58	128	1,54	143	2,95
114	0,64	129	1,62	144	3,05
115	0,70	130	1,71	145	3,16

Autoclave GIMA H100 automatic

4

DESCRIPTION

This autoclave is an extremely simple and economic unit, it is equipped with a control panel with every controls and required information for its proper use.

It has one sterilizing cycle which is optimized for a perfect and quick sterilization of the different materials used in hospital and/or surgery.

The unit is made of a boiler with electric heating resistance, a door, a gasket, a timer to count sterilizing time and a microswitch to check temperature.

Electric resistance is electronically checked to obtain a quick increase of water temperature and the steam keeping at the set temperature.

Sterilizing cycle may be described in four phases:

1. Loading of materials to be sterilized and locking of the door.
2. Heating of the chamber, thermodynamic vacuum of air and next increase of temperature and steam pressure to the set value.
3. Beginning of sterilization for the set time (timer decrement).
4. Manual steam exhaust or its condensation and bind on the bottom of sterilizing chamber.

Experience acquired in these years has brought to technological improvements for autoclave safety, especially:

- Safety lock , which prevents the opening of the door during the sterilization cycle; the door can only be opened when pressure inside the chamber is levelled to the ambient one.
- Safety valve for steam exhaust in case of damage with inside overpressure > 2.5 bar.
- Safety thermostat which occurs if the inner temperature of the chamber rises over 135°C .
- Thermodynamic vacuum.

SAFETY

- The unit has not been planned to be used in presence of explosive gas or fumes.
- You must not pour water or other liquids on the unit (only into the chamber) in order to avoid short circuits and corrosion.


WARNING

If it is not possible to disconnect electric energy and the cutout is far or cannot be seen by the one who is assistance, you must put the sign **WORKING** on the cutout after positioning **OFF** (see **picture no. 2**).


Make sure that electrical system has an earthing according to the country safety regulations (see **picture no. 3**)

- Do not remove any labels or signs; in case of need ask for a new one.
- Only ask for original spare parts.

If you do not respect what has been said above, our responsibility loses.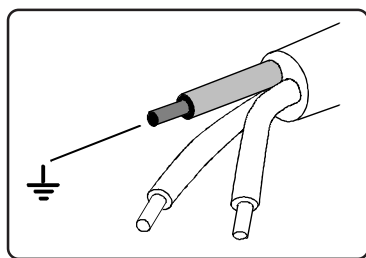

Picture no. 3

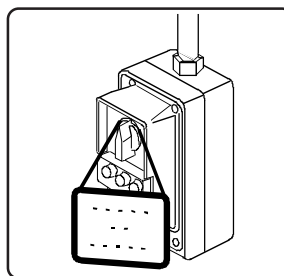Picture no. 2

- This electromedical equipment is manufactured according to current safety regulations and respects all protection requirements on electromagnetic compatibility (EMC); it's advisable to take some precautions on EMC ; this equipment must be installed and operated in conformity with EMC information, provided in this manual (see tables in the following pages).
- Portable and mobile radio communication equipments may influence the working of this electromedical equipment .

Autoclave GIMA H100 automatic

**WARNING:**

For separation distances between portable and mobile radio communication equipments and this electromedical equipment, see **table 206 page 54**

- Guidance and manufacturer's declaration :electromagnetic emissions, see **table 201 page 52.**
- Guidance and manufacturer's declaration : electromagnetic immunity, see **tables 202 and 204, page 53**

**WARNING:**

This electromedical equipment should not be used near other equipments or placed on top of them. If used near other equipments TABLE 201 or placed on top of them, it must be watched to ensure proper working.

TABLE 201

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
7

SIZES AND PACKING WEIGHTS

The unit is delivered in one packing with the following dimensions and weight :

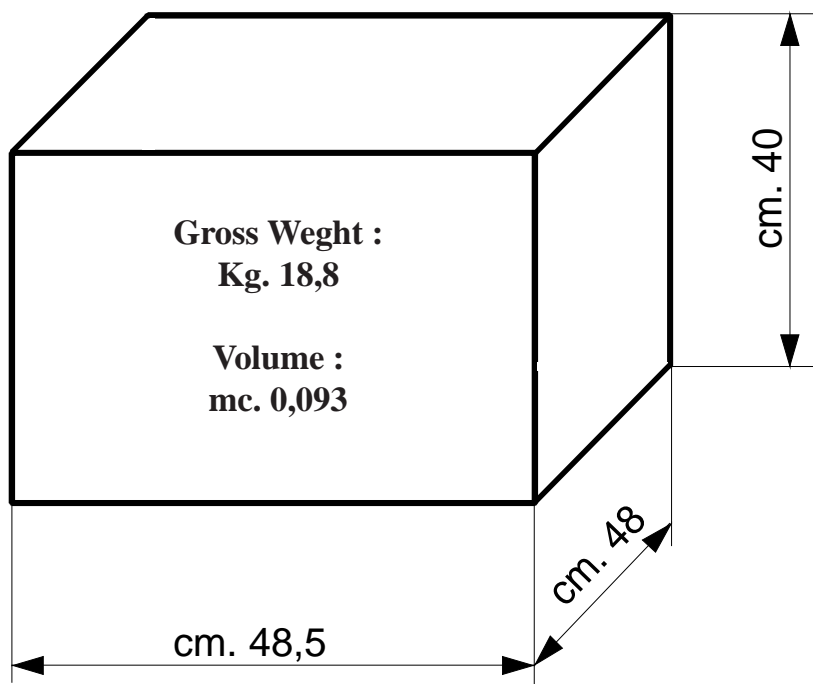Picture no. 4

INSTALLATION

The autoclave is tested and checked in the factory; so it does not need any other calibration before installing and for setting.

Unpack the unit and install it as instructed (**see picture no. 5**)

1. Put the autoclave on a level surface.
2. Leave at least cm 10 all around the autoclave to have enough airing.
3. Do not install the autoclave in places with poor ventilation.
4. Install the autoclave so that the supply cable is not folded and it can be free to socket.
5. Do not install the autoclave near sinks or other sources producing water sprinklings to avoid short circuits of the electrical system.
6. Do not install the autoclave near machines producing fumes or powders (ie. buffers, etc).


Attention

On the back of the autoclave there is a security valve : the autoclave has therefore to be put in a position where it cannot damage people or things in case of outputting hot steam.

Once installed and connected, the unit is ready for use.

Autoclave GIMA H100 automatic

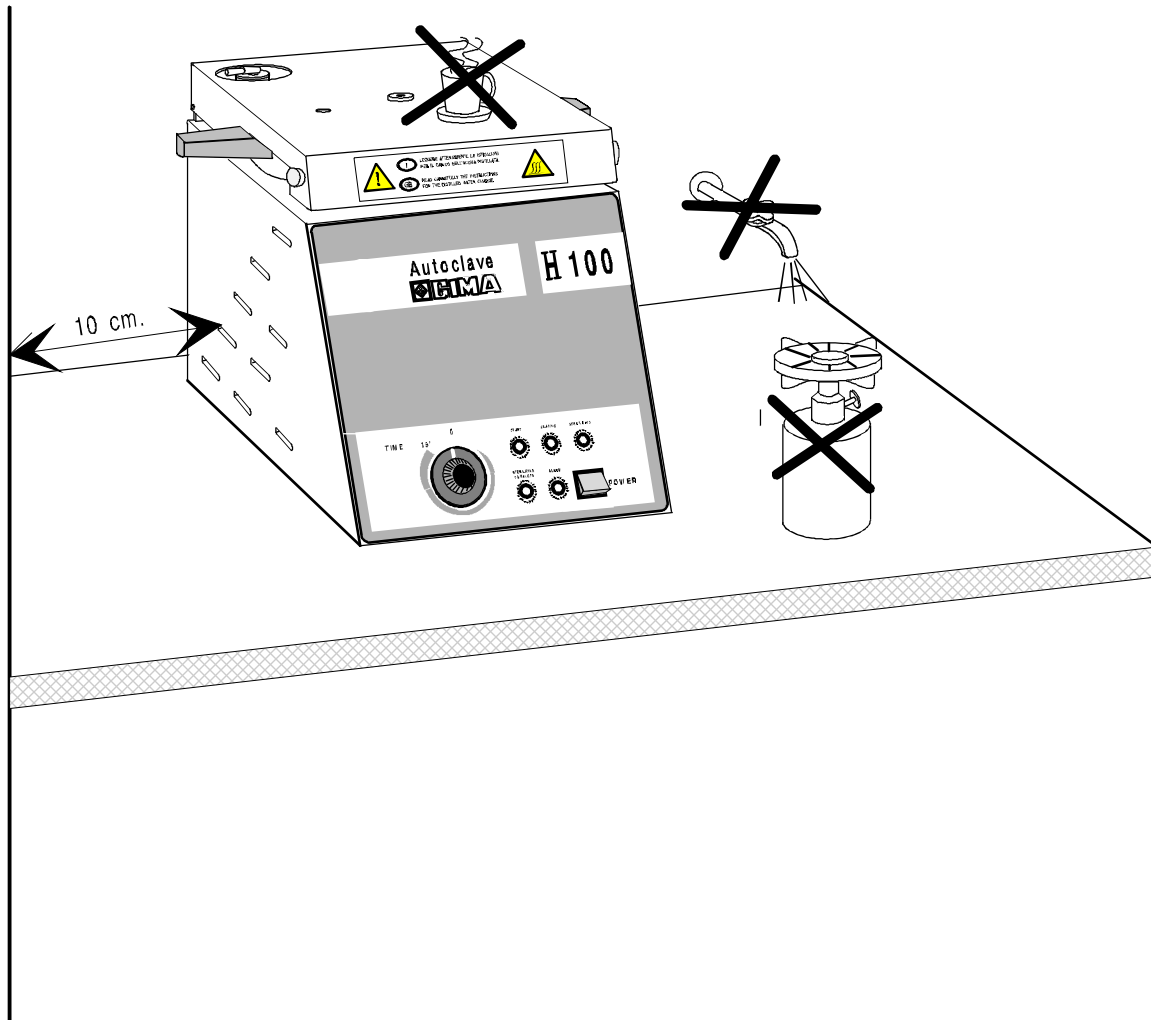
Picture no. 5

ELECTRIC CONNECTION

Check that supply voltage written on back label (1) corresponds to the one available in the installation place.

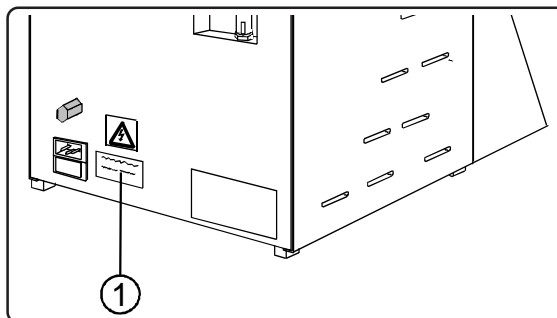Picture no. 6

The autoclave must be connected by a protection switch to a system supplied with earthing according to the country safety regulations (see picture no. 7).


WARNING

Do not cause any foldings to the supply cable and do not leave anything on it.

NOTE VALID FOR ITALY:

The system must be executed according to the regulations CEI 64-4 and a differential switch (3) must be installed with the following characteristics before the socket for the autoclave (4) :

- **nominal current** : **10 A.**
- **differential sensibility** : **0,03**

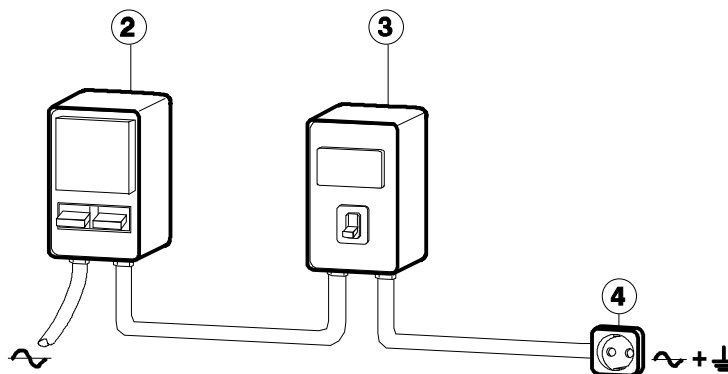Picture no. 7

Autoclave GIMA H100 automatic

WARNING:

The unit is made according to the safety regulations given by Security societies and it has a bipolarized plug with earth pole securing the complete earthing of the device.

As the electric safety of this unit is only assured when it is properly connected to an efficient earthing system (according to laws in force), it is therefore important to check this fundamental security requirement.

Make sure that the carrying current of the system and the outlet are adequate to the unit max. power given on the identification label.

In case of doubts ask for a control of the unit by skilled staff.

The constructor loses every responsibilities for any damages occurred to persons or things owing to a non -connection to the earth.


DIRECTIONS FOR USE

CONTROL PANEL: (see picture no. 8)

1. **MASTERSWITCH** : (greenlighted)
switch **on/off** to turn on/off the autoclave.
Symbols printed:
- "0" = OFF
- "T" = ON
2. **STARTING KNOB** : To start the cycle of sterilization turn it clockwise following the increasing symbols printed on the panel till the end (15 minutes)
3. **“START” LIGHT** : (green)
it shows the beginning of sterilization cycle
4. **“HEATING” LIGHT** : (green)
it shows when electric resistance is working, which means that it heats water.
5. **“STERILIZING” LIGHT** : (green)
it shows that the autoclave has achieved 125°C temperature and that it is in sterilizing phase (lasting 15 minutes).
6. **“ALARM” LIGHT** : (red)
it shows that temperature achieved 135°C temperature and safety thermostat has worked (for the new working of the autoclave you must push the red button of safety thermostat).
See chapter 11, on page 75).
7. **STERILIZING COMPLETE” LED** : (green lighted)
and shows that sterilization cycle has finished (the knob of the timer has automatically come back to “0”).

Autoclave GIMA H100 automatic

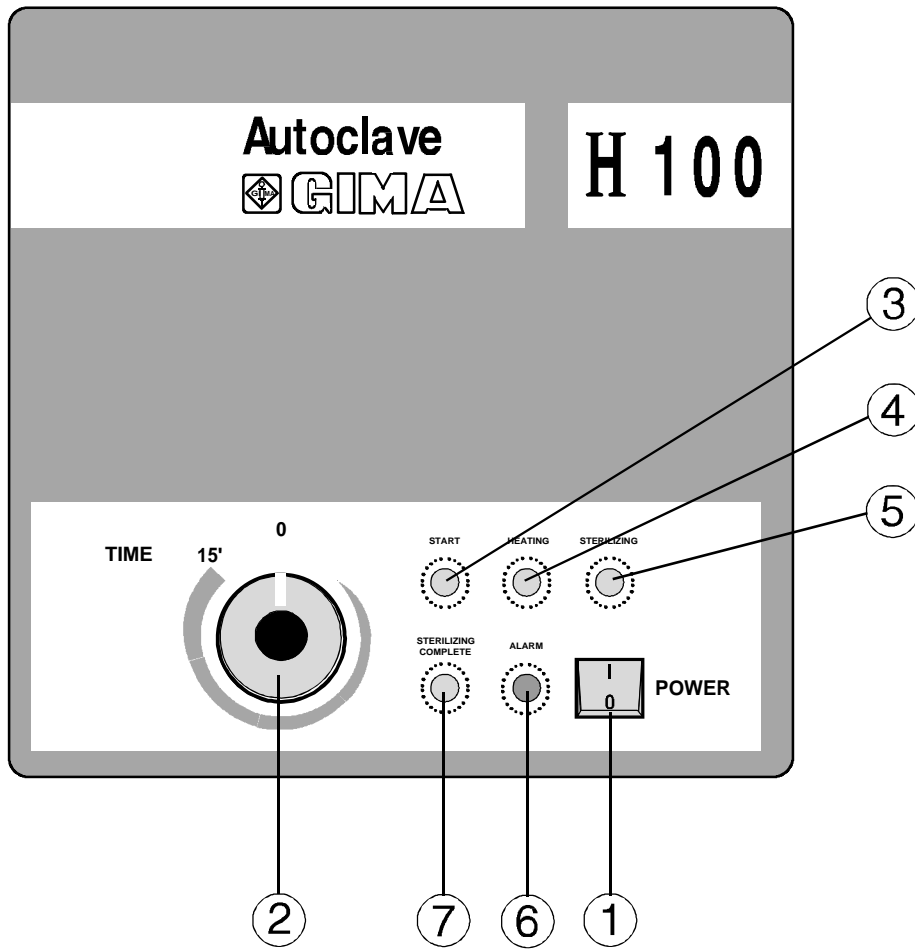Picture no. 8

HIGHER DEVICES WITH LOCKED LID (see picture no. 9)

1. **Safety bolt and valve indicating pressure**


When pressure inside the sterilizing chamber reaches 10 kPa (0.1 bar) the device provided with a red-coloured sign moves from its seat, informing the operator that inside the autoclave there is presence of pressure and preventing the lid from being opened by chance.

2. **Sticker** 

This sticker advises "High temperature" "Pay attention when opening the lid in order to avoid burnings.

3. **Sticker** 

The label recommends to read carefully the directions before filling up the distilled water.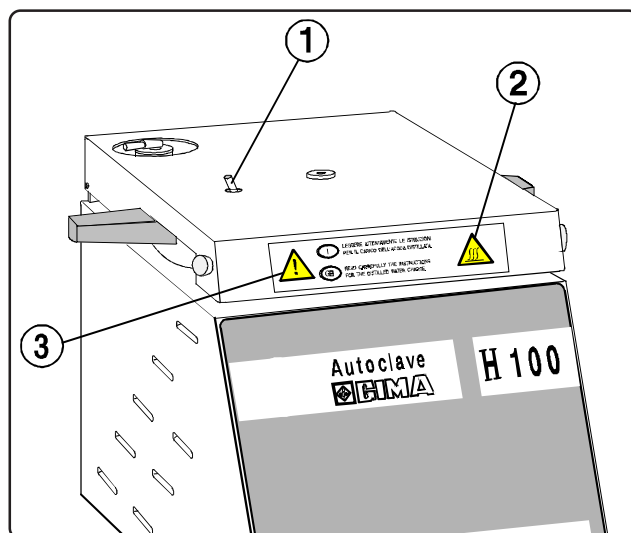

Picture no. 9

HIGHER DEVICES WITH OPEN LID (see picture no. 10)

4. **Working valve / steam bleeder**


Valve with double lock (patented) enables emission of air till the achievement of pressure corresponding to 100°C temperature. The valve closes after achieving this pressure. If pressure rises over 200 kPa, the valve escapes in order to keep pressure steady. At the end of the cycle you have a steam exhaust by lifting the little lever on the valve.

5. **Gasket**

There is a silicone gasket for sealing the lid.

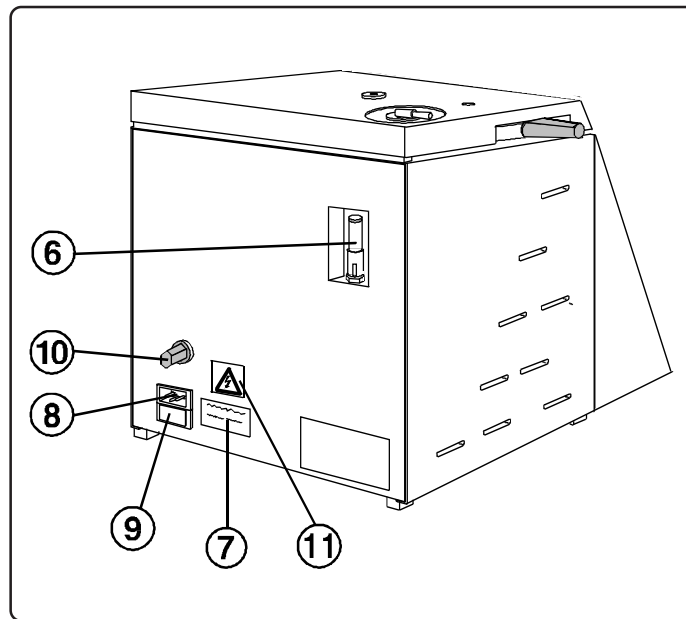
Autoclave GIMA H100 automatic

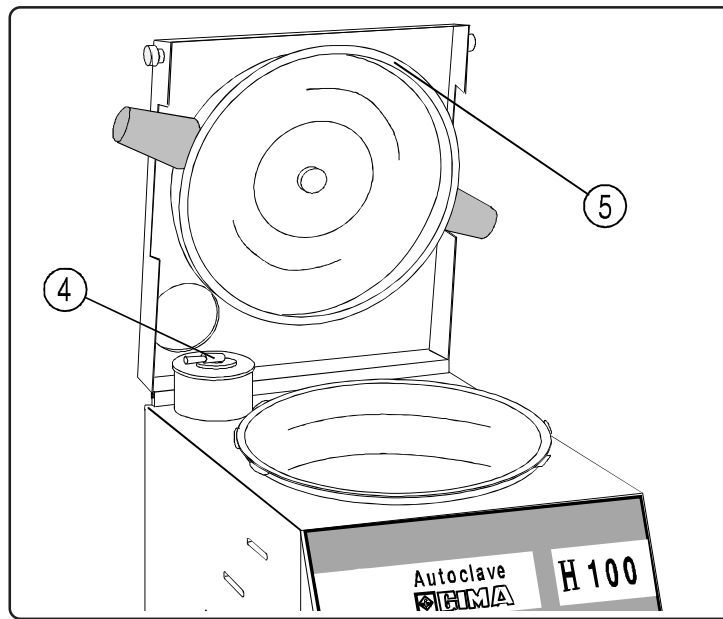Picture no. 10

DEVICES ON BACK PANEL (see Picture no. 11)


- | | |
|---|--|
| 6. Safety valve | If the pressure of the autoclave comes up to over 2,5 bar, the valve allows to release steam in excess. |
| 7. Identification label | This label contains the serial number, the year of the production, the catalogue number and all the data concerning the kind of power, the absorbed power, the type and amperage of the protective fuses, the maximum temperature and working pressure of the autoclave. |
| 8. Outlet for supply cable | Outlet for connection with the cable is in the equipments of the autoclave. |
| 9. Grid fuses | It is possible to reach the protection fuses with a small screwdriver (indication on fuse type and amperage are on the identification label). |
| 10. Button for safety thermostat | If during the sterilizing cycle the red Alarm light should go on, unscrew the black tap and push the red safety button. |
| 11. Sticker |  This sticker advise " Voltage". Don't touch any parts connected with the electric current without switching off electricity. |

Picture no. 11

FIRST START

1. Release the lid turning counterclockwise the two handles with strength (see picture no. 12).

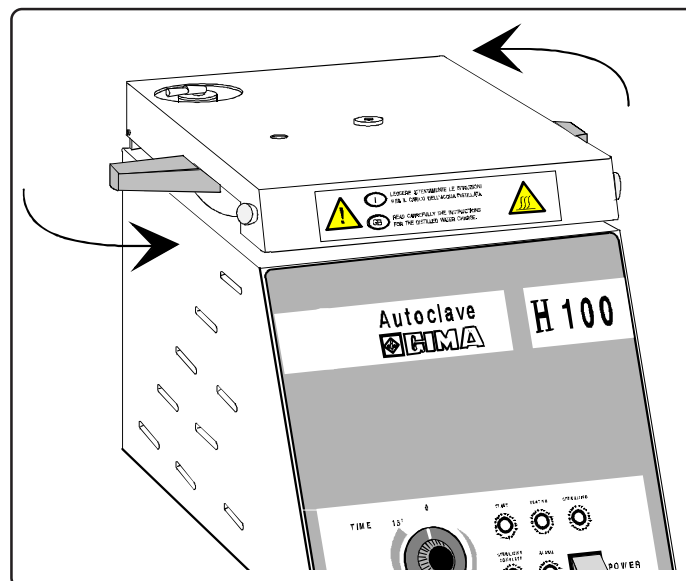
Picture no. 12

Autoclave GIMA H100 automatic

2. Lift the lid holding the white knobs on the two sides (see picture no. 13)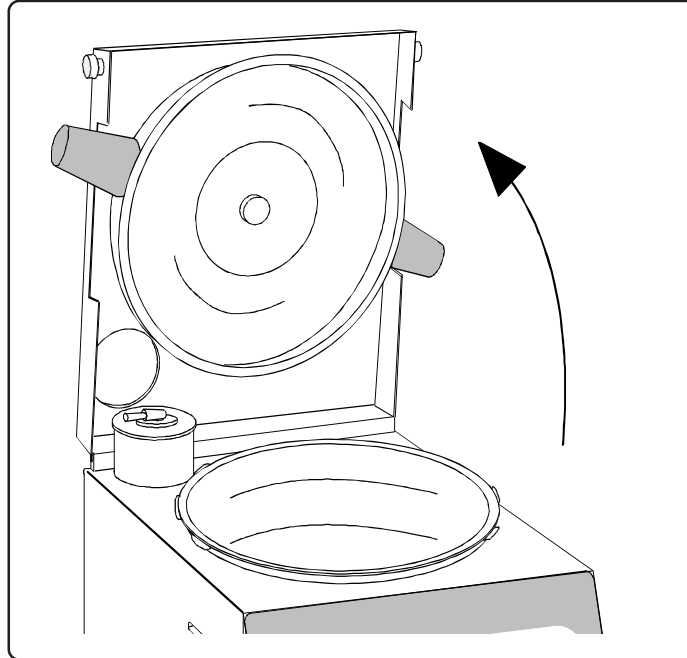

Picture no. 13

3. Remove every accessories from sterilizing chamber.
4. Connect the supply cable to an earthed safety outlet, as it is shown in chapter “**electric connection**” (section 9 on page 59)


FIRST POURING OF DISTILLED WATER

Warning :
Use always and only distilled or demineralized water.

5. Pour distilled water inside the sterilizing chamber (it is usually given with the autoclave) and cover the temperature control probe completely: about 1.0.75 (see picture no. 14)

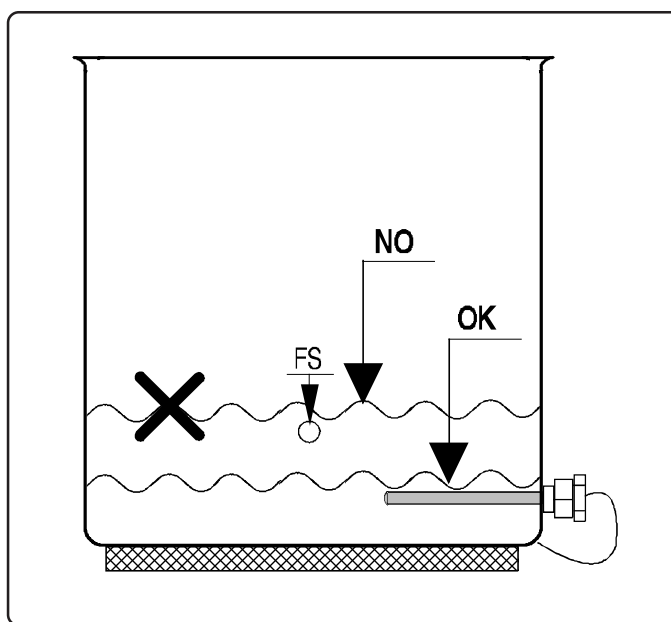Picture no. 14

Warning :
When filling up the water please follow carefully the above mentioned instructions. In any case do not fill up over the upper hole (FS) of the chamber (see picture no. 14).

Autoclave GIMA H100 automatic

PREPARATION BEFORE THE FIRST STERILIZATION :

CLEANING OF INSTRUMENTS

The instruments to be sterilized must be cleaned without any residue, such as fragments, blood, plaster, resin, tampons, etc. These substances may cause damages to the objects in the basket or to the autoclave itself.

FOLLOW THE DIRECTIONS GIVEN BELOW:

- clean the instruments after use to eliminate every residue
- use, if possible, an ultrasound unit, with detergent solution with a basis of distilled water.
- follow suggested instructions on use of the products for cleaning and lubrication of instruments after using the ultrasound unit.
- check all the directions given by the constructor to know which instruments can be put into the autoclave.

SETTING INTO THE BASKET OR INTO THE INSTRUMENTS HOLDER (optional)


- Make sure that the instruments of different materials (stainless steel, carbon steel, etc.) are separated.
- in case of carbon steel instruments, put a paper napkin between the basket and the instrument, in order to avoid any contact of the two different materials.
- put the instruments in the special holder inside its holes according to their diameter (Optional).
- apply a piece of sterilizing indicating tape directly to the instruments or the basket (not on the bottom).
- use a biological indicating test to examine the lack of spores every six months.
- to have a perfect sterilization do not overload the basket with instruments.
- empty containers must be left upsidedown to avoid any presence of water inside.

PROGRAM START


Close the lid, hold the two handles of the lid and turn them clockwise till perfect lock. Check if stainless steel lid is rightly hooked and locked (see Picture no. 15).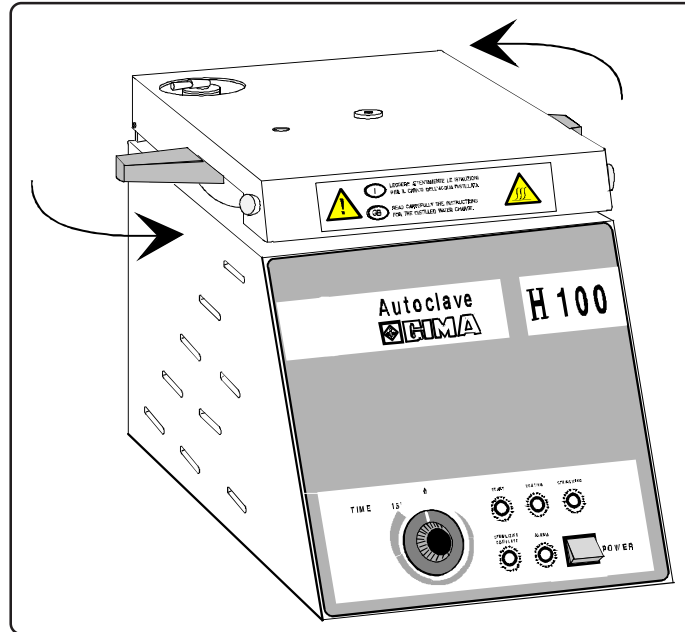

Picture no. 15

Check if the little lever of the working valve is in a horizontal position (see Picture no. 16)

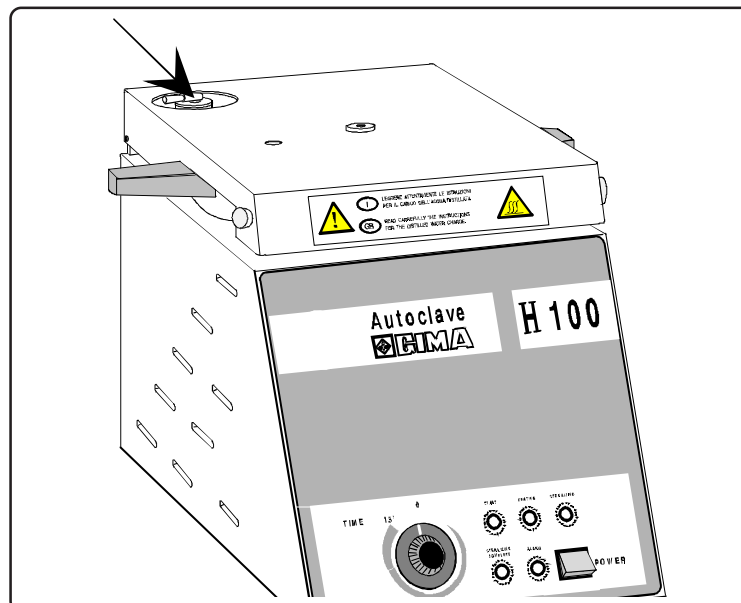
Picture no. 16


Autoclave GIMA H100 automatic

Switch the green cut-out on (see **Picture no. 17**)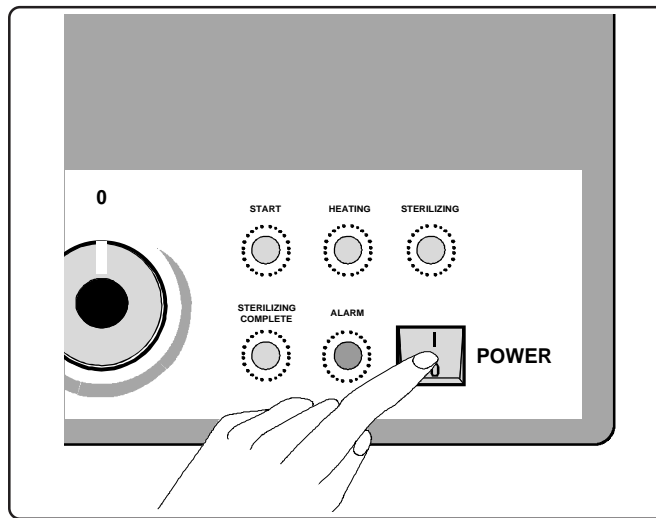

Picture no. 17

Turn the knob clockwise completely (15 minutes)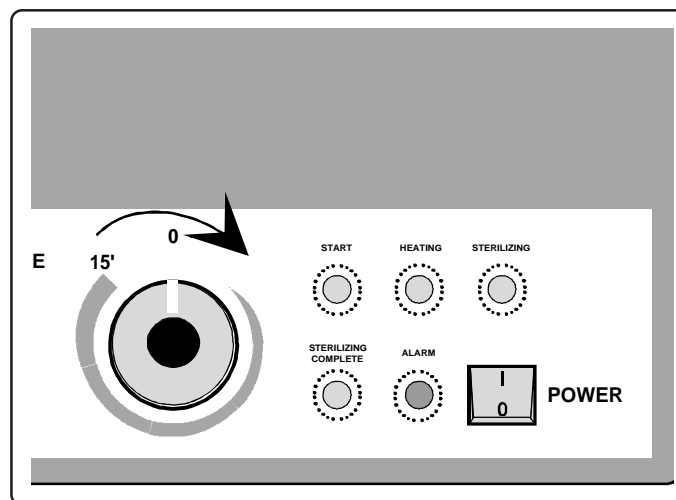


Picture no. 18

Now the automatic sterilization cycle begins and the green **START** light (3) together with the "heating" light (4) turn on. About two minutes after the pressure gauge (1) rises up (**picture no. 9 on page no. 70**) and from this moment on the lid cannot be opened any more.


When the pressure reaches the one corresponding to 100 °C the working valve rises up and closes the air bleeder.

At this moment the autoclave starts to rise its steam. When the temperature reaches 125 °C the real sterilizing cycle begins and therefore the green "sterilizing" gauge lights on (5). Now the programmer knob (2) begins automatically to turn counter-clockwise (see **picture no. 19**).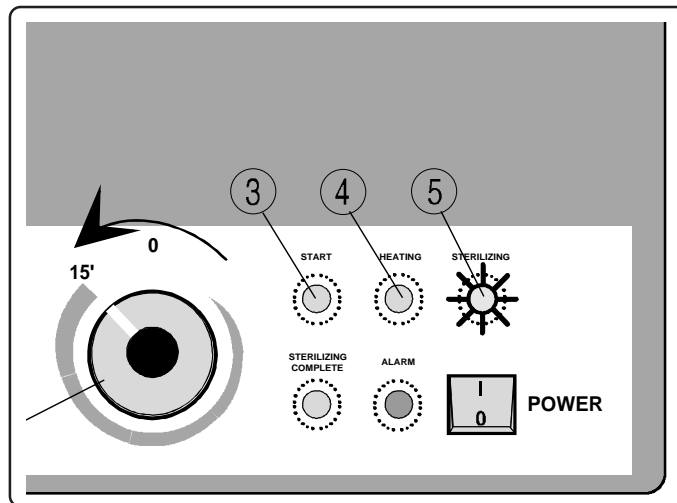

Picture no. 19

During this phase the green HEATING led lights on and off in order to keep temperature constant. After 15 minutes **START** , **HEATING** and **STERILIZING** leds are off and the green **STERILIZING COMPLETE** led is on (7).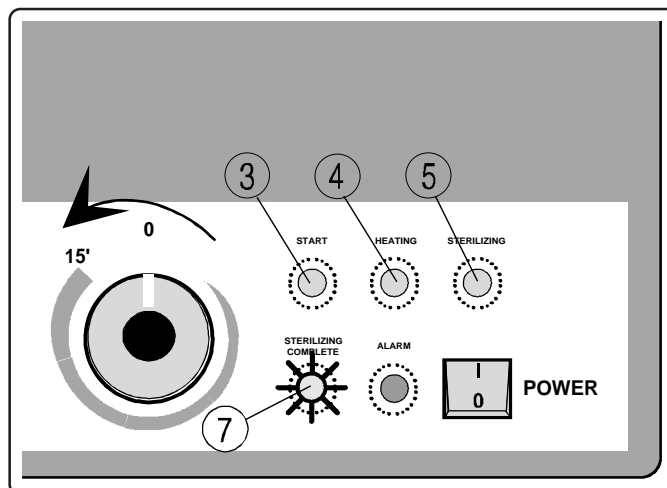

Picture no. 20

If something has been wrong during the cycle (lack of electric power, bad working) the green “sterilization” led indicating the complete sterilization will not light on. It is therefore necessary to repeat the sterilizing cycle.

END OF PROGRAM

Raise the little lever of the valve vertically to exhaust all the steam (see picture no. 21 of next page)


Attention :

Raise the lever very carefully in order to avoid being invested by high temperature steam.

Autoclave GIMA H100 automatic

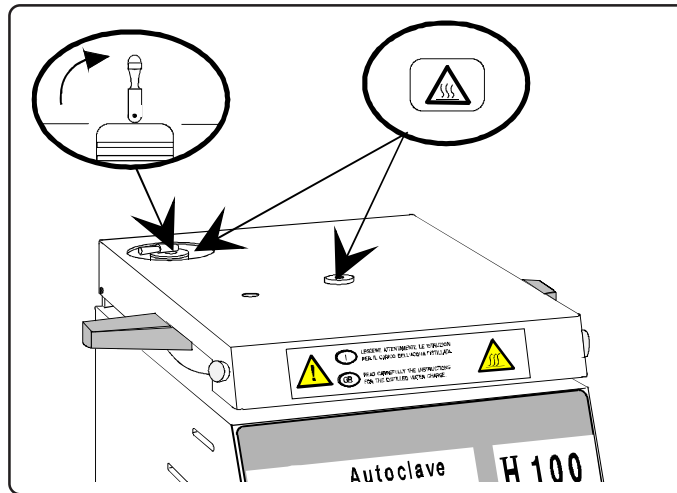
Picture no. 21

Attention :

Raise the lever very carefully in order to avoid being invested by high temperature steam.

When all the steam is out and the pressure alarm device is back inside the autoclave, unlock the lid by turning the handles counterclockwise (see picture), let the left steam go out and leave the lid ajar for 10/15 minutes, so that the sterilized instruments can dry well.

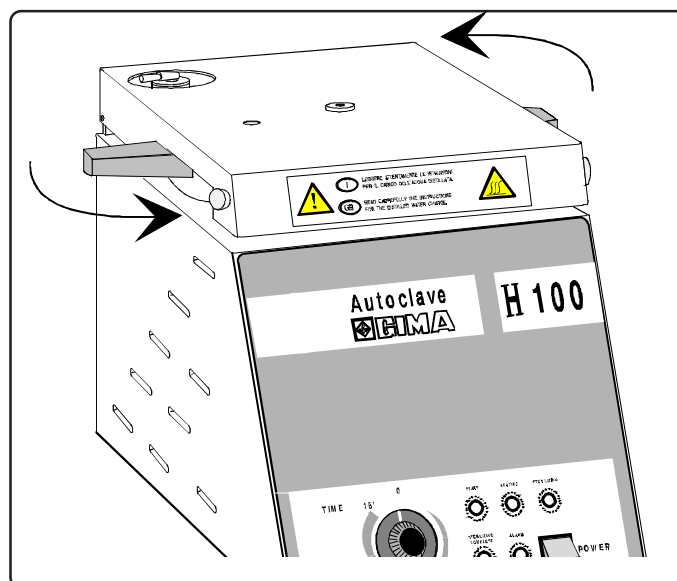Picture no. 22

DISTILLED WATER TOPPING-UP

Before starting a new cycle it is necessary to check the distilled water level. **Make sure that the temperature control probe is always and completely covered with water.**


Warning :

When filling up the water please follow carefully the above mentioned instructions. In any case do not fill up over the upper hole (FS) of the chamber (see picture no. 14 on page 67).

USED WATER DISCHARGE

Every 10 days, or after 5/10 cycles, empty the autoclave completely and incline it on one side (see picture no. 23).

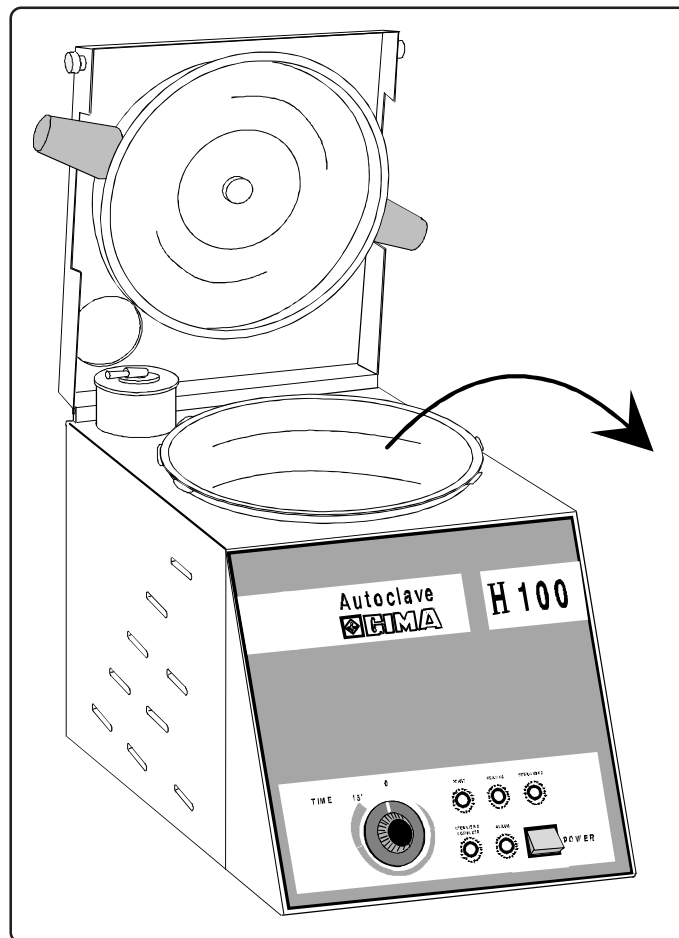Picture no. 23

WORKING ANOMALIES AND THEIR REMEDIES

Warning:

Only use distilled or demineralized water. The use of not distilled water or water demineralized with chemical stuff can cause:

1. : oxidation on sterilized instruments
2. : increase of calcareous residues on instruments, on holders and inside the sterilizing chamber
3. : pipes and valves scalings compromise the right working and the life of the autoclave.

If your autoclave does not work correctly, check as follows before calling the technical assistance:

(see next page)

Autoclave GIMA H100 automatic

PROBLEMS	CAUSES	REMEDIES
The autoclave does not switch on	cut-out switch of the unit and /or differential cut-out are OFF . lack of tension in the outlet where the unit is connected Mains fuses are interrupted the plug of current is not connected	position the switches ON check the cause of lack of tension in the outlet and find a remedy Replace them with new ones of the same value (see “ways of replacement”, chapter 12 “Maintenance and cleaning” on page no. 79) connect the plug.
START and HEATING leds do not light on when turning the programmer knob	check if the alarm led is on	If ALARM led is on you must push the safety thermostat button which is on the back of the autoclave
Heating led lights on but temperature does not increase	electric resistance is burnt electronic card is faulty	call technical assistance or send the autoclave back. call technical servicing or send the autoclave back
While the autoclave is working, the ALARM led lights on (6)	the safety thermostat has worked	Turn the autoclave off Wait till the autoclave is cold (10 / 15 min.) Unscrew the black cap situated on the back side of the autoclave (see detail no. 10 on page no. 65). With a <u>sharpened object</u> push the red button that is placed under the cap. Heck if the temperature control probe is completely covered with water. Repeat sterilizing cycle.

If bad working goes on, call technical assistance and communicate the model of the unit and its serial number, which is on the label, on the back of the unit, or on the warranty.

Autoclave GIMA H100 automatic

PROBLEMS	CAUSES	REMEDIES
The autoclave heats but pressure does not increase	<p>the lid is not correctly locked</p> <p>the valve has its lever open (vertical)</p> <p>the gasket is faulty</p>	<p>switch the unit off and close the lid properly</p> <p>close the valve positioning its lever horizontally</p> <p>substitute it as it is shown in chapter “Maintenance and Cleaning” - chapter 12 on page no. 79.</p>

If bad working goes on, call technical assistance and communicate the model of the unit and its serial number, which is on the label, on the back of the unit, or on the warranty.

MAINTENANCE AND CLEANING

Required controls are of ordinary maintenance made by the operator himself and of preventive and/or corrective measures made by the technical assistance.

Like any other electric appliances, this unit needs a proper use and maintenance or controls regularly.

This will grant you a continuous, safe and effective working of the unit. In order to prevent any risks to the operator, control and service the unit regularly.

WARNING:

In case of substitution of spare parts which have a direct influence on safety, please, only use original spare parts.

ORDINARY MAINTENANCE AND CLEANING

- For a good maintenance and cleaning of the autoclave, clean all the external parts periodically using a wet cloth and normal neutral cleansing which is not corrosive and abrasive.
- Before beginning any cycle, clean the lid gasket accurately using a wet cloth.
- The formation of white spots or rust traces on the bottom of the sterilizing chamber depends on the use of distilled or demineralized water of bad quality (rust traces come from the ferrous minerals of water).
- Do not use common abrasive cloth or metallic brushes to clean instruments.

Autoclave GIMA H100 automatic

PREVENTIVE MAINTENANCE

**Warning !**

Before making any maintenance, make sure that the line cord of the autoclave is not connected, in order to avoid electrical shocks or contacts with steam.

DAILY	Clean the gasket and the lid Clean the external surfaces Clean the internal surfaces
WEEKLY	Clean the sterilizing chamber Clean the basket and its bearing Clean and disinfect external surfaces
EVERY 10 DAYS OF AFTER 5/6 CYCLES	Empty the autoclave completely inclining it on one side, then pour some distilled water and cover the temperature control probe with water.
MONTHLY	maintenance of the working and pressure valves
EVERY SIX MONTHS	check sterilizing effectiveness with the biological indicators
YEARLY	complete control by the technical assistance
EVERY 2 YEARS	substitution of the lid gasket

CLEANING OF THE LID GASKET

Clean either the gasket or the stainless steel lid with a cloth and water or vinegar to eliminate calcareous scales. Use a non-abrasive wet cloth and normal cleansing for stainless steel to clean mirror side of the lid.

Warning:

Do not accumulate calcareous or dirt residues on the gasket because they can cause its damaging or even breaking.

CLEANING AND DISINFECTION OF EXTERNAL SURFACES

Periodically clean every external sides of the unit using a wet cloth and normal cleansing or simply water.

Use either denatured alcohol or cleansing with few hypochlorited sodium (or similar) to disinfect external sides.

Warning:

Do not wash the autoclave with direct or high pressure water jets. Any infiltrations inside electrical or electronic components could damage the regular working of the unit and its safety system.

CLEANING OF THE STERILIZING CHAMBER, THE BASKET AND THE HOLDER

Accurately clean (every week) the sterilizing chamber, the basket and the holder with a non-abrasive cloth and water or cleansing for stainless steel objects. Then rinse with alcohol and later with water.

Warning:

Never use disinfectants when cleaning the chamber.

Important :

Only use distilled water, otherwise calcareous deposits may obstruct valves..

SUBSTITUTION OF THE GASKET

Open the lid and remove the gasket from its seat. Clean the seat of the gasket with a cloth and alcohol. Insert the new one, after pouring some talcum on it, in the seat of the lid.

CLEANING OF THE WORKING AND SIGNAL VALVES

Regularly check that the working valve is always well cleaned and rises when achieving a 100°C temperature

Warning:

The rare emission of water drops and a light venthole during the sterilizing cycle means that the valve is working correctly.

- Check if the pressure signal is clean and runs freely in its seat.

REPLACEMENT OF THE FUSES

Unplug the power cable from the main power, move the power cable from the small container situated on the back side of the autoclave (see detail no. 8, picture no. 11 on page 65). With a small screwdriver unscrew the small drawer containing the fuses (detail no. 9, picture no. 11 on page no. 65). Remove the fuses and replace both of them with two of the same kind and value. To determine their value see on the serial label (detail no. 7, picture no. 11 on page no. 65).


Autoclave GIMA H100 automatic

13


ELECTRIC AND HYDRAULIC DIAGRAM

ELECTRIC DIAGRAM

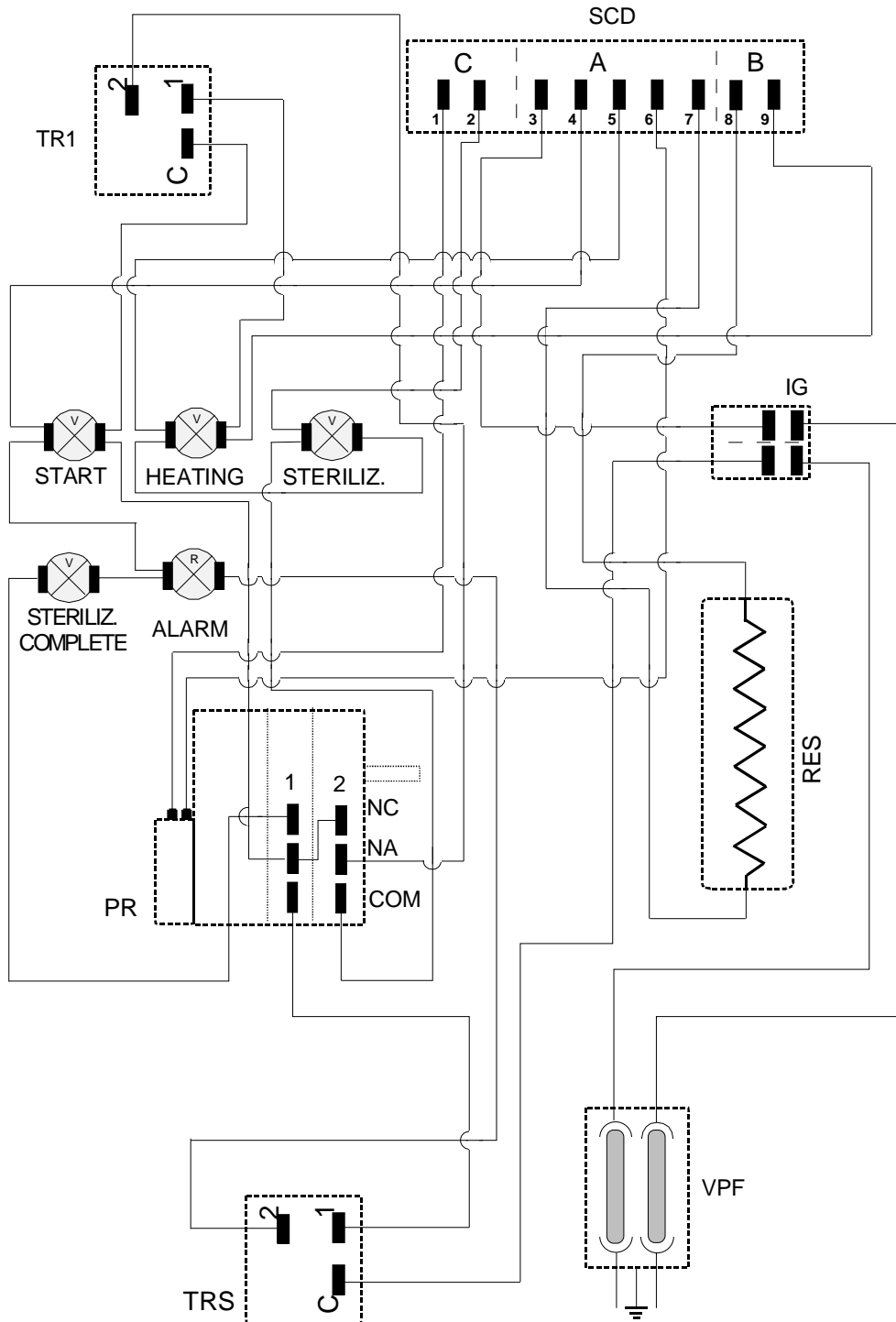HYDRAULIC DIAGRAM

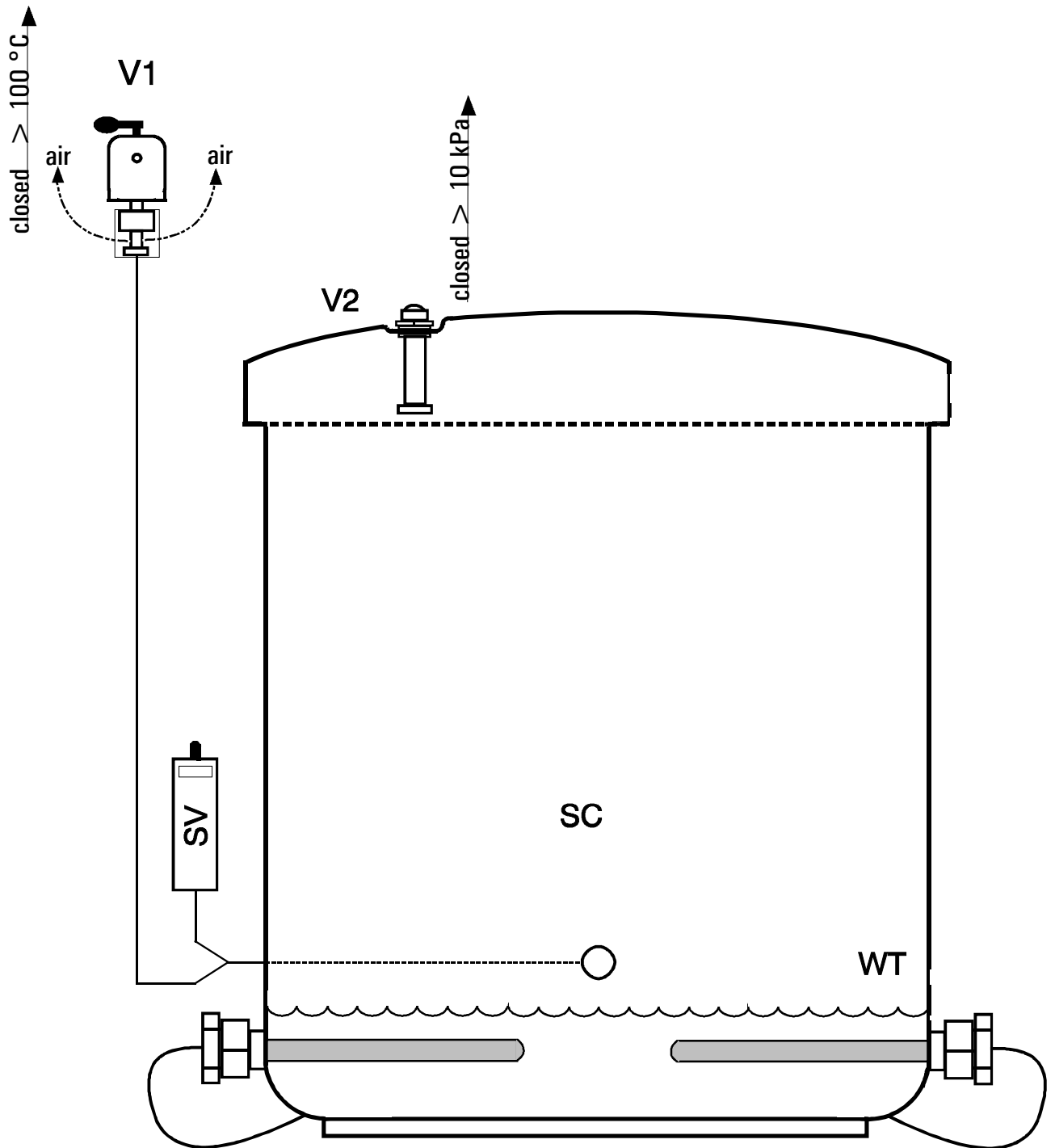Autoclave GIMA H100 automatic

FUNCTION OF THE ELECTRIC AND HYDRAULIC COMPONENTS

	V1	V2	SV	RS	TIMER
OFF	Open	Open	Closed	OFF	OFF
ON	Open	Open	Closed	OFF	OFF
START	Open	Open	Closed	ON	OFF
PRESSURE 0,1 bar	Open	Closed	Closed	ON	OFF
AIR EJECTION - 100 °C	Open	Closed	Closed	ON	OFF
PRESSURE	Open	Closed	Closed	ON	OFF
STERILIZATION	Closed	Closed	Closed	ON	ON
END (Steam recon- densing)	Closed	Closed	Closed	OFF	OFF
OFF	Open	Open	Closed	OFF	OFF

ABBREVIATIONS USED IN ELECTRIC AND HYDRAULIC DIAGRAMS

V1	=	WORKING VALVE / STEAM BLEEDER
V2	=	PRESSURE SIGNAL VALVE / LID LOCK
SV	=	SAFETY VALVE
SC	=	STERILIZING CHAMBER
AR	=	AIR RELEASE
WT	=	WATER LEVEL
PR	=	TIME PROGRAMMER
RES	=	CIRCULAR ELECTRIC RESISTANCE
TR1	=	TEMPERATURE CONTROL THERMOSTAT
TRS	=	RECHARGING SAFETY THERMOSTAT
SCD	=	INTERCONNECTING BOARD
START	=	GREEN LIGHT
HEATING	=	GREEN LIGHT
STERILIZING	=	GREEN LIGHT
STERILIZING COMPLETE	=	GREEN LIGHT
ALARM	=	RED LIGHT
VPS	=	POWER LOCKER WITH FUSE
IG	=	MAIN POWER SWITCH

Autoclave GIMA H100 automatic

14

SPARE PARTS

Description	Code
- white silicone ring gasket 60 SH	8 45 1045 002 0SB
- complete working valve	8 37 1095 D01 00N
- timer mod. D94 T-2 - 240 V. 50 hz.	8 11 1177 D94 0T2
- electric resistance \varnothing 175 - 4 hole - 230 V. 1500 W .	8 11 1056 D01 230
- 120 °C thermostat	8 11 1175 002 120
- adhesive panel GIMAH100	8 15 1117 D15 GIM
- complete cover in stainless steel	8 91 1047 S02 000
- packed basket for autoclave in stainless steel	8 90 1064 S01 000
- bottle of distilled water 1 l.	8 19 0087 001 000
OPTIONAL	
- indicating tape for autoclave mm. 19 x 50 mt.	8 11 0031 019 000
- packed instruments holder in stainless steel	8 91 1064 S03 000


(This page is intentionally left blank)

Autoclave GIMA H100 automatic

(This page is intentionally left blank)

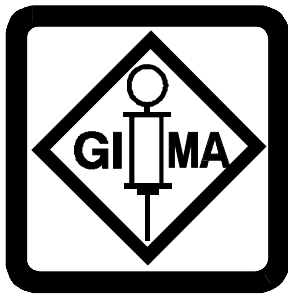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