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*Tape with change color indicator (indicators Type 1 e1) **DRY***

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*Single sided dry heat sterilization indicator tape for packing purposes, with excellent bond to almost all material. The ink printed on the surface of the tape changes its color from green to brown, to show visibly that the product has undergone a dry heat cycle of sterilization. **This product does not have the aim of validating the sterilization process.***

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Dry heat cycles: during the sterilization process, indicator printed on the tape changes its color: from green to brown.

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Dry heat cycles: at 160°C minimum sterilization time is 40 min

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Before use: before its use, store product at ambient temperature, away from direct sunlight, heat sources or too high humidity.

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After use: product is stable after sterilization and it does not require special storage conditions

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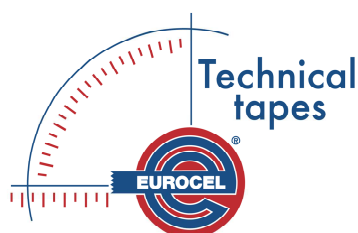
If stored according to instructions, expiry time is 30 months from production date.

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At normal sterilization conditions, the indicators of EURO TT 6120 turn from green to brown.

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Sicad S.p.A.