GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

STRUMENTARIO CHIRURGICO STERILE STERILE SURGICAL INSTRUMENTS INSTRUMENTS DE CHIRURGIE STÉRILE INSTRUMENTOS QUIRÚRGICOS ESTÉRIL STERILE CHIRURGISCHE INSTRUMENTE INSTRUMENTAL CIRÚRGICO ESTERILIZADO STERYLNE NARZĘDZIA CHIRURGICZNE STERIL SEBÉSZETI ESZKÖZÖK STERILE CHIRURGISCHE INSTRUMENTEN STERILA KIRURGISKA INSTRUMENT ΑΠΟΣΤΕΙΡΩΜΕΝΑ ΧΕΙΡΟΥΡΓΙΚΑ ΕΡΓΑΛΕΙΑ الأدوات الجراحية المعقمة

Manuale d'uso - User manual - Manuel de l'utilisateur
Guia de uso - Gebrauchs- und instandhaltungsanleitung
Guia para utilização - Instrukcja obsługi - Használati utasítás - Gebruikershandleiding - Användarhandbok
Οδηγίες χρήσης - دليل الإستعمال والرعاية

ITALIANO

Prodotti

Strumentario chirurgico sterile:

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Products

Sterile surgical instruments:

- Straight scissors, Lister Bandage Scissors, Magill Catheter Introducing Forceps, Hartmann Alligator Forceps: Medical device Directive 93/42 EEC and subsequent amendments to, or revisions of (2007/47 EEC), class I sterile devices, Annex IX.
- Mayo Dissecting Scissors, Metzenbaum Dissecting Scissors, Iris Scissors, Umbilical Scissors, Dressing Forceps, Hunter Splinter Dressing Forceps, Adson Haemostatic Forceps, McIndoe Dissecting Forceps, Pean Forceps, Spencer Wells Forceps, Halstead-Mosquito Haemostatic Forceps, Foerster Sponge Holding Forceps, Hegar-Mayo Needle Holder, Crile-Wood Needle Holder, Bellucci Otolaryngology Surgery Scissors, Suture kit: Medical device Directive 93/42 EEC and subsequent amendments to, or revisions thereof (2007/47 EEC), class IIa, Annex IX.

Instructions

Sterile surgical instrumentation includes various types of devices which have different uses according to their shape and size. They can be summarised as follows:

1. To cut threads, dressing, hair, etc.
2. To cut tight bandages
3. To grasp and position tracheal cannulas in the patient's trachea
4. To remove foreign objects from the ear
5. To cut hard tissue
6. To separate the anterior and posterior chambers of the eye
7. To cut the umbilical cord
8. To retract, grasp or dissect tissues during surgical procedures. To maintain and position dressings, pads, to remove dressings, pads and tissues; to insert/ablate foreign bodies
9. To maintain and position dressings, pads, to remove dressings, pads and tissues; to insert/ablate foreign bodies
10. To hold tissues/arteries/blood vessels in place during surgical procedures
11. To hold sponges and pads in place
12. To hold and move the needle through the tissue during suturing

Storage

Store the surgical instruments in a cool, dry place. Avoid impact which may damage the cutting edges.

Warnings

The disposable components supplied sterile, do not require further washing and sterilisation operations before use. Should the packaging be damaged, thereby compromising the sterility of the devices, or should the sterility expiry date be exceeded, the packaging including the enclosed components must be disposed of according to normal hospital procedures in a disposable container for hazardous infectious health waste intended for the incinerator by order of Presidential Decree 254/03 and subsequent amendments.

Reuse of the disposable device is prohibited. Reuse of the devices may compromise the characteristics of the material, result in the failure of the device and on different patients may cause serious infection.

Materials

All components have been manufactured using materials which comply with the current national or international regulations applicable to such devices.

In particular, the materials which come into contact with the patient are made of AISI 420 steel compliant with ASTM F899-09. Sterilisation

The surgical instruments are supplied sterile (gamma ray sterilisation) as indicated by the symbol on the label, which also indicates the expiry date and that it is disposable.

Disclaimer: GIMA S.P.A. is not responsible for any direct, indirect and/or incidental damage resulting from the use of the device, whether this derives from the lack of control of its functionality, sterilisation or validity, from the surgery or from the non-compliance with the above-mentioned Instructions for Use.

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