GIMA

PROFESSIONAL MEDICAL PRODUCTS

STURDY, FOLDING CANE WITH BUILT-IN LIGHTS

User manual

ATTENTION: *The operators must carefully read and completely understand the present manual before using the product.*

REF

**GB-801 (GIMA 43044)
GB-801 (GIMA 43045)**

Manufactured by: **Ningbo Gaobo Sports Co., Ltd.**
No.1, Dali, Shenzhen Town, Ninghai County Ningbo
315614, China
Made in China

Imported by **Gima S.p.A.**
Via Marconi, 1 - 20060 Gessate (Mi) - Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Dear Customer,

thank you for your choice.

Before using the cane it is necessary to read this manual, please carefully follow the instructions about use and maintenance. Consult a physician before use.

The manufacturer/supplier cannot be held responsible for any damage caused by improper use of the product.

This cane includes some breakable components and must be handled with care.

CONTRAINDICATIONS

Not intended for persons over 242 lbs (110 kg).

Not intended for icy or slippery conditions.

Not intended for persons with latex allergies.

Please read instructions carefully.

Keep out of reach of children.

This device is not designed for use by children or people with reduced physical, sensory or mental ability, unless they are not under the supervision of an adult responsible for their safety.
Keep out of reach of children.

- 1 Pole of the cane
- 2 Handle of the cane
- 3 LED lights
- 4 Push button to lock cane
- 5 ON/OFF key for LED lights
- 6 Battery compartment
- 7 String to fasten cane on wrist

To unfold your Cane

Simply hold on to the handle and let the foldable shaft segments unfold towards the floor.

To prevent falls, you should always check to make sure that the individual shaft segments (Fig. 1) are securely locked into place.

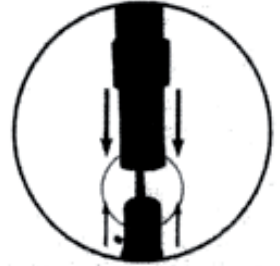
Fig. 1

Warning: Do Not disassemble the body of the cane. The cane contains an elastic cord inside.

Uncontrolled release of the elastic cord inside can cause severe injury to unprotect body parts, particularly eyes, if the elastic cord is cut, frayed or damaged discontinue use. DO NOT stretch the cord more than is needed to disassemble or assemble the segments.

To adjust the height of your cane

Loosen the twist lock (Fig. 2) by rotating it clockwise several turns. Depress the locking button (Fig. 3) and pull up on the handle to lengthen, or push down on the handle to shorten. Remember to retighten the twist lock after adjusting the cane to your desired height.

Fig. 2

Warning: To prevent falls, always ensure individual shaft segment are securely locked into place and locking button is fully engaged.

To collapse the cane

Your cane is easily foldable into a convenient travel size by separating the 4 individual shaft segments from each other. Hold each set of shaft segments near the joint and pull them apart (Fig. 4). The individual shaft segments will neatly fold together (Fig. 5).

Fig. 3

Fig. 4

Fig. 5

To use the lights on your cane

The button on the top of the cane handle (Fig. 6) activates the lights. Pressing the button once will activate the lights, and pressing it again will turn them off.

The lights will go off automatically after 20 minutes.

Fig. 6

Maintaining your cane

To replace batteries, use a head screwdriver to unscrew the battery compartment under the cane handle (Fig. 7). Remove any dead batteries and properly dispose of them. Using 3 AAA batteries, align + and - indicated, otherwise lights will not operate. Replace and tighten the screw on the battery door. Regularly inspect cane for sign of wear, including any fraying to the elastic cord or wear to the rubber feet. If you see signs of wear, discontinue use immediately.

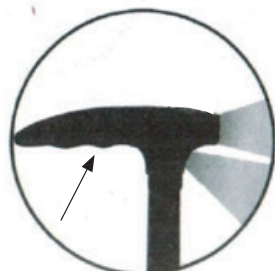
Fig. 7

Do not mix old and new batteries.

Do not mix alkaline, standard (carbon zinc) or rechargeable (nickel-cadmium) batteries.

Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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REF

GB-801 (SV) (GIMA 43044)
GB-801 (BK) (GIMA 43045)

Fabbricato da: **Ningbo Gaobo Sports Co., Ltd.**
No.1, Dali, Shenzen Town, Ninghai County Ningbo
315614, China
Made in China

Importato da **Gima S.p.A.**
Via Marconi, 1 - 20060 Gessate (Mi) - Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

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