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ENGLISH
TRIPODS - QUADRIPODS

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Our 3/4-leg cane is designed to support people with movement difficulty. It has a light weight structure, ergonomic handle and non-slip tips; it facilitates the upright position and the path, widening the base of support and providing greater stability and balance. To ensure stability of the patient will have to take short steps.

ATTENTION: use caution on downhill floor, stairs, wet or slippery surfaces. Not indicated for patients with serious motor pathology. Check the status periodically. Keep it away from children to avoid any possible risks. Before use verify that the products are not damaged, that they are adjusted correctly and that the anti-skid rubber tips are not worn. **Maximum load capacity 100 kg.**

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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Foshan Dongfang Medical Equipment Manufactory (LTD.)
5 Xingfu Road, Zone A, Shishan Industrial Park,
Nanhai District, Foshan City, China, 528000
Made in China

FS926 - FS922 - FS924 - FS9212L

Importato da / Imported by
Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

GIMA 43065 - 43070 - 43071 - 43072

MedNet EC-Rep GmbH
Borkstrasse 10. 48163 Muenster. Germany