# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**PEDALIERA PIEGHEVOLE PER RIABILITAZIONE CON DISPLAY**  
**FOLDABLE PEDAL EXERCISER WITH DISPLAY**  
**APPAREIL D'EXERCICE À PÉDALE PLIABLE AVEC ÉCRAN**  
**PEDALEADOR PLEGABLE CON PANTALLA**  
**BICICLETA DE EXERCÍCIO DOBRÁVEL COM ECRÃ**  
**ZUSAMMENKLAPPBARER PEDAL-EXERCISER MIT DISPLAY**  
**ΠΤΥΣΣΟΜΕΝΗ ΠΕΤΑΛΙΕΡΑ ΑΠΟΚΑΤΑΣΤΑΣΗΣ ΜΕ ΟΘΟΝΗ**  
**SALIEKAMA PEDĀĻU IERĪCE REHABILITĀCIJAI AR DISPLEJU**  
**PEDALIER PLIABIL PENTRU RECUPERARE CU DISPLAY**  
**SKLADACÍ PEDÁLOVÝ TRENAŽÉR NA REHABILITÁCIU S DISPLEJOM**  
**ZLOŽLJIVO KOLO ZA REHABILITACIJO Z NASLONOM**  
**HOPFÄLLBAR BENCYKEL FÖR REHABILITERING MED DISPLAY**

Manuale utente - User manual - Notice d'utilisation - Manual del usuario - Guia para utilização - Gebrauchs- und instandhaltungsanleitung - Εγχειρίδιο χρήσης - Lietotāja rokasgrāmata - Používateľ'ská príručka - Uporabniški priročnik - Användarmanual

GIMA 43151

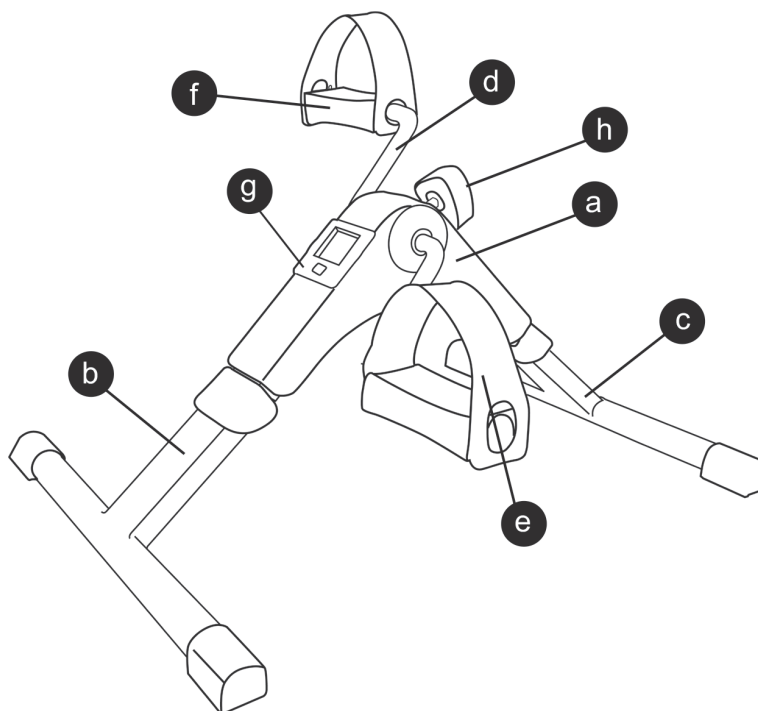
Fabbricato da/Manufactured by/Fabriqué par/Fabricado por/Hergestellt von/Tillverkad av Ražots/  
Proizvajalec/Κατασκευασμένο από/Produs de/Vyrába  
**Zhejiang Todo Hardware Manufacture co., Ltd**  
No.31th, Jinmao Building, CBD Centre,  
Yongkang, Zhejiang, China  
Made in China

**REF** TD001P-3

Importato da/Imported by/Importé par/Importado por/Eingeführt von  
Importerad av/Importēja/Uvozil/Εισαγωγή από/Importat de/Dovážal  
Gima S.p.A. - Via marconi, 1 - 20060 Gessate (MI) - Italy  
gima@gimnaitaly.com - export@gimnaitaly.com  
www.gimnaitaly.com

## DESCRIZIONE

## MONTAGGIO

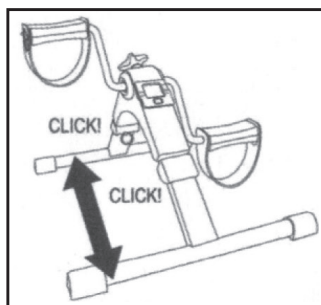
fig. 1

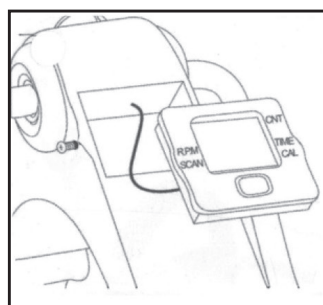
fig. 2

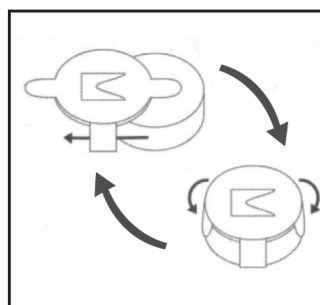
fig. 3

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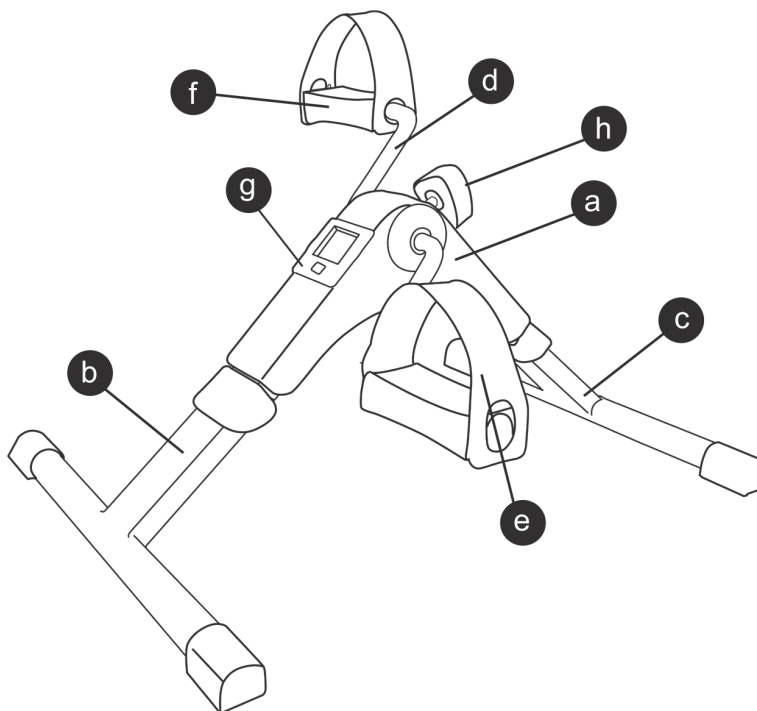
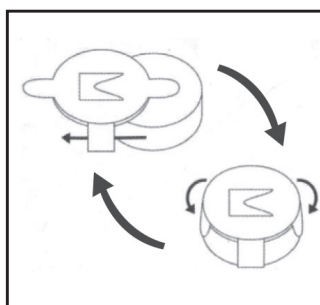
fig. 3

## **PURCHASE ACKNOWLEDGEMENT**

Thank you for the trust placed in the purchase of our product and we are certain that you will always be satisfied with any of our products.

For better service of all your questions and needs, customer service is at your disposal to solve any query:

## **PRODUCT DESCRIPTION**

- |                |                                    |
|----------------|------------------------------------|
| a. Main body   | e. Right pedal                     |
| b. Front base  | f. Left pedal                      |
| c. Rear base   | g. Counter and battery compartment |
| d. Handle bars | h. Effort regulator                |

## **SAFETY WARNINGS**

- These instructions are very important. Please read this instructions manual carefully before using the appliance and keep it at hand to refer to it at any time.
- No responsibility is assumed for the improper use of the product, or any other use that is not described in this manual
- Before using the product for the first time, unpack the product and check that it is in perfect conditions. If not, do not use the appliance and contact, as the guarantee covers any damage from origin or manufacturing defects.
- This product is only indicated for domestic use. Do not use the appliance outdoors. Never leave the appliance exposed to climatic agents such as rain, snow, sun, etc.
- This appliance can be used by children of 8 and more years old as well as by disable people, people with reduced sensorial o mental capacities, people with lack of experience and knowledge, if the appropriate supervision and instruction of use was provided them in reliable way and they understand danger it entails. The cleaning and maintenance allow to be realized by the user, must no be realized by the children, only in case they are more than 8 years old and only under supervision.
- Keep the appliance out of reach of the children younger 8 years old.
- Children should be supervised to ensure they do not play with the device.
- Do not immerse the appliance in water or any other liquid.
- Do not mix new batteries or the old and worn-out.

## **ASSEMBLY**

1. Unfold the front and the rear bases till it sounds a “click”. It means that the structure is properly attached in order to practise the exercise. Before doing any exercise, please check both bases are properly fixed (as fig. 1).
2. Place the display unit into its gap. Note that the wire must be properly -placed inside (as fig. 2).
3. Place the intensity regulator screw into its slot, placed on the top part of the mini exercise bike.

## **INSTRUCTIONS FOR USE**

**This pedaler has 5 different functions:**

- CNT – COUNTER: Total of laps done with the device during this training session.
  - TIME: Counts the effective exercising time during this training session.
  - CAL - CALORIES: Calories burnt during the exercise.
  - RPM - Revolutions per minute: It shows the average of laps per minute based on the intensity of the pedal exercise.
  - SCAN It shows the other functions every 5 seconds in turn.
  - Before starting the exercise, place the device on a flat surface and adjust the pedaling resistance according to the strength desired for the exercise (h). In order to obtain more resistance, turn the dial counterclockwise.
1. Press the red button located in the counter (g) or start pedaling. In this way the counter (g) will be started.
    - The word STOP will appear at the display when you stop pedaling and it will disappear when you renew the exercise.
    - The display turns off in a few minutes after you stop pedaling.
  2. To check the results obtained with the exercise read carefully the following procedure:

**Automatic mode to see the results:**

- Press the red button once. "SCAN" will flash. All the obtained results (TIME, RPM, CALORIES) will be displayed. They will appear for 5 seconds approximately.

**Manual mode to see the results:**

- In order to set the function you want to see during the exercise, and to pass from one function to the next, press 2 times consecutively the red button to set the desired function on the screen (TIME, CAL, RPM). If you press the red button only once, it returns to automatic mode without setting any particular function.

NOTE: To set the counter to "ZERO": press the red button for some seconds. All the indicators will be set to "0".

3. Batteries replacement: Remove the counter (g) for its original position using the notches located around the counter. You will reach the battery cabinet and there you can replace it without using any tool (as fig. 3).

## **CLEANING AND STORAGE**

- Clean first with a damp cloth and then a dry cloth.
- Keep the appliance in a safe place.
- In order to storage the device, pull the rings until both bases stop being fixed.

## **TECHNICAL DATA**

1,5V 1 = x LR44 1,5V

This device conforms to the safety requirements and provisions of directives 2014/35/EU on Low tension devices and 2014/30/EU on Electromagnetic Compatibility.

## **PROTECTION OF THE ENVIROMENT - DIRECTIVE 2011/65/EU**

This product has been designed and manufactured with high quality materials and components that can be recycled and reused.

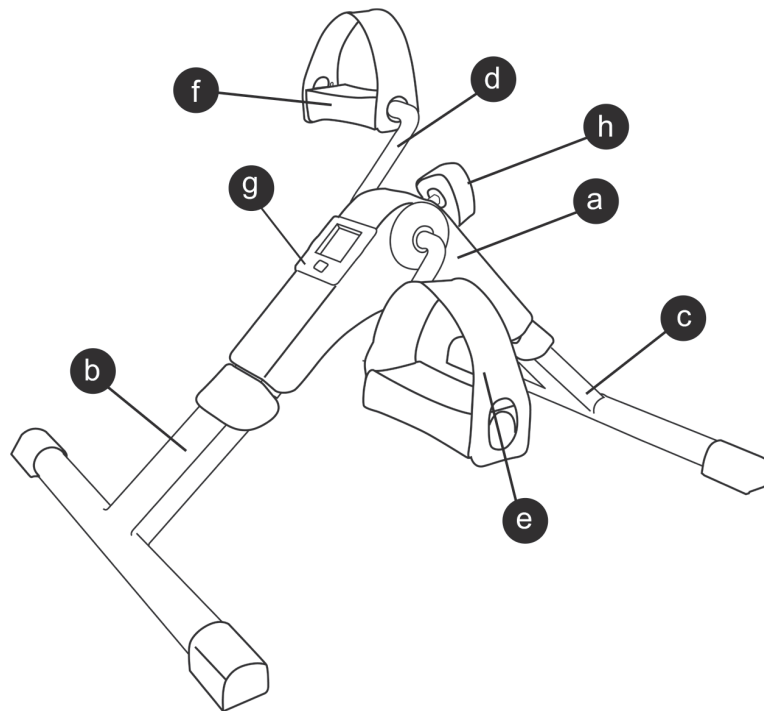
When you see the symbol of a crossed out litter bin on wheels next to a product, this means that the product conforms to the European Directive 2011/65/EU.

■ Please obtain information about the local recycling system for electric and electronic products.

Follow the local rules and do not dispose of used products by throwing them in the normal bins at home. Proper recycling of your used product will help avoid negative impacts on the environment and people's health.

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