GIMA

PROFESSIONAL MEDICAL PRODUCTS

TAVOLO PER MASSAGGI PORTATILE A 2 SEZIONI
2 SECTION PORTABLE MASSAGE TABLE
TABLE DE MASSAGE PORTABLE EN 2 SECTIONS
2-TEILIGER TRAGBARER MASSAGETISCH
CAMILLA DE MASAJE PORTÁTIL DE DOS SECCIONES
MESA DE MASSAGEM PORTÁTIL DE 2 SEÇÕES

Fabbricante/Manufacturer/Fabricant/Fabricante/Hersteller:
Anji Sukar Furniture Co., Ltd.
Tangpu Industrial Zone, Dipu Town, Anji, Zhejiang Province, China. P.C.: 313300
peter@zjruixiang.com

GIMA 44000, 44001, 44002, 44003

REF WT221A

Importato da/imported by/importé par/eingeführt von/importado por:
Gima S.p.A. - Via Marconi, 1
20060 - Gessate (MI) - Italy
gima@gimaitaly.com - export@gimaitaly.com www.gimaitaly.com
Made in China

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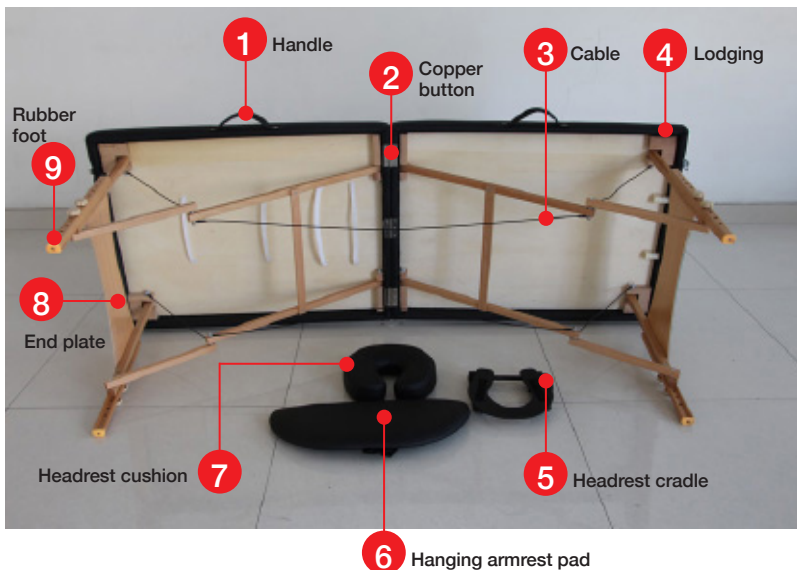
PRECAUTIONARY INSTRUCTIONS

1. Carry out periodic maintenance on the table. Make sure all screws are secure and the cable is not deteriorated or frayed. Make sure there are no cracks in the wooden structure.
2. Do not overload the table beyond the recommended weight limit. Doing so could result in serious injury to the user. Maximum working load of 200 kgs.
3. Do not use alcohol-based cleaners when cleaning the vinyl. They could cause damage to the vinyl material. Use warm soapy water to clean, then wipe dry.
4. When using the table, make sure that all four legs are at the same level. Using the legs at different levels can cause the table to be unstable and could lead to injury.
5. Do not allow children to assemble or disassemble the massage table, unless under your supervision.
6. Be careful when using sharp objects, as they can pierce the covering. Keep the table's covering away from sharp items.

SETTING UP YOUR TABLE

Steps are as below:

1

2

3

4

5

6

1. Release the latch.
2. Lay the table on its side with two rubber feet on the floor, open up the table with four legs extended as fully as possible. Take out all accessories.
3. Lift the table up to position with all legs on the floor.
4. Put the accessories into the corresponding lodging, fasten the hanging armrest or sling on the headrest cradle.
5. Release the side handle and push it in once the headrest has been adjusted to the required position.
6. Now the table is ready for your use.

The height of the table is adjustable by turning the knobs on the leg and taking out the movable leg to adjust the height, by choosing the holes on the movable leg. Then reinsert the leg and firmly fasten the knobs. All four legs should be adjusted in the same way, one by one, being sure to adjust all legs at the same height level.

CLOSING AND TRANSPORTING THE TABLE

1. Lift up the backrest to the maximum position and then push it down, lock the latch.
2. Take out all accessories.
3. Lay the table on its side with the two of the rubber feet on the ground.
4. Put all accessories under the elastic strap beneath the top veneer.
5. Fold the legs on each end internally by pressing the corresponding support arms.
6. Fold the table up and lock the latch.
7. The table can be carried like a suitcase by grasping the handle.

Warning: If the frame/legs/table structure is damaged in any way, the table should not be used. Using this table when damaged may cause harm or serious injury to you or your patient. It should be repaired immediately.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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