GIMA

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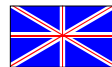
FEATURES

Product description

GIMA bone measure (cod. 27325) is an orthopedic instrument suitable to measure the bone diameter of the human body (knee, elbow, shoulder, etc.) with a tolerance of ± 3 mm up to a maximum of 150 mm.

The bone measure is equipped with a slider composed of an anticorrosive aluminium graduated rule and two plastic “jaws”.

PRESCRIPTIONS

Accident prevention

The product shall be used by qualified personnel only.

Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

TRANSPORT

Unpacking

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.

They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

OPERATION

Operating instructions

The bone measure is easy to use. It is sufficient to bring the two jaws near the part of the body to measure, make the moving jaw slide until it is in contact with the bone and read the value showed by the graduated scale.

MAINTENANCE

General maintenance instructions

The product has been conceived to last forever; it is resistant to most part of chemicals except for methyl ethyl ketones (M.E.K.), diluents and nail polish removers.

The measure should be cleaned with a cloth dampened with materials, which do not damage aluminium, for example alcohol and detergent for glasses.

After the use it is advisable to put the measure in its case and keep it in dry environment.

GUARANTEE

General guarantee conditions

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, doppler probes, SpO₂ sensors, lamps, batteries, electrodes, handpieces, washers and packings, resistances, etc.

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
No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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Fabbricante/Manufacturer: Metrica S.p.A., Via Grandi n.18, 20097 San Donato (MI) - Italia