GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

LETTINO 3 PEZZI POLIVALENTE

3-SECTION MULTIFUNCTIONAL BED

DIVAN POLYVALENT 3 PIÈCES

CAMILLA 3 PIEZAS POLIVALENTE

Manuale utente - User manual - Notice d'utilisation - Manual del usuario

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ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

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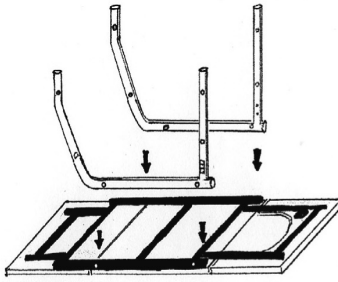
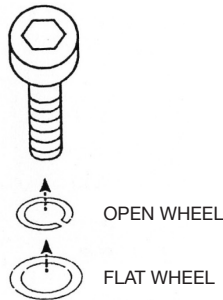
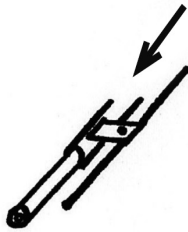
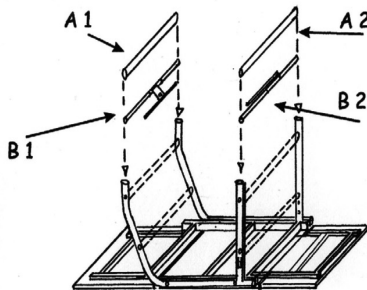
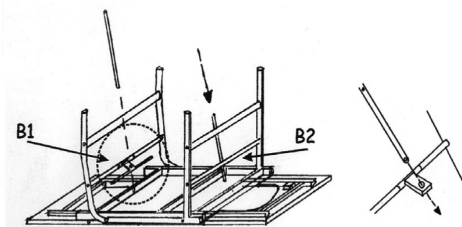
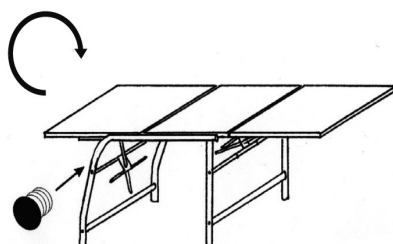
ENGLISH

Img. 1

- Pull out legs and crossbars from the box.
- Place the legs facing the threaded sleeve into the frame of the padded platform by keeping hole with bigger diameter on the external side of the bed.

Img. 2

- Insert the screws (make sure that their wheels are correctly placed into allen screw).

Note: Screws must be firmly fixed at the end of assembly.

Img. 3

- Insert the accessory for blocking the cushions into the chromed crossbar.

Img. 4

- Place the crossbars as shown in the drawing:
B1 blocking with long crossbar
B2 blocking with short crossbar
- Insert the screws (image 2).

Note: for an optimal functioning keep chromed sleeve of b1 and b2 with bigger diameter towards the padded frame.

Img. 5

- Insert the long chromed rod in B1 and the short rod in B2 (Images 5 and 5.1).
Fasten the rods to the cushions with proper screws after checking that the block accessory (Image 3) faces the center of bed.

Img. 6

Note: To place the bed in working position is strongly recommended to ask help of another person as item cod. 27505 is quite heavy.

- Fasten the screws firmly
- Insert 12 plastic head covers with a hammer

Note: the rotation of bed could cause sudden movements of the cushions, please be careful.

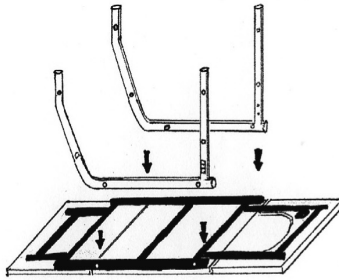
Fig. 1

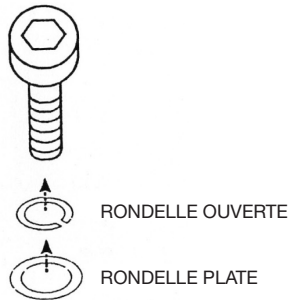
Fig. 2

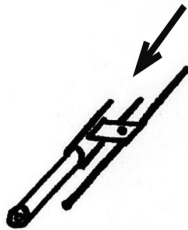
Fig. 3

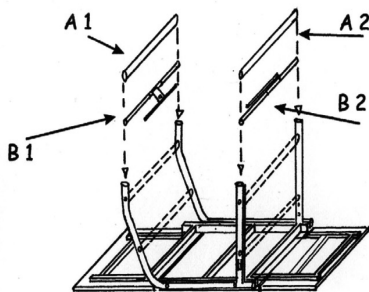
Fig. 4

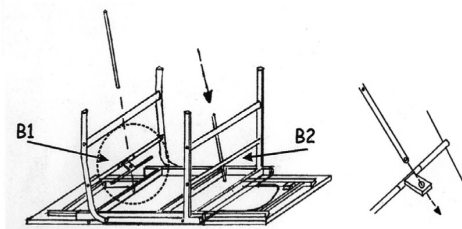
Fig. 5

Fig. 5.1

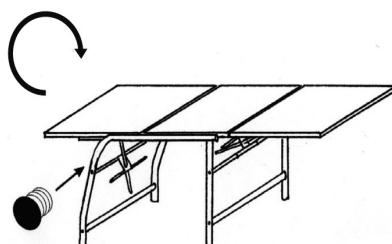
Fig. 6

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