

SIFT[®] SYRINGE

Description

The SIFT[®] syringe includes three parts:

- A polypropylene body with a Luer Lock connector,
- A polypropylene anti-reflux screw plunger,
- A plunger seal in synthetic isoprene.

Volume: 20 ml.
Milliliter graduations.

Class I sterile, medical device, complying with Directive 93/42/EEC.
Single use.
Latex-free.
Individual packaging.
Sterilised by ethylene oxide.

Indications

- The SIFT[®] syringe can be connected to any injection system fitted with a Luer Lock connector.
- In hysterosalpingography or methylene blue test, the SIFT[®] syringe is connected to a RIGISTER[®] or SLIDING RIGISTER[®] device.
- The contrast medium can be injected very regularly due to the presence of the screw thread.

Instructions for use

- To connect a SIFT[®] syringe to a Luer Lock base, bring the two pieces together and rotate the syringe a quarter turn to have a perfect seal.
- To load the syringe or inject contrast material, turn the screw thread plunger.

Reference

1412300 SIFT[®] SYRINGE 20 ml *box of 10 units*

Storage

To store in a dark place.