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| Guide and Statement of manufacturer – Electromagnetic emissions |                      |                                                                                                                                                                                                                           |
|-----------------------------------------------------------------|----------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Emission test                                                   | IEC 60601 Conformity | Electromagnetic environment guidance                                                                                                                                                                                      |
| RF Emission <b>CISPR11</b>                                      | Group 1              | The scale uses RF energy only for its internal function. Therefore, the RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.                                            |
| RF Emission <b>CISPR11</b>                                      | Class B              | The products is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| Harmonic emission <b>IEC 61000-3-2</b>                          | Class A              |                                                                                                                                                                                                                           |
| Voltage fluctuations / flicker emissions <b>IEC 61000-3-3</b>   | Compliant            |                                                                                                                                                                                                                           |

| Guidance and manufacturer's declaration - Electromagnetic Immunity                                         |                                                                                                                                                                                 |                                                                                                                                                                                                                                          |
|------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Immunity test                                                                                              | IEC 60601 Compliance level                                                                                                                                                      | Electromagnetic environment guidance                                                                                                                                                                                                     |
| Electrostatic discharges (ESD) <b>IEC 61000-4-2</b>                                                        | ± 8 kV contact<br>± 2 kV, ± 4 kV,<br>± 8 kV, ± 15 kV air                                                                                                                        | The floors should be made of wood, concrete or ceramic. If the floors are covered in synthetic material, the relative humidity should be at least 30%.                                                                                   |
| Electrical fast transient / burst <b>IEC 61000-4-4</b>                                                     | ± 2kV for power supply lines<br>± 1kV for input/output lines                                                                                                                    | The power supply should be of the type used typically in commercial or hospital environments.                                                                                                                                            |
| Surge <b>IEC 61000-4-5</b>                                                                                 | ± 1kV line(s) to line(s)<br>± 2kV line(s) to earth                                                                                                                              | The power supply should be of the type used typically in commercial or hospital environments.                                                                                                                                            |
| Voltage dips, short interruptions and voltage variations on power supply input lines <b>IEC 61000-4-11</b> | 0% UT for 0,5 cycle<br>0% UT for 1 cycle<br>70% UT (30% dip in UT) for 25s<br>0% UT for 5 s<br><b>Note: UT is the A. C. main voltage prior to application of the test level</b> | The power supply should be of the type used typically in commercial or hospital environments. If the user requires continued operation, it is recommended that the product is powered from an uninterruptible power supply or a battery. |
| Power frequency (50, 60 Hz) Magnetic field <b>IEC 61000-4-8</b>                                            | 30 A/m                                                                                                                                                                          | The product power frequency magnetic fields should be at levels of a typical location in a typical commercial or hospital environment.                                                                                                   |

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| Manufacturer's guide and declaration - Electromagnetic emissions |                                                                           |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                    |
|------------------------------------------------------------------|---------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Immunity test                                                    | IEC 60601 Compliance Level                                                | Electromagnetic environment-guidance                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
| Conducted RF<br>IEC 61000-4-6                                    | 3Vrms 150kHz to 80MHz (for appliances that are not life supporting)       | Portable and mobile RF communications equipment should be used no closer to any part of the product including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.<br>Recommended separation distance<br>$d = 1.2 \sqrt{P}$<br>$d = 1.2 \sqrt{P}$ from 80 MHz to 800 MHz<br>$d = 2.3 \sqrt{P}$ from 800 MHz to 2.5 GHz<br><b>P</b> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <b>d</b> is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>1</sup> , should be less than the compliance level in each frequency range <sup>2</sup> . Interference may occur in the proximity of equipment marked with the following symbol:<br> |
| Radiated RF<br>IEC 61000-4-3                                     | 3 V/m<br>80MHz to 2,7 GHz<br>(for appliances that are not life equipment) |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                    |

<sup>1</sup> From 80 MHz to 800 MHz is applied the higher frequency range.

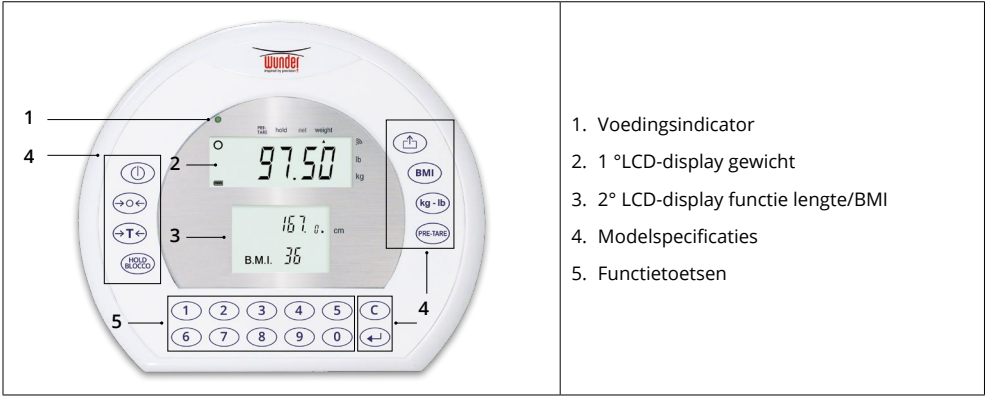
<sup>2</sup> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a) The intensity of the field for fixed transmitters such as base stations for radio, mobile and cordless phones and land radio mobile, amateur radio, radio transmitters in the AM and FM and TV transmitters cannot be predicted theoretically with accuracy. To establish an electromagnetic environment due to fixed RF transmitters, it should consider the electromagnetic survey of the site. If the field strength measured at the place where you use the instrument exceeds the applicable level of compliance of the above, the device should be observed to verify normal operations. If you notice abnormal performance, it may take additional measures such as a different orientation of the device or re-locate it.

b) The field strength over a frequency range of 150 kHz to 80 MHz should be less than 3 V/m.

| Recommended separation distance between Scale and mobile RF communications equipment                                                                                                                                                                                                                                                                                                                                          |                                                               |                            |                             |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------|----------------------------|-----------------------------|
| The scale is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the product can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the product as recommended below, according to the maximum output power of the communications equipment. |                                                               |                            |                             |
| Output power rating of the transmitter (W)                                                                                                                                                                                                                                                                                                                                                                                    | Separation distance according to frequency of transmitter (m) |                            |                             |
|                                                                                                                                                                                                                                                                                                                                                                                                                               | 150 kHz to 80 MHz d=1,2 √P                                    | 80 MHz to 800 MHz d=1,2 √P | 800 MHz to 2,5 GHz d=2,3 √P |
| 0,01                                                                                                                                                                                                                                                                                                                                                                                                                          | 0,12                                                          | 0,12                       | 0,23                        |
| 0,1                                                                                                                                                                                                                                                                                                                                                                                                                           | 0,38                                                          | 0,38                       | 0,73                        |
| 1                                                                                                                                                                                                                                                                                                                                                                                                                             | 1,2                                                           | 1,2                        | 2,3                         |
| 10                                                                                                                                                                                                                                                                                                                                                                                                                            | 3,8                                                           | 3,8                        | 7,3                         |
| 100                                                                                                                                                                                                                                                                                                                                                                                                                           | 12                                                            | 12                         | 23                          |
| For transmitters with maximum rated power output not reported above, the recommended separation distance <b>d</b> in meters (m) can be calculated using the equation applicable to the frequency of the transmitter, where <b>P</b> is the maximum rated power output of the transmitter in Watt (W) according to the manufacturer of the transmitter.                                                                        |                                                               |                            |                             |

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**Wunder Sa.Bi. S.r.l.**