## Automatic Upper Arm Blood Pressure Monitor

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The **Bluetooth®** word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by OMRON HEALTHCARE Co., Ltd. is under license. Other trademarks and trade names are those of their respective owners. App Store and App Store logo are service marks of Apple Inc., registered in the U.S and other countries. Google Play and Google Play logo are trademarks of Google LLC.

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Intellisense

IM2-HEM-7155T-E-05-01/2022  
2870441-5D

### Read instruction manual ① and ② before use.

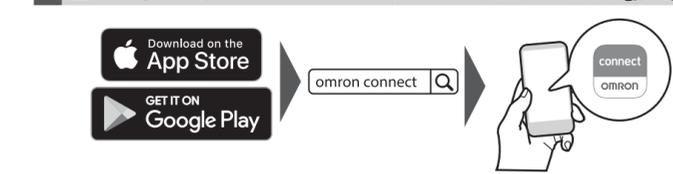
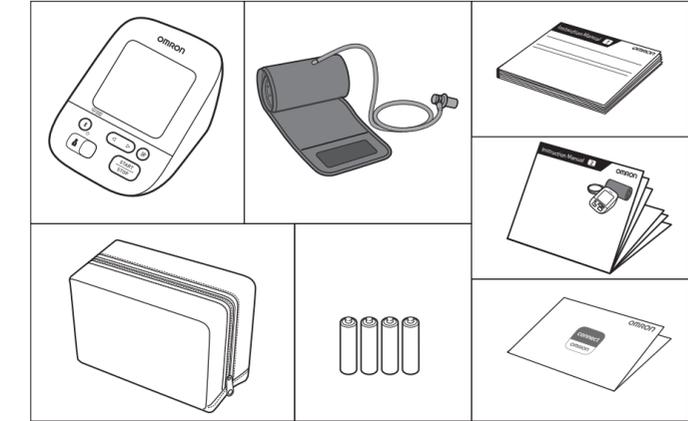
- FR Lire le mode d'emploi ① et ② avant l'utilisation.
- DE Lesen Sie vor der Verwendung die Gebrauchsanweisung ① und ②.
- IT Leggere il manuale di istruzioni ① e ② prima dell'uso.
- ES Lea el manual de instrucciones ① y ② antes del uso.
- NL Lees de gebruiksaanwijzing ① en ② voor gebruik.
- RU Прочтите руководство по эксплуатации ① и ② перед использованием.
- TR Kullanmadan önce, kullanım kılavuzu ① ve ②'yi okuyun.

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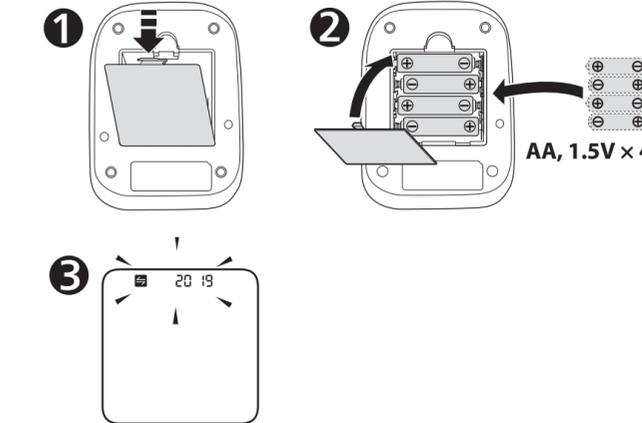
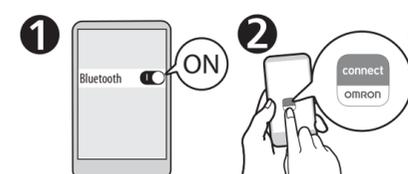
#### The date and time will automatically be set when your monitor is paired with the app.

- FR La date et l'heure seront automatiquement réglées lorsque votre tensièmètre sera jumelé avec l'application.
- DE Wenn Ihr Messgerät mit der App verknüpft ist, werden Datum und Uhrzeit automatisch eingestellt.
- IT La data e l'ora verranno impostate automaticamente una volta che il misuratore verrà associato alla app.
- ES La fecha y la hora se ajustarán automáticamente cuando el monitor se sincronice con la aplicación.
- NL De datum en tijd worden automatisch ingesteld wanneer uw bloeddrukmeter is gekoppeld aan de app.
- RU Дата и время установятся автоматически после синхронизации с приложением.
- TR Ölçüm cihazınız uygulamayla eşleştirildiğinde tarih ve saat otomatik olarak ayarlanır.

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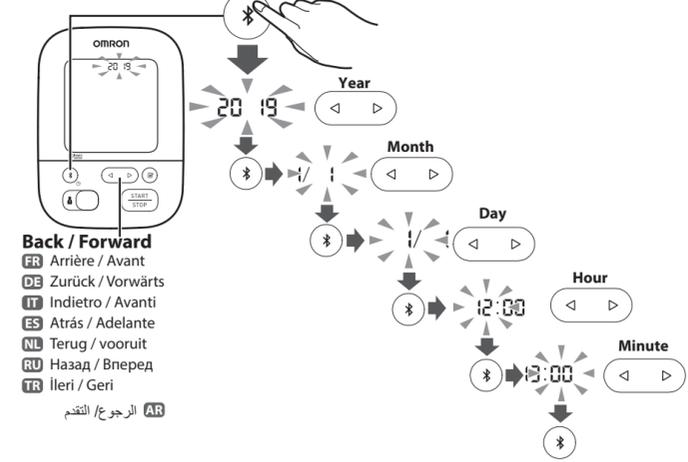
#### If your monitor is paired with your smart device, date and time is set automatically. When you need to set them manually, set year > month > day > hour > minute.

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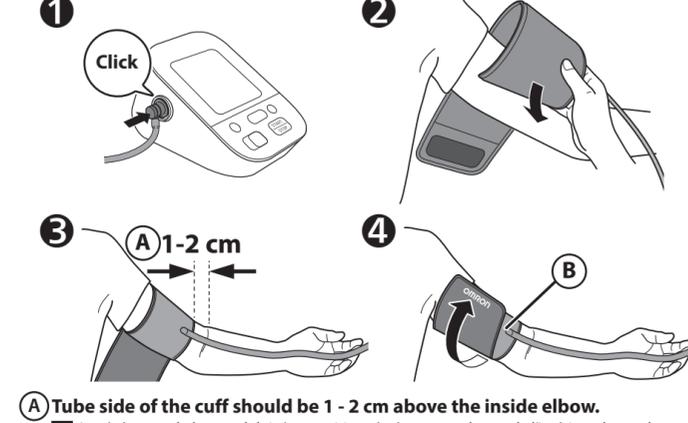
#### A Tube side of the cuff should be 1 - 2 cm above the inside elbow.

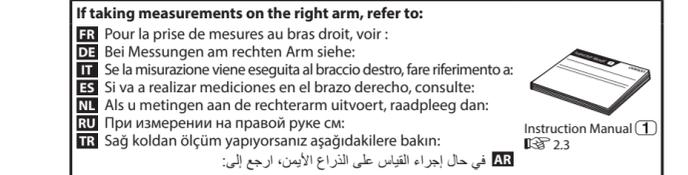
- FR Le côté tuyau du brassard doit être positionné 1 à 2 cm au-dessus de l'intérieur du coude.
- DE Das Manschettenstück mit dem Schlauch muss 1 bis 2 cm oberhalb des Ellbogens liegen.
- IT Il lato del bracciale con il tubo deve trovarsi al di sopra dell'interno del gomito, a una distanza di circa 1 o 2 cm.
- ES El lado del tubo del manguito deberá quedar 1 o 2 cm por encima de la parte interna del codo.
- NL De kant met de slang van de manchet moet 1 - 2 cm boven de binnenkant van de elleboog liggen.
- RU Край манжеты с трубкой должен находиться на 1-2 см выше локтевого сгиба.
- TR Kolluğun boru tarafı, dirsek içinin 1 ila 2 cm üstünde olmalıdır.

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#### B Make sure that air tube is on the inside of your arm and wrap the cuff securely so it can no longer slip round.

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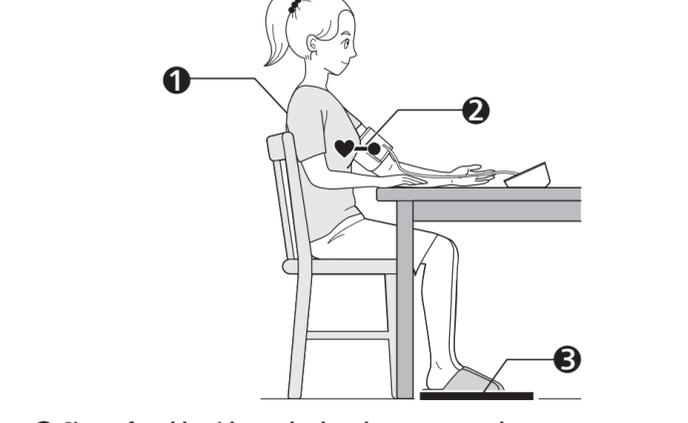
#### 1 Sit comfortably with your back and arm supported.

- FR S'asseoir confortablement de manière à ce que le dos et le bras soient bien soutenus.
- DE Bequem hinsetzen, mit dem Rücken anlehnen und den Arm auf eine Unterlage legen.
- IT Sedere comodamente con la schiena e il braccio ben sostenuti.
- ES Siéntese cómodamente de modo que la espalda y el brazo estén bien apoyados.
- NL Neem een comfortabele zitpositie in, met uw rug en arm ondersteund.
- RU Сядьте удобно, чтобы спина и рука опирались на что-либо.
- TR Sirtınız ve kolunuz desteklenerek rahat bir şekilde oturun.

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#### 2 Place the arm cuff at the same level as your heart.

- FR Le brassard doit se trouver au même niveau que votre cœur.
- DE Die Manschette auf Herzhöhe platzieren.
- IT Posizionare il bracciale allo stesso livello del cuore.
- ES Coloque el manguito al mismo nivel que el corazón.
- NL Plaats de armmanchet op hetzelfde niveau als uw hart.
- RU Манжета должна находиться на уровне сердца.
- TR Kolluğu kalbinizle aynı düzeye getirin.

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#### 3 Keep feet flat, legs uncrossed, remain still and do not talk.

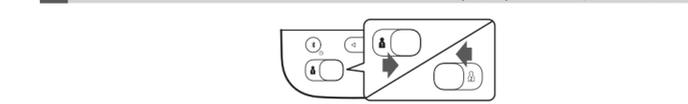
- FR Garder les pieds à plat, les jambes non croisées, ne pas bouger et ne pas parler.
- DE Füße flach auf den Boden stellen, die Beine nicht kreuzen, still halten und nicht sprechen.
- IT Tenere i piedi ben poggiati, non incrociare le gambe, rimanere fermi e non parlare.
- ES Mantenga los pies planos, las piernas sin cruzar, quédese quieto y no hable.
- NL Houd de voeten plat, benen niet gekruist, zit stil en praat niet.
- RU Поставьте ноги ровно, не перекрещивая их, не двигайтесь и не разговаривайте.
- TR Ayaklarınızı düz, bacaklarınızı açık tutun, hareketsiz kalın ve konuşmayın.

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#### Switching user ID enables you to save readings for 2 people.

- FR Changer d'ID Utilisateur permet d'enregistrer les résultats pour 2 personnes.
- DE Mit verschiedenen Benutzer-IDs lassen sich Messwerte für 2 Personen speichern.
- IT Cambiare ID utente permette di registrare i risultati relativi a 2 persone.
- ES Cambiar de ID de usuario le permite guardar las lecturas de 2 personas.
- NL Door het wisselen van gebruikers-ID kunt u metingen voor 2 personen opslaan.
- RU Переключение идентификатора пользователя позволяет сохранять результаты для 2-x людей.
- TR Kullanıcı kimlikleri arasında geçiş yapma, 2 kişi için ölçüm değerlerini kaydetmenizi sağlar.

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**When the [START/STOP] button is pressed, the measurement is taken and saved automatically. Open the app to transfer the reading.**

**FR** Après une pression sur le bouton [START/STOP], la mesure démarre et le résultat est enregistré automatiquement. Ouvrir l'application pour transférer le résultat.  
**DE** Beim Drücken der Taste [START/STOP] erfolgt die Messung. Die Messwerte werden automatisch gespeichert. Öffnen Sie die App, um die Messwerte zu übertragen.  
**IT** Quando si preme il pulsante [START/STOP], la misurazione viene eseguita e salvata automaticamente. Aprire la app per trasferire i risultati.  
**ES** Al pulsar el botón [START/STOP], se obtiene la medición, que se guarda automáticamente. Abra la aplicación para transferir la lectura.  
**NL** Wanneer op de knop [START/STOP] wordt gedrukt, wordt de meting automatisch uitgevoerd en opgeslagen. Open de app om de meting te versturen.  
**RU** После того, как кнопка [START/STOP] нажата, измерение выполняется и сохраняется автоматически. Откройте приложение, чтобы передать результаты.  
**TR** [START/STOP] düğmesine basıldığında ölçüm yapılır ve otomatik olarak kaydedilir. Ölçüm değerini aktarmak için uygulamayı açın.

**The guest mode can be used to take a single measurement for another user. No readings are stored in the memory.**

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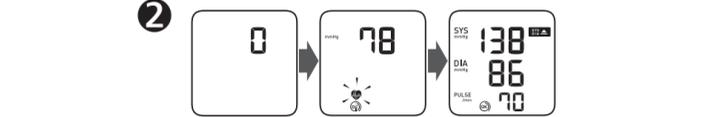
**If your systolic pressure is more than 210 mmHg:**

After the arm cuff starts to inflate, press and hold the [START/STOP] button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure.

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
**1 Appears if "SYS" is 135 mmHg or above and/or "DIA" is 85 mmHg or above.**

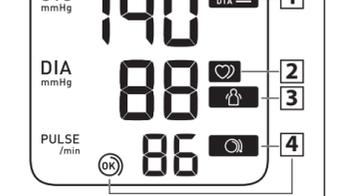**FR** S'affiche si « SYS » est égale ou supérieure à 135 mmHg et/ou « DIA » égale ou supérieure à 85 mmHg\*.  
**DE** Wird angezeigt, wenn „SYS“ 135 mmHg oder mehr beträgt und/oder wenn „DIA“ 85 mmHg\* oder mehr beträgt.  
**IT** Viene visualizzato se la pressione sistolica «SYS» è pari o superiore a 135 mmHg e/o la pressione diastolica «DIA» è pari o superiore a 85 mmHg\*.  
**ES** Aparece si "SYS" es 135 mmHg o superior y/o "DIA" es 85 mm Hg\* o superior.  
**NL** Verschijnt als "SYS" 135 mmHg of hoger is en/of "DIA" 85 mmHg\* of hoger is.  
**RU** Отображается, если «SYS» 135 мм.рт.ст. и выше и/или «DIA» 85 мм.рт.ст.\* и выше.  
**TR** "SYS" 135 mmHg ya da üstünde olduğunda ve/veya "DIA" 85 mmHg\* ya da üstünde olduğunda görünür.  
**AR** تظهر في حال كان "SYS" يبلغ 1٣٥ ملليمترًا زئبقيًا أو أكثر أو "DIA" ٨٥ ملليمترًا زئبقيًا\* أو أكثر.

**2 Appears when an irregular rhythm\*\* is detected during a measurement. If it appears repeatedly, OMRON recommends to consult your physician.**

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**3 Appears when your body moves during a measurement. Remove the arm cuff, wait 2-3 minutes and try again.**

**FR** S'affiche si vous bougez pendant une mesure. Retirez le brassard, attendre 2 à 3minutes et essayer à nouveau.  
**DE** Wird angezeigt, wenn Sie sich während der Messung bewegen. Nehmen Sie die Manschette ab, warten Sie 2–3 Minuten und versuchen Sie es erneut.  
**IT** Viene visualizzato se l'utilizzatore si muove durante la misurazione. Rimuovere il bracciale, attendere 2-3 minuti e riprovare.  
**ES** Aparece cuando se mueve el cuerpo durante una medición. Retire el manguito, espere unos 2 o 3 minutos e inténtelo de nuevo.  
**NL** Verschijnt wanneer uw lichaam tijdens een meting beweegt. Verwijder de armband, wacht 2-3 minuten en probeer het opnieuw.  
**RU** Отображается при движении тела во время измерения. Снимите манжету, подождите 2-3 минуты и попробуйте еще раз.  
**TR** Bir ölçüm esnasında vücudunuz hareket ettiğinde görünür. Kolluğu çıkarın, 2-3 dakika bekleyip tekrar deneyin.  
**AR** تظهر عند تحرك جسمك في أثناء عملية القياس. قم بإزالة الشريط الضاغط للذراع وانتظر من ٢ إلى ٣ دقائق المحاولة مرة أخرى.

**4 Cuff is tight enough.**  
**FR** Le brassard est suffisamment serré.  
**DE** Manschette sitzt ausreichend straff.  
**IT** Il bracciale è stretto a sufficienza.  
**ES** El manguito está lo suficientemente prieto.  
**NL** Manchet zit strak genoeg.  
**RU** Манжета затянута достаточно туго.  
**TR** Kolluk yeterince sikirdi.  
**AR** الشريط الضاغط محكم بما يكفي.

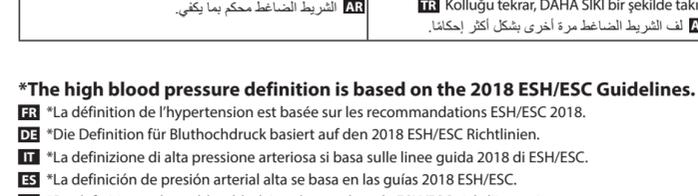
**4 Apply cuff again MORE TIGHTLY.**  
**FR** Poser le brassard en le serrant davantage.  
**DE** Manschette STRAFFER ziehen.  
**IT** Applicare di nuovo il bracciale STRINGENDOLO DI PIÙ.  
**ES** Vuelva a poner el manguito MÁS PRIETO.  
**NL** Breng de manchet STRAKKER aan.  
**RU** Наложите манжету еще раз БОЛЕЕ ТУГО.  
**TR** Kolluğu tekrar, DAHA SIKI bir şekilde takın.  
**AR** قبل استخدام وظائف الذاكرة، حدد معرف المستخدم الخاص بك.

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

**12.2 Average of the Latest 2 or 3 Readings Taken within a 10 Minute Span**

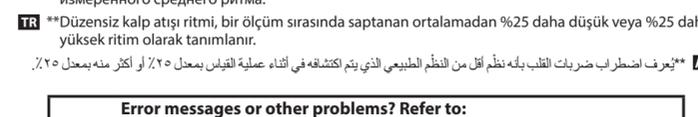**FR** Moyenne des 2 ou 3 dernières mesures prises en l'espace de 10 minutes  
**DE** Mittelwert der letzten 2 oder 3 Messwerte, die in einem 10-Minuten-Zeitraum erfasst wurden  
**IT** Media degli ultimi 2 o 3 risultati ottenuti nell'arco di 10 minuti  
**ES** Promedio de las 2 o 3 últimas lecturas realizadas en un intervalo de 10 minutos  
**NL** Gemiddelde van de laatste 2 of 3 metingen uitgevoerd binnen een tijdsperiode van 10 minuten  
**RU** Среднее значение из последних 2-х или 3-х результатов полученных в течение 10 минут  
**TR** 10 Dakika Aralıkta Alınan Son 2 veya 3 Ölçüm Değerinin Ortalaması  
**AR** متوسط آخر قراءتين أو ٣ قراءات تم الحصول عليها في مدة زمنية تبلغ ١٠ دقائق

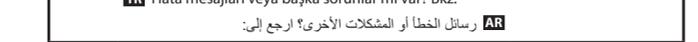
**12.3 Deleting All Readings**  
**FR** Suppression de toutes les mesures  
**DE** Löschen aller Messwerte  
**IT** Cancellazione di tutti i risultati  
**ES** Eliminación de todas las lecturas  
**NL** Alle metingen verwijderen  
**RU** Удаление всех результатов  
**TR** Tüm Ölçüm Değerlerini Silme  
**AR** حذف جميع القراءات

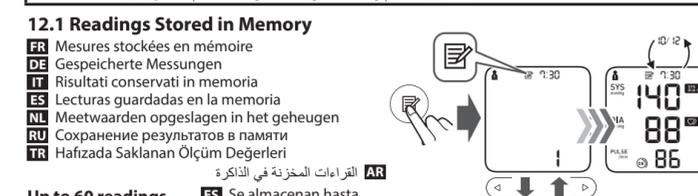**13 Other Settings**  
**FR** Autres réglages  
**DE** Weitere Einstellungen  
**IT** Altre impostazioni  
**ES** Otros ajustes

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