

JELCO

Jelco IntuitIV Safety IV Catheter™ (Straight)

en INSTRUCTIONS FOR USE	nl GEBRUIKSAANWIJZING
fr MODE D'EMPLOI	sv BRUKSANVISNING
de GEBRAUCHSANWEISUNG	da BRUGSANVISNING
it ISTRUZIONI PER L'USO	no BRUKSANVISNING
es INSTRUCCIONES DE USO	fi KÄYTTÖOHJEET
pt INSTRUÇÕES DE UTILIZAÇÃO	el ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

CE 0086


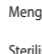
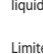
Manufacturer:
Smiths Medical International Ltd.
St. Crispin Way, Haslingden,
Rossendale, Lancashire, BB4 4PW, UK.
Tel: +44 (0)1706 233800

Distributed by:
Smiths Medical ASD, Inc.
201 West Queen St,
Southington, CT 06489, USA.
Tel: +1 860 621 9111

Australian Representative:
Smiths Medical Australasia Pty. Ltd.
61 Brandl Street, Eight Mile Plains,
Brisbane, QLD 4113, Australia.
Tel: +61 (0)7 3340 1300

www.smiths-medical.com **smiths medical**

CCN.004008/015 REV.001 06/12

en English	fr Français	de Deutsch	it Italiano	es Español	pt Português	nl Nederlands	sv Svenska	da Dansk	no Norsk	fi Suomi	el Ελληνικά
 Caution	 Précaution	 Achtung	 Cautela	 Aviso	 Aviso	 Let op	 Försiktighetsåtgärd	 Forsigtig	 Forsiktig	 Varoituss	 Προσοχή
Do not re-use	Ne pas réutiliser	Nicht zur Wiederverwendung	Non riutilizzare	No reutilizar	Não reutilizar	Niet opnieuw gebruiken	Får inte återvändas	Må ikke genanvendes	Til engangsbruk	Ei saa käyttää uudelleen	Μην επαναχρησιμοποιείτε
Do not re-sterilize	Ne pas restériliser	Nicht reesterilisieren	Non risterilizzare	No reesterilizar	Não reesterilizar	Niet opnieuw steriliseren	Får ej omsteriliseras	Må ikke reesteriliseres	Skal ikke reesteriliseres	Äls steriloi uudelleen	Μην επαναποστεριώνετε
 REF	 REF	 REF	 REF	 REF	 REF	 REF	 REF	 REF	 REF	 REF	 REF
Catalogue Number	Numéro de catalogue	Bestellnummer	Numero di catalogo	Número de catálogo	Número de catálogo	Catalogusnummer	Katalognummer	Katalognummer	Katalognummer	Luettelonumero	Αριθμός καταλόγου
 LOT	 LOT	 LOT	 LOT	 LOT	 LOT	 LOT	 LOT	 LOT	 LOT	 LOT	 LOT
Batch code	Numéro de lot	Chargenbezeichnung	Codice del lotto	Código de lote	Código de série	Batchcode	Batchkod	Batchkode	Kode for parti	Erkoodi	Κωδικός παρτίδας
 DATE	 DATE	 DATE	 DATE	 DATE	 DATE	 DATE	 DATE	 DATE	 DATE	 DATE	 DATE
Date of manufacture	Date de fabrication	Herstellungsdatum	Data di fabbricazione	Fecha de fabricación	Data de fabrico	Fabricagedatum	Tillverkningsdatum	Fremstillingsdato	Produksjonsdato	Valmistuspäivä	Αριθμός παρτίδας
 USE BY	 USE BY	 USE BY	 USE BY	 USE BY	 USE BY	 USE BY	 USE BY	 USE BY	 USE BY	 USE BY	 USE BY
Use by	Utiliser avant le	Verwendbar bis	Utilizzare entro	Fecha de caducidad	Utilizar até	Te gebruiken voor	Använd före	Anvendes inden	Brukes innen	Käytettävä ennen	Ημερομηνία λήξης
 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER	 MANUFACTURER
Manufacturer	Fabricant	Hersteller	Fabbricante	Fabricante	Fabricante	Fabrikant	Tillverkare	Fabrikant	Produsent	Valmistaja	Κατασκευαστής
 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN	 DOES NOT CONTAIN
Does not contain Natural Rubber Latex	Ne contient pas de latex de caoutchouc naturel	Enthält keinen Latex aus Naturkautschuk	Non contiene lattice di gomma naturale	No contiene látex de caucho natural	Não contém látex de borracha natural	Bevat geen natuurlijke rubberlatex	Innehåller inte naturligt latexgummi	Indeholder ikke naturgummilætex	Inneholder ikke naturlig gummilætex	Ei sisältä luonnonkumi lateksia	Περιέχει φυσικό ελαστικό λατέξ ή υπάρχει παρούσα φυσικού ελαστικού λατέξ
 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY	 QUANTITY
Quantity	Quantité	Menge	Quantità	Cantidad	Quantidade	Aantal	Kvantitet	Antal	Antall	Määrä	Ποσότητα
 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE	 STERILIZED USING ETHYLENE OXIDE
Sterilized using ethylene oxide	Stérilisé à l'oxyde d'éthylène	Sterilisiert mit Ethylenoxid	Metodo di sterilizzazione con ossido di etilene	Método de esterilización utilizando óxido de etileno	Esterilizado utilizando óxido de etileno	Gesteriliseerd met ethyleenoxide	Steriliserad med etylenoxid	Steriliseret med etylenoxid	Sterilisert med etylenoksid	Steriloitu etyleenoksidilla	Αποστειρώθηκε με χρήση οξειδίου του αιθυλενίου
 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC	 FLUID PATH COMPONENTS NON-PYROGENIC
Fluid path components non-pyrogenic	Éléments de trajet du liquide non pyrogéniques	Komponenten des Flüssigkeitssystems nicht pyrogen	Componenti del percorso del fluido apirogeni	Los componentes de los conductos del fluido son apirógenos	Componentes da via de passagem do fluido apirógenicos	Onderdelen vloeistofpad niet-pyrogeen	Vätskeledarkomponenter är ej pyrogena	Veskebanens komponenter er ikke-pyrogene	Ikke-pyrogene veskebanekomponenter	Nestepolun osat ei pyrogeenisia	Εξαρτήματα διαδρομής υγρού μη πυρετογόνα
 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION	 TEMPERATURE LIMITATION
Temperature Limitation	Limite de température	Temperaturbereich	Limiti di temperatura	Límite de temperatura	Limites de temperatura	Temperatuurlimiet	Temperaturbegränsning	Temperaturbegränsning	Temperaturgrense	Lämpötilarajoitus	Περιορισμός θερμοκρασίας
 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP	 THIS WAY UP
This way up	Haut	Oben	Alto	Este lado hacia arriba	Este lado para cima	Deze kant boven	Denna sida upp	Denne vej op	Denne siden opp	Tämä puoli ylös	Επάνω πλευρά
 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE	 FRAGILE HANDLE WITH CARE
Fragile, handle with care	Fragile : manipuler avec soin	Vorsicht, zerbrechlich	Fragile, maneggiare con cura	Frágil, manipúlese con prudencia	Frágil, manusear com cuidado	Breekbaar, voorzichtig behandelen	Bräcklig, hanteras varsamt	Forsigtig	Skjære saker, håndteres forsiktig	Helposti särkyvä, käsiteltävä varovasti	Ευθραστο, ο χειρισμός του πρέπει να γίνεται με προσοχή
 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED	 DO NOT USE IF PACKAGE IS DAMAGED
Do not use if package is damaged	Ne pas utiliser si l'emballage est endommagé	Inhalt bei beschädigter Verpackung nicht verwenden	Non utilizzare se la confezione è danneggiata	No utilizar si el envase está dañado	Componetes da via de passagem do fluido apirógenicos	Niet gebruiken indien de verpakking beschadigd is	Får inte användas om förpackningen är skadad	Må ikke anvendes, hvis emballagen er beskadiget	Må ikke brukes hvis emballasjen er ødelagt	Ei saa käyttää, jos pakkaus on vahingoittunut	Μην το χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά
 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY	 KEEP DRY
Keep dry	Conserver au sec	Trocken aufbewahren	Tenere all'asciutto	Manténgase en lugar seco	Manter seco	Droog houden	Förvaras torrt	Oppbevares tart	Oppbevares tart	Pidettävä kuivana	Να διατηρείται στεγνό
 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT	 KEEP AWAY FROM SUNLIGHT
Keep away from sunlight	Conserver à l'abri de la lumière du soleil	Vor Sonnenlicht schützen	Tenere al riparo dalla luce solare	Manténgase lejos de la luz solar	Manter afastado da luz solar	Niet blootstellen aan zonlicht	Skyddas för solljus	Må ikke udsættes for sollys	Oppbevares utenfor direkte sollys	Suojattava auringonvalolta	Διατηρείται μακριά από το ηλιακό φως

