160mm

Value & Safe

FFP2 NR

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Respiratory protective particle filtering mask
Earloops - with valve

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REF 20737 / LK-F1710 LOT

Model / Modello / Modèle / Modelo / Modelo LK-F1710


Anhui Lekang Sanitary Materials Co., Ltd.
Qingcao Industrial Park, Tongcheng,
Anqing City, Anhui Province - Made in China

Importato da / Imported by / Importé par /
Importado por / Importado por:
Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

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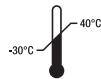GB - CONDITIONS OF USE AND STORAGE

Store in a dry, well-ventilated place away from direct sunlight. Keep away from flammable and polluting materials.

Expiry/Shell life: 3 years.
Materials: 42.8% non-woven fabric; 28.6% hot air cotton; 28.6% melt-blown fabric.

EN149:2001+A1: 2009
FILTRATION EFFICIENCY FOR OIL-FREE MISTS $\geq 95\%$.

1. Hold the mask in your hand with the nose piece facing upwards; fit the mask so that it adheres under your chin; pull on the elastic ear loops and place them behind your ears. Using both hands, adapt the nose piece to the shape of your nose to ensure proper adherence. Place both hands on the mask without moving the mask.
2. Breathe in deeply, hold your breath for a few seconds and check that the mask visibly moves inwards.
3. Breathe out heavily, hold your breath for a few seconds and check that the mask visibly swells outwards.

If air escapes, repeat the operation 2-3 times until the mask complies with the requirements. Check that the face mask is not damaged or seriously contaminated, that the elastic ear loops

are intact with good elasticity, and that the nose piece is intact. Check the exhalation valve and the valve disc to avoid damage or injury. Do not use with oxygen and use only in ventilated areas with 19% oxygen and for one shift only. Any modifications, damage or the presence of hair and beards will decrease efficiency. If you feel unwell, leave the work area and return to areas with fresh air. Avoid use in highly contaminated areas or at risk of explosion.

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