SPATOLA

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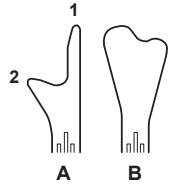
SPATULA

GB Plastic spatula used for taking escocervical and endocervical cytologic samples (Pap Test)
The device should only be used by staff who is qualified, trained and authorized to take Pap Test samples.
Disposable device: any reuse could cause the risk of cross-contaminations and/or failed sterility or cleaning.

- Do not expose to heat;
 - Remove the device from the package immediately before use;
 - Before use visually inspect the device to make sure it is intact.
- Expected use:** The spatula is used for taking a sample of the cells which exfoliate from the escocervix and from the endocervical uterine canal.
- Indications:** it is recommended for taking a sample of the cells which exfoliate from the escocervix and from the endocervical uterine canal

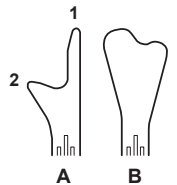
Contraindications: in pregnancy it is not recommended to use end A.

Use: insert the speculum inside the vagina to inspect the uterine cervix; remove the Spatula from the package holding the central part of the stem; visually check that the device is still intact; slowly introduce the tip of point 1 of the end A into the cervical canal until point 2 adheres to the escocervical area; make a complete rotation; slowly extract; rub both areas of the spatula on the glass slide.

If the woman shows a cervical stenotic canal or is pregnant, use end B; ensure end B adheres to the

escocervical area and procede as indicated above. After use, dispose of as normal hospital waste.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located

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