GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

TEST MONOFASE MARIJUANA STRISCE URINA
ONE STEP MARIJUANA TEST STRIP (URINE)
1 ETAPE TEST DE CANNABIS SUR BANDELETTE (URINE)
EINSTUFEN MARIHUANA TESTSTREIFEN (URIN)
PRUEBA DE MARIHUANA EN UN SOLO PASO EN TIRA (ORINA)
TIRA PARA TESTE DE MACONHA EM UM SÓ PASSO (URINA)
ΤΕΣΤ ΜΑΡΙΧΟΥΑΝΑΣ ΤΑΙΝΙΑΣ ΕΝΟΣ ΣΤΑΔΙΟΥ (ΟΥΡΩΝ)
فحص أحادي الطور ماريوانا شرائح بول

Manuale d'uso - User manual

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| Method | Other THC Rapid Test | | | Total Results |
|---------------|-------------------------|----------|----------|---------------|
| | Results | Positive | Negative | |
| | THC One Step Test Strip | Positive | 140 | |
| | Negative | 3 | 157 | 160 |
| Total Results | | 143 | 157 | 300 |
| % Agreement | | 98% | >99% | 99% |

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| Method | GC/MS | | Total Results | |
|---------------|-------------------------|----------|---------------|----------|
| | Results | Positive | | Negative |
| | THC One Step Test Strip | Positive | | 118 |
| | Negative | 4 | 156 | 160 |
| Total Results | | 122 | 178 | 300 |
| % Agreement | | 97% | 88% | 91% |

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| Method | GC/MS | | Total Results | |
|---------------|-------------------------|----------|---------------|----------|
| | Results | Positive | | Negative |
| | THC One Step Test Strip | Positive | | 135 |
| | Negative | 6 | 154 | 160 |
| Total Results | | 141 | 159 | 300 |
| % Agreement | | 96% | 97% | 96% |

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| 11-nor- Δ^9 -THC-9 COOH Concentration (ng/mL) | Percent of Cut-off | n | Visual Result | |
|--|--------------------|----|---------------|----------|
| | | | Negative | Positive |
| 0 | 0 | 30 | 30 | 0 |
| 25 | -50% | 30 | 30 | 0 |
| 37.5 | -25% | 30 | 12 | 18 |
| 50 | Cut-off | 30 | 1 | 29 |
| 62.5 | +25% | 30 | 1 | 29 |
| 75 | +50% | 30 | 0 | 30 |

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Non Cross-Reacting Compounds

| | | | |
|---------------------------|------------------------|-------------------------|-------------------------------|
| 4-Acetamidophenol | Deoxycorticosterone | (+),3,4-Methylenedioxy- | Prednisolone |
| Acetophenetidin | Dextromethorphan | amphetamine | Prednisone |
| N-Acetylprocainamide | Diazepam | (+),3,4-Methylenedioxy- | Procaine |
| Acetylsalicylic acid | Diclofenac | methamphetamine | Promazine |
| Aminopyrine | Diflunisal | Methylphenidate | Promethazine |
| Amitypyline | Digoxin | Methyprylon | D,L-Propranolol |
| Amobarbital | Diphenhydramine | Morphine-3- | D-Propoxyphene |
| Amoxicillin | Doxylamine | β -D-glucuronide | D-Pseudoephedrine |
| Ampicillin | Ecgonine hydrochloride | Nalidixic acid | Quinidine |
| L-Ascorbic acid | Ecgonine methylester | Nalorphine | Quinine |
| D,L-Amphetamine | (-)- Ψ -Ephedrine | Naloxone | Ranitidine |
| L-Amphetamine | Erythromycin | Naltrexone | Salicylic acid |
| Apomorphine | β -Estradiol | Naproxen | Secobarbital |
| Aspartame | Estrone-3-sulfate | Niacinamide | Serotonin (5-Hydroxytyramine) |
| Atropine | Ethyl-p-aminobenzoate | Nifedipine | Sulfamethazine |
| Benzilic acid | Fenoprofen | Norcodein | Sulindac |
| Benzoic acid | Furosemide | Norethindrone | Temazepam |
| Benzoylcegonine | Gentisic acid | D-Norpropoxyphene | Tetracycline |
| Benzphetamine | Hemoglobin | Noscapine | Tetrahydrocortisone |
| Bilirubin | Hydralazine | D,L-Octopamine | 3-acetate |
| (\pm)-Brompheniramine | Hydrochlorothiazide | Oxalic acid | Tetrahydrocortisone |
| Caffeine | Hydrocodone | Oxazepam | 3 (β -D glucuronide) |
| Cannabidiol | Hydrocortisone | Oxolinic acid | Tetrahydrozoline |
| Chloralhydrate | O-Hydroxyhippuric acid | Oxycodone | Thebaine |

| | | | |
|-----------------------|-------------------|-------------------------------|-----------------|
| Chloramphenicol | 3-Hydroxytyramine | Oxymetazoline | Thiamine |
| Chlordiazepoxide | Ibuprofen | p-Hydroxy- methamphetamine | Thioridazine |
| Chlorothiazide | Imipramine | Papaverine | D,L-Thyroxine |
| (±)Chlorpheniramine | Iproniazid | Penicillin-G | Tolbutamine |
| Chlorpromazine | (±)-Isoproterenol | Pentazocine | Triamterene |
| Chlorquine | Isoxsuprine | Pentobarbital | Trifluoperazine |
| Cholesterol | Ketamine | Perphenazine | Trimethoprim |
| Clomipramine | Ketoprofen | Phencyclidine | Trimipramine |
| Clonidine | Labetalol | Phenelzine | Tryptamine |
| Cocaine hydrochloride | Levorphanol | Phenobarbital | D,L-Tryptophan |
| Codeine | Loperamide | Phentermine | Tyramine |
| Cortisone | Maprotiline | L-Phenylephrine | D,L-Tyrosine |
| (-)-Cotinine | Meprobamate | β-Phenylethylamine | Uric acid |
| Creatinine | Methadone | Phenylpropanolamine | Verapamil |
| | Methoxyphenamine | | Zomepirac |

**Indice dei Simboli / Index of symbols / Liste des symboles/ Symbole /
Indice de símbolos / Índice de Símbolos / Πίνακας συμβόλων**

| | | |
|--|---|--|
|  | <p>Leggere e seguire attentamente le istruzioni per l'uso <i>Read instructions carefully</i> Lire et suivre attentivement la notice <i>Gebrauchsanweisung sorgfältig lesen und befolgen</i></p> | <p>Leer y seguir atentamente las instrucciones de uso <i>Ler e seguir atentamente as instruções de uso</i> Διαβάστε και ακολουθήστε προσεκτικά τις οδηγίες χρήσης اقرأ بدقة وحرص تعليمات الاستخدام</p> |
|  | <p>Solo per uso diagnostico in vitro <i>For in vitro diagnostic use only</i> Uniquement pour usage diagnostique in vitro <i>Nur zum Gebrauch für In-vitro-Diagnostik</i></p> | <p>Solo para uso diagnóstico in vitro <i>Apenas para diagnóstico in vitro</i> Μόνο για διαγνωστική χρήση σε δοκιμαστικό σωλήνα فقط للاستخدام التشخيصي المجهرى</p> |
|  | <p>Conservare tra _ e _°C <i>Store between _ and _°C</i> À conserver entre _ et _°C <i>Lagern bei °C</i></p> | <p>Conservar entre _ y _°C <i>Guardar entre _ e _°C</i> Διατηρείται μεταξύ _ και _°C يحفظ بين و درجة مئوية</p> |
|  | <p>Conservare in luogo fresco ed asciutto <i>Keep in a cool, dry place</i> À conserver dans un endroit frais et sec. <i>An einem kühlen und trockenen Ort lagern</i></p> | <p>Conservar en un lugar fresco y seco <i>Armazenar em local fresco e seco</i> Διατηρείται σε δροσερό και στεγνό περιβάλλον يحفظ في مكان بارد وجاف</p> |
|  | <p>Fabricante <i>Manufacturer</i> Fabricant <i>Hersteller</i></p> | <p>Fabricante <i>Fabricante</i> Παραγωγός الشركة المصنعة</p> |
|  | <p>Dispositivo monouso, non riutilizzare <i>Disposable device, do not re-use</i> Dispositif pour usage unique, ne pas réutiliser <i>Für einmaligen Gebrauch, nicht wiederverwenden</i></p> | <p>Dispositivo monouso, no reutilizable <i>Dispositivo descartável, não reutilizar</i> Προϊόν μιας χρήσεως. Μην το χρησιμοποιείται εκ νέου أداة أحادية الاستخدام، لا تستخدم من جديد</p> |
|  | <p>Data di scadenza (vedi scatola / bustina) <i>Expiration date (see box / package)</i> Date d'échéance (voir boîte/sachet) <i>Ablaufdatum (siehe Schachtel / Beutel)</i></p> | <p>Fecha de caducidad (ver caja / sobre) <i>Data de validade (ver caixa/saqueta)</i> Ημερομηνία λήξεως (βλέπε συσκευασία/ σακουλάκι) تاريخ انتهاء الصلاحية (انظر العبوة / المغلف)</p> |
|  | <p>Numero di lotto (vedi scatola / bustina) <i>Lot number (see box / package)</i> Numéro de lot (voir boîte/sachet) <i>Chargennummer (siehe Schachtel / Beutel)</i></p> | <p>Número de lote (ver caja / sobre) <i>Número de lote (ver caixa/saqueta)</i> Αριθμός παρτίδας (βλέπε συσκευασία/ σακουλάκι) رقم الدفعة (انظر العبوة / المغلف)</p> |
|  | <p>Leggere attentamente le istruzioni per l'uso <i>Please read instructions carefully</i> Lire attentivement la notice <i>Gebrauchsanweisung sorgfältig lesen</i></p> | <p>Leer atentamente las instrucciones de uso <i>Ler atentamente as instruções de uso</i> Διαβάστε προσεκτικά τις οδηγίες χρήσης اقرأ بدقة وحرص تعليمات الاستخدام</p> |

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1. Tietz NW. *Textbook of Clinical Chemistry*. W.B. Saunders Company. 1986; 1735
2. Baselt RC. *Disposition of Toxic Drugs and Chemicals in Man*. 2nd Ed. Biomedical Publ., Davis, CA. 1982; 488
3. Hawks RL, CN Chiang. *Urine Testing for Drugs of Abuse*. National Institute for Drug Abuse (NIDA), Research Monograph 73, 1986