

**JELCO**

# Jelco IntuitIV Safety IV Catheter™ (Side Port)

<b>en</b> INSTRUCTIONS FOR USE	<b>nl</b> GEBRUIKSAANWIJZING
<b>fr</b> MODE D'EMPLOI	<b>sv</b> BRUKSANVISNING
<b>de</b> GEBRAUCHSANWEISUNG	<b>da</b> BRUGSANVISNING
<b>it</b> ISTRUZIONI PER L'USO	<b>no</b> BRUKSANVISNING
<b>es</b> INSTRUCCIONES DE USO	<b>fi</b> KÄYTTÖOHJEET
<b>pt</b> INSTRUÇÕES DE UTILIZAÇÃO	<b>el</b> ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

**CE****0086**

Manufacturer:  
**Smiths Medical International Ltd.**  
St. Crispin Way, Haslingden,  
Rossendale, Lancashire, BB4 4PW, UK.  
Tel: +44 (0)1706 233800



















Distributed by:  
**Smiths Medical ASD, Inc.**  
201 West Queen St,  
Southington, CT 06489, USA.  
Tel: +1 860 621 9111

Australian Representative:  
**Smiths Medical Australasia Pty. Ltd.**  
61 Brandl Street, Eight Mile Plains,  
Brisbane, QLD 4113, Australia.  
Tel: +61 (0)7 3340 1300

www.smiths-medical.com

smiths medical

CCN:004008/014 REV:001 06/12

<b>en</b> English	<b>fr</b> Français	<b>de</b> Deutsch	<b>it</b> Italiano	<b>es</b> Español	<b>pt</b> Português	<b>nl</b> Nederlands	<b>sv</b> Svenska	<b>da</b> Dansk	<b>no</b> Norsk	<b>fi</b> Suomi	<b>el</b> Ελληνικά
 <b>Caution</b>	<b>Précaution</b>	<b>Achtung</b>	<b>Cautela</b>	<b>Aviso</b>	<b>Aviso</b>	<b>Let op</b>	<b>Försiktighetsåtgärd</b>	<b>Forsigtig</b>	<b>Forsiktig</b>	<b>Varoitus</b>	<b>Προσοχή</b>
 Do not re-use	Ne pas réutiliser	Nicht zur Wiederverwendung	Non riutilizzare	No reutilizar	Não reutilizar	Niet opnieuw gebruiken	Får inte återanvändas	Må ikke genanvendes	Til engangsbruk	Ei saa käyttää uudelleen	Μην επαναχρησιμοποιείτε
 Do not re-sterilize	Ne pas restériliser	Nicht reesterilisieren	Non risterilizzare	No reesterilizar	Não reesterilizar	Niet opnieuw steriliseren	Får ej omsteriliseras	Må ikke reesteriliseres	Skal ikke reesteriliseres	Älä steriloitua uudelleen	Μην επαναποστεριώνετε
 Catalogue Number	Numéro de catalogue	Bestellnummer	Numero di catalogo	Número de catálogo	Número de catálogo	Catalogusnummer	Katalognummer	Katalognummer	Katalognummer	Luettelonumero	Αριθμός καταλόγου
 Batch code	Numéro de lot	Chargenbezeichnung	Codice del lotto	Código de lote	Código de série	Batchcode	Batchkod	Batchkode	Kode for parti	Eräkoodi	Κωδικός партиδας
 Date of manufacture	Date de fabrication	Herstellungsdatum	Data di fabbricazione	Fecha de fabricación	Data de fabrico	Fabricagedatum	Tillverkningsdatum	Fremstillingsdato	Produksjonsdato	Valmistuspäivä	Αριθμός партиδας
 Use by	Utiliser avant le	Verwendbar bis	Utilizzare entro	Fecha de caducidad	Utilizar até	Te gebruiken voor	Använd före	Anvendes inden	Brukes innen	Käytettävä ennen	Ημερομηνία λήξης
 Manufacturer	Fabricant	Hersteller	Fabbricante	Fabricante	Fabricante	Fabrikant	Tiliverkare	Fabrikant	Produsent	Valmistaja	Κατασκευαστής
 Does not contain Natural Rubber Latex	Ne contient pas de latex de caoutchouc naturel	Enthält keinen Latex aus Naturkautschuk	Non contiene lattice di gomma naturale	No contiene látex de caucho natural	Não contém látex de borracha natural	Bevat geen natuurlijke rubberlatex	Innehåller inte naturligt latexgummi	Indeholder ikke naturgummi latex	Inneholder ikke naturlig gummi latex	Ei sisällä luonnonkumi lateksia	Περιέχει φυσικό ελαστικό λατέξ ή υπάρχει παρουσία φυσικού ελαστικού λατέξ
 Quantity	Quantité	Menge	Quantità	Cantidad	Quantidade	Aantal	Kvantitet	Antal	Antall	Määrä	Ποσότητα
 Sterilized using ethylene oxide	Stérilisé à l'oxyde d'éthylène	Sterilisiert mit Ethylenoxid	Metodo di sterilizzazione con ossido di etilene	Método de esterilización utilizando óxido de etileno	Esterilizado utilizando óxido de etileno	Gesteriliseerd met ethyleenoxide	Steriliserad med etylenoxid	Steriliseret med etylenoxid	Sterilisert med etylenoksid	Steriloitu etyleenioksidilla	Αποστειρώθηκε με χρήση οξειδίου του αιθυλενίου
 Fluid path components non-pyrogenic	Éléments de trajet du liquide non pyrogéniques	Komponenten des Flüssigkeitssystems nicht pyrogen	Componenti del percorso del fluido apirogeni	Los componentes de los conductos del fluido son apirógenos	Componentes da via de passagem do fluido apirogénicos	Onderdelen vloeistofpad niet-pyrogeen	Vätskeledarkomponenter är ej pyrogena	Væskebanens komponenter er ikke-pyrogene	Ikke-pyrogene væskebanekomponenter	Nestepolun osat ei-pyrogeenisia	Εξαρτήματα διέλευσης υγρού μη πυρεταγόνα
 Temperature Limitation	Limite de température	Temperaturbereich	Limiti di temperatura	Limite de temperatura	Limites de temperatura	Temperatuurlimiet	Temperaturbegränsning	Temperaturbegrænsning	Temperaturgrense	Lämpötilarajoitus	Περιορισμός θερμοκρασίας
 This way up	Haut	Oben	Alto	Este lado hacia arriba	Este lado para cima	Deze kant boven	Denna sida upp	Denne vej op	Denne siden opp	Tämä puoli ylös	Επάνω πλευρά
 Fragile, handle with care	Fragile ; manipuler avec soin	Vorsicht, zerbrechlich	Fragile, maneggiare con cura	Frágil, manipúlese con prudencia	Frágil, manusear com cuidado	Breekbaar, voorzichtig behandelen	Bräcklig, hanteras varsamt	Forsigtig	Sjåre saker, håndteres forsiktig	Helpesti särkyvä, käsiteltävä varovasti	Εύθραστο, ο χειρισμός του πρέπει να γίνεται με προσοχή
 Do not use if package is damaged	Ne pas utiliser si l'emballage est endommagé	Inhalt bei beschädigter Verpackung nicht verwenden	Non utilizzare se la confezione è danneggiata	No utilizar si el envase está dañado	Componentes da via de passagem do fluido apirogénicos	Niet gebruiken indien de verpakking beschadigd is	Får inte användas om förpackningen är skadad	Må ikke anvendes, hvis emballagen er beskadiget	Må ikke brukes hvis emballasjen er ødelagt	Ei saa käyttää, jos pakkaus on vahingoittunut	Μην το χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά
 Keep dry	Conserver au sec	Trocken aufbewahren	Tenere all'asciutto	Manténgase en lugar seco	Manter seco	Droog houden	Förvaras torrt	Oppbevares tørt	Oppbevares tørt	Pidetettävä kuivana	Να διατηρείται στεγνό
 Keep away from sunlight	Conserver à l'abri de la lumière du soleil	Vor Sonnenlicht schützen	Tenere al riparo dalla luce solare	Manténgase lejos de la luz solar	Manter afastado da luz solar	Niet blootstellen aan zonlicht	Skyddas för solljus	Må ikke udsættes for sollys	Oppbevares utenfor direkte sollys	Suojattava auringonvalolta	Διατηρείται μακριά από το ηλιακό φως

