**X36-1 LETTINO DA VISITA ELETTRICO**  
**X36-1 ELECTRIC EXAMINATION TABLE**  
**X36-1 DIVAN D'EXAMEN ÉLECTRIQUE**  
**X36-1 CAMILLA DE EXPLORACIÓN ELÉCTRICA**  
**X36-1 ELEKTRISCHE UNTERSUCHUNGSLIEGE**  
**X36-1 MESA DE EXAME ELÉTRICA**  
**X36-1 ΗΛΕΚΤΡΙΚΟ ΕΞΕΤΑΣΤΙΚΟ ΚΡΕΒΑΤΙ**  
**X36-1 ЕЛЕКТРИЧЕСКА МАСА ЗА ПРЕГЛЕД**  
**X36-1 ELEKTRICKÝ VYŠETŘOVACÍ STŮL**  
**X36-1 ELEKTRISK UNDERSØGELSESELEJE**  
**X36-1 ELEKTRILINE LÄBIVAATUSLAUD**

**X36-1 SÄHKÖINEN TUTKIMUSPÖYTÄ**  
**X36-1 ELEKTROMOS VIZSGÁLÓASZTAL**  
**X36-1 ELEKTRINĚ APŽIŮROS KUŠETĚ**  
**X36-1 ELEKTRISKAIS IZMEKLĒŠANAS GALDS**  
**X36-1 ELEKTRISCHE ONDERZOEKSTAFEL**  
**X36-1 ELEKTRYCZNY STÓŁ DO BADAŃ**  
**X36-1 CANAPEA ELECTRICĂ DE EXAMINARE**  
**X36-1 ELEKTRICKÝ VYŠETROVACÍ STŮL**  
**X36-1 ELEKTRIČNA PREGLEDNA MIZA**  
**X36-1 ELEKTRISKT UNDERSÖKNINGSBORD**

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| Guidance and Manufacturer's Declaration – Electromagnetic Immunity                                                                                                  |            |                                                                                                                                                                                                                  |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The V8v8c is intended for use in the following electromagnetic environment.<br>The customer or user of the V8v8c should make sure it is used in such an environment |            |                                                                                                                                                                                                                  |
| Emission test                                                                                                                                                       | Compliance | Electromagnetic Environment Guidance                                                                                                                                                                             |
| RF Emission to CISPR 11                                                                                                                                             | Group 1    | The bed uses RF energy for its internal function                                                                                                                                                                 |
| RF Emission to CISPR 11                                                                                                                                             | Class B    |                                                                                                                                                                                                                  |
| Harmonics according to IEC 61000-3-2                                                                                                                                | Class A    | The bed is intended for use in all types of establishment including residential and similar user that are directly connected to a public supply network that also serves buildings used for residential purposes |
| Voltage fluctuations/ flicker acc. to IEC 61000-3-3                                                                                                                 | Complies   |                                                                                                                                                                                                                  |

| Guidance and Manufacturer's Declaration - Electromagnetic Immunity                                                           |                                                                                                                                                                                                                                                                                                                                                                                                                                              |                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
|------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The V8v8c is intended for use within healthcare facility environment.<br>Users or customers must use bed in this environment |                                                                                                                                                                                                                                                                                                                                                                                                                                              |                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
| Immunity test                                                                                                                | IEC 60601-1-2<br>EN 60601-1-2<br>Test level                                                                                                                                                                                                                                                                                                                                                                                                  | Electromagnetic Environment Guidance                                                                                                                                                                                                                                                                                                                                                                                                                |
| Electrostatic discharge (ESD).....<br>IEC 61000-4-2<br>/EN 61000-4-2                                                         | Contact discharge: +8kV<br><br>Air discharge: +2kV, +4kV, +8kV, +15kV                                                                                                                                                                                                                                                                                                                                                                        | Contact discharge: +8kV<br><br>Air discharge: +2kV, +4kV, +8kV, +15kV<br><br>Floor should be made of wood and concrete or be tiled with ceramic tiles. If the floor is covered with synthetic flooring material, the relative air humidity must be at least 30%<br><br>Can be used when higher ESD levels are present                                                                                                                               |
| Electrical<br>fast transient/burst IEC 61000-4-4<br>/EN 61000-4-4                                                            | +2kV Power supply line<br><br>+1kV Input/Output lines                                                                                                                                                                                                                                                                                                                                                                                        | +2kV Power supply line<br><br>The quality of the supply voltage should be equivalent to that of a typical business or hospital environment.                                                                                                                                                                                                                                                                                                         |
| Surge<br>IEC 61000-4-5<br>/EN 61000-4-5                                                                                      | Line - line: +0.5kV, +1.0kV<br><br>Line - ground: +0.5kV, +1.0kV, +2.0kV                                                                                                                                                                                                                                                                                                                                                                     | Line - line: +0.5kV, +1.0kV<br><br>Line - ground: +0.5kV, +1.0kV, +2.0kV<br><br>The quality of the supply voltage should be equivalent to that of a typical business or hospital environment.                                                                                                                                                                                                                                                       |
| Power Frequency Magnetic Fields<br>IEC 61000-4-8<br>/EN 61000-4-8                                                            | 30 A/m<br><br>50Hz/60Hz                                                                                                                                                                                                                                                                                                                                                                                                                      | 30 A/m<br><br>50Hz/60Hz<br><br>Magnetic fields with a network frequency should be equivalent to those to be found in a typical business or hospital environment.                                                                                                                                                                                                                                                                                    |
| Voltage Dips IEC 61000-4- 11<br>/EN 61000-4-11                                                                               | 0% UT: 0.5 cycles<br><br>Phase: 00, 450, 900, 1350, 1800, 2250, 2700, 3150                                                                                                                                                                                                                                                                                                                                                                   | 0% UT: 0.5 cycles<br><br>Phase: 00, 450, 900, 1350, 1800, 2250,                                                                                                                                                                                                                                                                                                                                                                                     |
|                                                                                                                              | 0% UT: 1 cycle<br><br>70% UT: 25/30 cycles Single phase: 00<br><br>0% UT ; 300 cycle<br><br>The quality of the supply voltage should be equivalent to that of a typical business or hospital environment.<br><br>If the person using the bed requires that the bed functions must continue despite any interruptions in the energy supply, it is recommended that the bed be connected to an uninterruptible electricity supply or a battery | 0% UT: 1 cycle<br><br>70% UT: 25/30<br><br>cycles Single phase: 00<br><br>0% UT ; 300 cycle<br><br>The quality of the supply voltage should be equivalent to that of a typical business or hospital environment.<br><br>If the person using the bed requires that the bed functions must continue despite any interruptions in the energy supply, it is recommended that the bed be connected to an uninterruptible electricity supply or a battery |

|                                                                    |                       |                                                                                                                                                                                                                                                                                                                                                                               |
|--------------------------------------------------------------------|-----------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Short interruptions IEC 61000-4-11<br>/EN 61000-4-11               | 0% UT: 250/300 cycles | 0% UT: 250/300 cycles<br><br>The quality of the supply voltage should be equivalent to that of a typical business or hospital environment.<br><br>If the person using the bed requires that the bed functions must continue despite any interruptions in the energy supply, it is recommended that the bed be connected to an uninterruptible electricity supply or a battery |
| Note: UT is the AC Power voltage before applying in the test level |                       |                                                                                                                                                                                                                                                                                                                                                                               |

| Guidance and Manufacturer's Declaration - Electromagnetic Immunity                                                                                                                                                                                                                                                 |                                              |                        |              |                                                                |            |                   |              |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------|------------------------|--------------|----------------------------------------------------------------|------------|-------------------|--------------|
| The V8v8c is intended for use in an electromagnetic environment that used wireless RF transceiver. Electromagnetic interference can be prevented by maintaining the minimum distance between RF transceiver and V8v8c depend on the maximum output and frequency of the communication device as recommended below. |                                              |                        |              |                                                                |            |                   |              |
| Immunity test                                                                                                                                                                                                                                                                                                      | IEC 60601-1-2<br>/EN 60601-1-2<br>Test level | Compliance level (V/m) | Band d (MHz) | Service                                                        | Modulation | Maximum power (W) | Distance (m) |
| Enclosure port to RF wireless communication equipment IEC 61000-4-3<br>/EN 61000-4-3                                                                                                                                                                                                                               | 27V/m<br>385MHz                              | 27                     | 380-390      | TETRA 400                                                      |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>450MHz                              | 28                     | 430-470      | GMRS 460, FRS 460                                              |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 9V/m<br>710MHz                               | 9                      | 704-787      | LTE Band 13, 17                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 9V/m<br>745MHz                               |                        |              |                                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 9V/m<br>780MHz                               |                        |              |                                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>810MHz                              | 28                     | 800-960      | GSM 800/900, TETRA 800, iDen 820, CDMA 850, LTE Band 5         |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>870MHz                              |                        |              |                                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>930MHz                              |                        |              |                                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>1720MHz                             | 28                     | 1700-1990    | GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1,3, 4,25; UMT S |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>1845MHz                             |                        |              |                                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>1970MHz                             |                        |              |                                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 28V/m<br>2450MHz                             | 28                     | 2400-2570    | Bluetooth, WLAN, 802.11 b/g/n RFID 2450, LTE band 7            |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 9V/m<br>5240MHz                              | 9                      | 5100-5800    | WLAN 802.11 a/n                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 9V/m<br>5500MHz                              |                        |              |                                                                |            |                   |              |
|                                                                                                                                                                                                                                                                                                                    | 9V/m<br>5785MHz                              |                        |              |                                                                |            |                   |              |

| Guidance and Manufacturer's Declaration - Electromagnetic Immunity                                                                                                   |                                                         |                                                         |                                                                                                                                                                                                                                                                                                                                                         |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------|---------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The V8v8c is intended for use in the following electromagnetic environment.<br>The customer or user of the V8v8c should make sure it is used in such an environment. |                                                         |                                                         |                                                                                                                                                                                                                                                                                                                                                         |
| Immunity test                                                                                                                                                        | IEC 60601-1-2<br>/EN 60601-1-2<br>Test level            | Compliance level                                        | Electromagnetic Environment - Guidance                                                                                                                                                                                                                                                                                                                  |
| RF Conductivity<br>IEC 61000-4-6<br>EN 61000-4-6                                                                                                                     | 3V<br>0.15MHz-80MHz                                     | 3V<br>0.15MHz-80MHz                                     | Field intensity from the fixed RF transmitter determined by a field investigation of the magnetic field must be lower than the compliance level of the respective frequency band.<br><br>Near devices with the following symbol, interference could occur.<br><br> |
|                                                                                                                                                                      | 6V in ISM bands between 0.15MHz- 80MHz<br>80%AM at 1kHz | 6V in ISM bands between 0.15MHz- 80MHz<br>80%AM at 1kHz |                                                                                                                                                                                                                                                                                                                                                         |
| Radioactive RF electromagnetic field<br>IEC 61000-4-3<br>/EN 61000-4-3                                                                                               | 3V/M<br>80MHZ-2.7GHz<br>80%AM at 1kHz                   | 3V/M<br>80MHZ-2.7GHz<br>80%AM at 1kHz                   |                                                                                                                                                                                                                                                                                                                                                         |

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5. Doživiljenjske storitve proizvajalca: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD.