

SILENT KNIGHT PILL CRUSHING DEVICE



Product Code	25872
Unit of sale	1 pc
Minimum order	1
Type	No medical device
Class	

UK-REP	Not required
CH-REP	Not required
EAN/UPC	3700446000653
PARAF	977811001

Description

SILENT KNIGHT HOSPITAL PILL CRUSHING DEVICE

Professional, silent crusher for hospital to effortlessly and easily reduce pills in fine powder in few-seconds.
Pouches avoid any cross contamination and eliminate loss of medication.
Silent Knight is supplied with 50 pouches.

Ergonomic handle for effortless use.
Place for 50 pouches.
Light and sturdy aluminium base and handle.
Perfect stability.

- step 1: place pills into pill crusher pouch. Lift handle and place pouch down between the crusher plates to initiate crushing.
- step 2: press handle down to crush pills.
- step 3: press the handle down to activate grinding movement. Press handle down several times to grind pills into fine powder.
- step 4: remove the pouch.