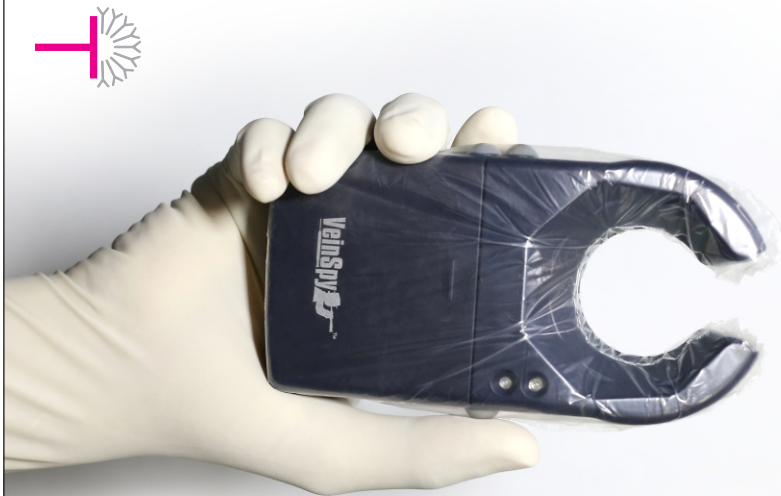


Ver. 2.0/08A21/V/DPS		DISPOSABLE PLASTIC SLEEVES		DISPOSABLE PLASTIC SLEEVES							
DISPOSABLE PLASTIC SLEEVES FOR 											
CE		<table><tr><th>Cat No.</th><th>Content</th><th>Quantity</th></tr><tr><td>825DS5000050</td><td>Disposable Plastic Sleeves</td><td>50nos.</td></tr></table>		Cat No.	Content	Quantity	825DS5000050	Disposable Plastic Sleeves	50nos.	MD	
Cat No.	Content	Quantity									
825DS5000050	Disposable Plastic Sleeves	50nos.									
How to Cover VeinSpy™ with Disposable Plastic Sleeves  1. Insert the VeinSpy™ device into the disposable plastic sleeve.  2. Pull the disposable plastic sleeve right upto the bottom of the device.  3. Fold the disposable plastic sleeve & close at the bottom part of the VeinSpy™  4. Use the covered VeinSpy™ for finding vein. Note : 1. Discard the plastic sleeve after use. 2. For each patient use a fresh disposable plastic sleeve.		DISPOSABLE PLASTIC SLEEVES									
Tulip Diagnostics (P) Ltd. Dukle Heaven, Block III, 301 & 401, St. Inez, Taleigao Road, Taleigao, Tiswadi, North Goa, Goa, 403 002, INDIA. Tel.: 7066031234/35/36 E-mail: sales@tulipgroup.com Website: www.tulipgroup.com EC REP CMC Medical Devices & Drugs S.L., C/Horacio Lengo N°18, CP 29006, Malaga, Spain. Tel.: +34951214054 Fax: +34952330100 Website: www.cmcmedicaldevices.com											