DOMICILE - HOMECARE

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MKT-H-NO-CPF-C

2025-02-07

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EN - Instructions for installation and use

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HERDEGEN SAS,
46bis avenue du Maine
75015 PARIS – France
qualite@hacare.com

Copyright 2025 par HERDEGEN SAS– ALL RIGHTS RESERVED

In the interests of continuous improvement, HERDEGEN SAS reserves the right to change any detail or specification in this document without prior notice.

The information contained in this document is confidential. It may only be shared with a third party with the prior written consent of HERDEGEN SAS. This document may not be reproduced or communicated, in full or in part, in any way or by any means, whether technical or electronic, by photocopying, recording, or using any information or data extraction systems.

You can obtain a new copy of this document from HERDEGEN SAS by contacting your regional sales representative or directly contacting HERDEGEN SAS.

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FOREWORD

You have just purchased a HACARE product and we are pleased to have you as one of our clients. We also thank you for placing your trust in our company and our products.

Your trust assures us that the HACARE company continues to meet your specific requirements and needs. That's why we always strive to innovate, providing you with quality solutions and products that improve the comfort and quality of patient care while making the caregiver's daily tasks easier.

HACARE products have gained the confidence of many professionals and institutions due to their functional and aesthetic qualities, and are now considered to be a solid benchmark. Simplicity, reliability and safety are the hallmarks of our electrical healthcare cradles.

You've chosen HACARE, and we're committed to satisfying your needs.

However, you are responsible for the proper use of the cradles. That's why we invite you to familiarize yourself with the technical specifications, and the instructions for use and maintenance of the equipment by reading through this manual.

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4. NETTOYAGE, DESINFECTION, MAINTENANCE

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EN - Please read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place. If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

1. SAFETY INDICATION, ABNORMAL USES, DANGERS

- Always consult with your physician or supplier to determine proper adjustment and use of the device.
- Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature).
- Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- Before seating make sure that you have correctly placed the seatplate on the chair.
- Check carefully all parts of this item (screws, tubes, tips, grips)..
- Check regularly if all screws are tightened.
- Use only as a commode chair and in proper way : do not stand on the chair.
- Always observe the weight limit on the labelling of your product.
- No modification must be made on this device. Use only original

parts to replace the broken or defect one and let this be done by the technical staff.

- After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before using. Some chairs are already mounted.
- DO NOT hang anything on the Chair.
- ALWAYS test to see that the chair and attachments are properly and securely locked in place before using.
- In case you didn't use your chair for a long time, let it checked by the technical staff, before using.
- Do not store outdoor, the chair must be used at home.
- Do not install or remove the pan while the patient is seating on the chair: High risk of sex injuries.
- Be careful when removing the armrests to avoid trapping your fingers.
- Check the chair before using, in case of doubt ask your physician or provider.
- Use only the chair on a flat, solid and dry ground.
- It is forbidden to seat on the armrests.
- Do not lean out of the chair to avoid losing balance.

- Any serious incident that has occurred in relation to the device shall be reported to the manufacturer (qualite@hacare.com) and the competent authority of the member state in which the user and/or the patient is established.

- The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts. Likewise, the warranty excludes parts or working parts subject to natural wear and tear

2. INTENDED USE, INDICATIONS AND CONTRAINDICATIONS

Commode chairs can be used at home or in nursing homes to assist a person who has difficulty getting to the bathroom by bringing the toilet into the bedroom. Standard, foldable or on castors, they shall be use only indoor.

Indications:

Used in the home when the patient is too debilitated to reach the bathroom.

Contraindications:

- cognitive disorder, loss of balance, siting impairment.
- Patient weight is greater than the product's supported weight.
- Other contraindications to be determined by the prescribing physician based on the patient's condition

3. INSTALLATION

Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Installing/removing the bucket

Best up XXL:

1- from the left or right side under the seat, install the bucket onto the metal supports

2- lift up the seatplate and remove the bucket from the top, see photo [B].

New Open, New Club:

Lift up the seatplate and remove the bucket from the top, see photo [B].

Close the bucket with the lid before transporting it.

Mounting the seatplate

The seatplate can be easily removed for cleaning or for replacement

as it is only connected to the frame with 2 clips see photo [A].

To remove the seatplate, take out the clips from the frame by lifting them up, see photo [B].

To mount the seatplate, clip the two clips on the back pipe of the metal frame, see photo [C].

By adding the cover on the hole, the commode chair can be used as a regular chair.

Adjusting the height (only for New Open Adjustable)

Take out the clip from the tubes using both thumbs, then adjust the interior pipe to the desired height and lock again with the clip. Before using the chair, check that the metal pin of the clip has gone through both pipes and exits from the outer pipe.

Rental: This product can be rented to different patients. A thorough maintenance, cleaning and disinfection must be performed before the product is given to a new patient in accordance with the following steps:

1-Visual inspection: ensure that the product has not been damaged and check for any defect.

2-Functional inspection: ensure that the product functions properly and repair it if needed.

3-Cleaning and disinfection: this product can be cleaned and disinfected with mild, chlorine-free detergents such as Survanios (Anios laboratories).

Do not use any abrasive cleaners or high-pressure cleaners that could damage the product.

Use only appropriate tools to perform the maintenance, cleaning and disinfection.

The user's manual is an integral part of the product and must be handed over to every new user.

4. CLEANING AND DISINFECTION

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth.

Urine is very corrosive and attacks steel tubes very quickly. It is therefore important to clean and dry the chair daily.

Do not use any abrasive cleaners or high-pressure cleaners that could damage the product

5. DISPOSAL

Dispose of this product in accordance with your regional waste disposal regulations.

6. TECHNICAL SPECIFICATIONS

See table page 6. Biocompatible. The warranty duration of the product and its various elements is specified in the commercial conditions.

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