



GIMA

PROFESSIONAL MEDICAL PRODUCTS

OROLOGIO DIGITALE PER INFERMIERI

DIGITAL NURSE WATCH

MONTRE D'INFIRMIÈRE DIGITALE

RELOJ DIGITAL PARA ENFERMERA

Manuale d'uso - User manual
Manuel de l'utilisateur - Guía de uso

ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.

ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

GIMA 20480



Nox Industry Co., Limited
603-5 NO.17-2 Wanghai Road, Siming District
Xiamen, China, P.C. 361008
info@noxtime.com
Made in China



REF NS888



Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com





Functions

6 digits display: Month, Date, Week day, Hour, Minute, Second
 Stop watch
 12/24 hours
 Alarm
 EL background light



Instructions

Set Time & Date, week day

3 x Press B, second flashes, set time by pressing D, C, by pressing C to confirm & return normal display. Please set time by following steps.

3xPress B (Example: **13:02:23**)

Second (23) flashes, press D to increase the seconds, press C back to 00, press C to confirm.

Minute (02) flashes, press D to increase the numbers, press C to confirm.

Hour (13) flashes, press D to increase the numbers, press C to confirm.

Date flashes, press D to increase the numbers, press C to confirm

Month flashes, press D to increase the numbers, press C to confirm

Week flashes, press D move the flash to the correct week day, then press B to confirm all setups, and back to normal display.



Stop watch

Under normal display, 1 x press B
enter stopwatch mode,
1 x press D Start/stop to counting time
1 x press C clear data.
1 x press B, back to normal display.

12/24 Hours

Under normal display , 3 x press B,
enter time setup mode,
2 x press C enter 12/24 switch mode,
press D until 12/24 displays.
1 x press B back to normal display.

EL background light

1 x press A, EL light on 3 seconds.

Set Alarm

Under normal display.
2 x press B, enter alarm mode, hour flashes, press D to adjust the hour, then press C,
minute flashes, press D to adjust minute, press B back to normal display.

Alarm On/Off

Press C(don't remove), press D at the same time to set ON/OFF of alarm. Alarm
keeps on 1 minute.

Hourly bell

Press C(don't remove),press B to set hourly bell on/off. When hourly bell is on, all
week day display in on.
Otherwise means hourly bell is off.

Battery replacement

On the back of the watch, there is a notch to open the housing and get to the battery.
Lift the rear with a sharp and thin instrument. Remove the metal cover and the battery
by sliding it on the side.
Insert the new battery complying with the correct polarity with the “+” facing up.
Replace the metal cover and close the case.

Weight: 45g

Battery: 3V Lithium Battery CR2025



Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

