


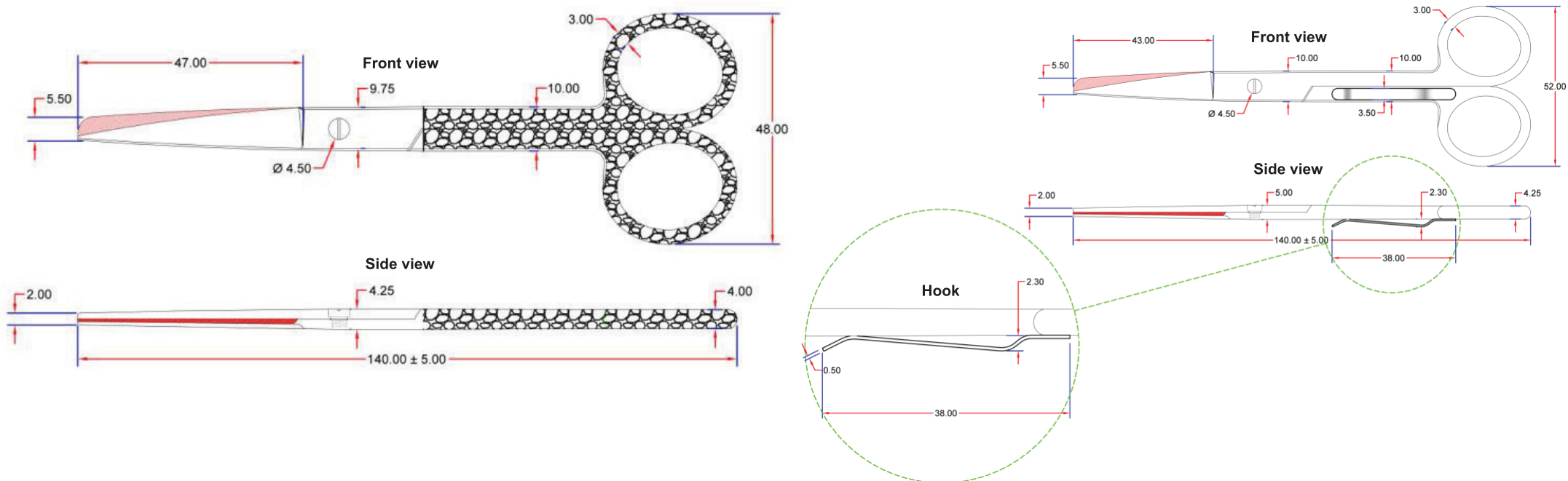

ENGLISH

SCISSORS FOR MEDICAL MATERIALS

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Devices

Scissors for medical materials (Non Sterile, Stainless steel).

Product Specification

The product is made from high-grade stainless steel for the production of medical instruments according to ISO and ASTM standards.

Pre-cleaning

Remove gross deposits using a laparotomy sponge and sterile water during the procedure.

Manual cleaning

To minimize the risk to personnel undertaking manual cleaning, splashing and the creation of spray must be avoided at all times. Staff carrying out manual cleaning should wear PPE at all times.

Devices should be:

- 1) Cleaned using a non-linting cloth, impregnated with the appropriate detergent solution, followed by a clean, damp, non-linting cloth; and then
- 2) Dried using another clean, non-linting cloth. Alcohol-impregnated wipes may be used following a manual cleaning process.

Detergents: Use specific detergents: do not use common dish detergent.

Disinfection

GIMA powder / paper coated instruments are not autoclavable but they can be cleaned/disinfected with disinfectants like Endo Star (high level instruments disinfectant) or with any other better disinfectant/sterilizing solution.

Point of use handling

May only be used for the purpose of which they are designed, by adequately qualified personal only.

Storage & maintenance

Products should be stored away from direct sunlight and water, in a safe, dry and cool environment.

Warning

Don't use the rusty instrument. Wash the hands with anti-bacterial soap or use approved hand sanitizer before use.
For professional use only - disinfect before use.

Conclusion

This Medical Devices classified as class-I as per Medical Device Regulation (MDR) 2017/745.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

	IT Codice prodotto GB Product code FR Code produit DE Erzeugniscode ES Código producto PT Código produto		IT Fabbricante GB Manufacturer FR Fabricant ES Fabricante DE Hersteller PT Fabricante		IT Non sterile GB Non-sterile FR Pas stérile ES No estéril PT Não estéril DE Nicht steril		IT Dispositivo medico GB Medical Device FR Dispositif médical DE Medizinprodukt ES Producto sanitario PT Dispositivo médico
	IT Leggere le istruzioni per l'uso GB Consult instructions for use FR Consulter les instructions d'utilisation ES Consultar las instrucciones de uso PT Consulte as instruções de uso DE Gebrauchsanweisung beachten		IT Data di fabbricazione GB Date of manufacture FR Date de fabrication ES Fecha de fabricación PT Data de fabrico DE Herstellungsdatum		IT Numero di lotto GB Lot number FR Numéro de lot ES Número de lote PT Número de lote DE Chargennummer		IT Conservare in luogo fresco ed asciutto GB Keep in a cool, dry place FR À conserver dans un endroit frais et sec ES Conservar en un lugar fresco y seco PT Armazenar em local fresco e seco DE An einem kühlen und trockenen Ort lagern
	IT Dispositivo medico conforme al regolamento (UE) 2017/745 GB Medical Device compliant with Regulation (EU) 2017/745 PT Dispositivo médico em conformidade com a regulamento (UE) 2017/745 FR Dispositif médical conforme au règlement (UE) 2017/745 DE Medizinprodukt im Sinne der Verordnung (EU) 2017/745 ES Producto sanitario conforme con el reglamento (UE) 2017/745				IT - Rappresentante autorizzato nel Regno Unito GB - Authorized Representative in the UK FR - Représentant autorisé au Royaume-Uni ES - Representante autorizado en el Reino Unido PT - Representante autorizado no Reino Unido DE - Autorisierter Vertreter im Vereinigten Königreich		IT Conservare al riparo dalla luce solare GB Keep away from sunlight FR À conserver à l'abri de la lumière du soleil ES Conservar al amparo de la luz solar PT Guardar ao abrigo da luz solar DE Vor Sonneneinstrahlung geschützt lagern
	IT - Identificatore univoco del dispositivo GB - Unique device identifier PT - Identificador exclusivo do dispositivo FR - Identifiant unique de l'appareil DE - Unique Device Identifier (Eindeutige Kennung des Geräts) ES - Identificador de dispositivo único				IT - Rappresentante autorizzato in Svizzera GB - Authorized Representative in Swiss FR - Représentant autorisé au Suisse ES - Representante autorizado en Suízo PT - Representante autorizado no Suíço DE - Autorisierter Vertreter im Schweizer		IT Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB Caution: read instructions (warnings) carefully FR Attention: lisez attentivement les instructions (avertissements) ES Precaución: lea las instrucciones (advertencias) cuidadosamente PT Cuidado: leia as instruções (avisos) cuidadosamente DE Achtung: Anweisungen (Warnungen) sorgfältig lesen

 **20520 - 20521 - 20522 - 20529**
20531 - 20532 - 20533 - 20534