



1



2



3



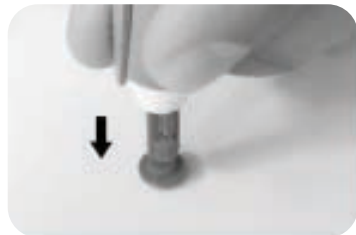
4



5



6



7



8

ENGLISH
AUTO LANCET DEVICE
The name of the device :

Lancing Device

Model

LDC, LDE, LDF

Principle and Intended use:

Assisting the lancet for peripheral blood sampling from the fingertip or the paw for blood glucose testing or other testing utilising a small amount of blood, it is the non-invasive device and does not come into contact with blood, it only makes a brief contact with the skin surface for a few seconds. The lancing device is non-sterile.

Caution:

1. The lancing device is for single patient use. Do not share it with anyone including other family members! Do not use on multiple patients!
2. Do not reuse lancet, always use a new, sterile lancet for testing.
3. Use caution whenever the lancet needle is exposed, especially when removing the used lancet from the device and when disposing the used lancet.
4. A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Shelf-life:

5 years or 5,000 times, whichever comes first.

Use Instructions:

The device can be reused, but you must keep the Lancing Device clean. Disinfect the lancing Device Cap with an alcohol pad or cotton balls before use. You should also wash your hands in warm, soapy water, rinse well and dry completely. Warming fingers can increase the blood flow.

1. The device can be reuse. Keep the Lancing Device clean, Disinfection of lancing Device Cap with alcohol pad or cotton balls before use. Twist off the Lancing Device Cap.
2. Insert a new sterile lancet into the Lancet Holder. Push down firmly until it is fully seated.
3. Twist off the Lancet Protective Cap and save the Cap for later use. Replace the Lancing Device Cap.
4. Set the lancing level by twisting the Adjustable Tip until the number lines up with the Arrow. To select the best depth:
 - 1-2 for soft or thin skin
 - 3 for average skin
 - 4-5 for thick or calloused skin
5. Slowly pull back the Cocking Control. You will hear a click, indicating that the Lancing Device is now loaded and ready to use.
6. Hold the Lancing Device firmly against the side-tip of the finger. Press the Release Button. Set the Lancing Device aside and proceed with your blood glucose test according to your blood glucose meter's directions.
7. After use, twist off the Lancing Device Cap. Push the exposed tip of the lancet into its Protective Cap.
8. Slide the Lancet Ejector forward and disposing the used lancet in an approved container. Discard the used lancet according to your country's safety regulations. Replace the Lancet Device Cap.

The Performance characteristics of the devices:













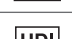

The product consists of the main body, Popup mechanism and adjustment set.

Transportation and Storage:


Common conveyance are available in the transport, please prevent heavy Pressure, rainwater and snow and collision; the packed products shall be stored in a well ventilated room free of any poisonous, harmful and corrosive substances.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

	Authorized representative in the European Union
	Medical Device compliant with Regulation (EU) 2017/745
	Date of manufacture
	Manufacturer
	Product code
	Lot number
	Temperature limit
	Humidity limit
	Keep in a cool, dry place
	Consult instructions for use
	Caution: read instructions (warnings) carefully
	Medical Device
	Unique device identifier
	Imported by

GIMA 23919

 Shandong Lianfa Medical Plastic Products Co.,Ltd
No.1 Shuangshan Sanjian Road, Zhangqiu ,
Jinan City, 250200, Shandong
PEOPLE'S REPUBLIC OF CHINA
Made in China

REF LDE

EU **REP** Linkfar Healthcare GmbH
Niederrheinstraße 71, 40474 Düsseldorf,
Germany



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

