



GIMA

PROFESSIONAL MEDICAL PRODUCTS

BILANCIA QUADRANTE GRANDE GIMA ANALOGICA
GIMA BIG DIAL ANALOGUE SCALE
BALANCE ANALOGIQUE À LARGE CADRAN GIMA
MECHANISCHE PERSONENWAAGE MIT GROSSANZEIGE GIMA
BALANZA PANTALLA GRANDE GIMA ANALÓGICA
BALANÇA ANALÓGICA GIMA COM MARCADOR GRANDE
ΑΝΑΛΟΓΙΚΗ ΖΥΓΑΡΙΑ ΜΕΓΑΛΗ ΟΘΟΝΗ GIMA
ميزان بشاشة عرض كبيرة GIMA تناظرية

Fabbricante / Manufacturer / Hersteller / Fabricant /
Fabricante / Κατασκευαστής : عناصر

Momert Zrt.

2400 Dunaújváros - HUNGARY, Papírgyári út 12-14

info@momert.hu - www.momert.hu

Made in Hungary

GIMA 27239

REF 5210

Distribuito da/Distributed by/Distribué par/ Vertrieben von
Distribuido por/Distribuido por/ Διανέμετ όπα ια/عزوت

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com



FEATURES

The GIMA big dial analogue scale is a device for measuring the weight of the human body. The wide measurement range makes it ideal for any kind of use.

Technical Features

Maximum capacity: 160 kg

Graduation: 1 kg

Accuracy: 0-50 kg ± 1 kg, 50-100 kg $\pm 1,5$ kg, 100-160 kg ± 2 kg

Dimensions: 320 x 70 x 430 mm

Weight of unit: 3 kg

Not certified product class III

PRESCRIPTIONS



Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING



Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled. They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

OPERATION

- Position the scale on the floor in a stable position (ex.: not on a rug)
- Verify that the pointer is set to zero
- Put and remove a weight from the scale a few times and verify that the pointer returns to its position exactly on the zero, if necessary reposition the pointer using the adjustment wheel.
- Step onto the scale and verify that both feet are positioned in a manner that will distribute your weight evenly over the entire scale platform, read weight.
- Try to stand as still as possible for a precise measurement
- To obtain valid results for statistical purposes, weigh yourself every day possibly at the same time. It is best to weigh yourself without clothes and in the morning as soon as you wake up.



Caution: the scale can hold a maximum weight of 160 kg, do not overload the scale or it could get damaged!

IMPORTANT SAFETY INSTRUCTIONS

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance
- Use the product only for the intended purpose specified on the present manual.
- The unit is suitable for domestic use only. It is not designed for commercial operation.
- The product is a fragile measuring device. Prevent it from crashing or falling.

CLEANING AND MAINTENANCE

The device was designed to last a lifetime. The surface of the unit may only be cleaned using a wet cloth. Do not use any detergents or hard objects as they may damage the units' surface! Never clean the unit under running water, do not rinse it, and do not submerge in water!

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Simboli - Symbols - Symboles - Symbole - Simbolos - Símbolos - Συμβολα - حرف

	IT Codice prodotto GB Product code FR Code produit DE Erzeugniscode ES Código producto PT Código produto GR Κωδικός προϊόντος SA كود المنتج
	IT Numero di lotto GB Lot number FR Numéro de lot DE Chargennummer ES Número de lote PT Número de lote GR Αριθμός παρτίδας SA رقم الدفعة
	IT Conservare in luogo fresco ed asciutto GB Keep in a cool, dry place FR À conserver dans un endroit frais et sec ES Conservar en un lugar fresco y seco DE An einem kühlen und trockenen Ort lagern PT Armazenar em local fresco e seco GR Διατηρείται σε δροσερό και στεγνό περιβάλλον SA يحفظ في مكان بارد وجاف
	IT Conservare al riparo dalla luce solare GB Keep away from sunlight FR À conserver à l'abri de la lumière du soleil DE Vor Sonneneinstrahlung geschützt lagern ES Conservar al amparo de la luz solar PT Guardar ao abrigo da luz solar GR Κρατήστε το μακριά από ηλιακή ακτινοβολία SA يحفظ بعيداً عن أشعة الشمس
	IT Leggere le istruzioni per l'uso GB Consult instructions for use FR Consulter les instructions d'utilisation DE Gebrauchsanweisung beachten ES Consultar las instrucciones de uso PT Consulte as instruções de uso GR Διαβάστε προσεκτικά τις οδηγίες χρήσης SA اقرأ بدقة وحرص تعليمات الاستخدام
	IT Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB Caution: read instructions (warnings) carefully FR Attention: lisez attentivement les instructions (avertissements) DE Achtung: Anweisungen (Warnungen) sorgfältig lesen ES Precaución: lea las instrucciones (advertencias) cuidadosamente PT Cuidado: leia as instruções (avisos) cuidadosamente GR Προσοχή: διαβάστε προσεκτικά τις οδηγίες (ενοστάσεις) SA الحذر: قراءة التعليمات (التحذيرات) بعناية