

seca 210

Deutsch	Seite 3
Englisch	Page 5
Français	Page 7
Español	Página 9
Italiano	Pagina 11
Dansk	Side 13
Svenska	Sida 15
Norsk	Side 17
Suomi	Sivu 19
Nederlands	Pagina 21
Português	Página 23
Ελληνικά	Σελίδα 25
日本語	ページ 27
Polski	Strona 29
Română	Pagina 31

Intended use

The mechanical baby length measuring device supports physicians in decision-making regarding diagnosis or therapy, based on the patient's length or growth.

Safety precautions

⚠ WARNING!

Injury from falling

Measuring devices for babies are usually placed on elevated work surfaces. If the baby were to fall from the work surface, it could suffer serious irreversible or fatal injuries.

- ▶ Make sure that the device is standing or lying on firm and even ground.
- ▶ Never leave a baby unsupervised.

Markings on the device and on the type plate


Text/ symbol	Meaning
UK CA xxxx	Device complies with United Kingdom directives xxxx: Notified Body for Medical Devices of the United Kingdom
 UK REP	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom

Text/ symbol	Meaning
 CH REP	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schönmatt Str. 2 CH-4153 REINACH
MD	Medical device in accordance with Regulation (EU) 2017/745

Operation

Measurement method:

1. Lie the baby on the measuring mat. The baby's head must lie against the head stop (1) of the measuring mat.
2. Using your right hand, push the foot stop (2) in the direction of the baby's feet. The left hand holds the baby in position.
3. Stretch the baby's legs by pressing its knees carefully and gently down to the mat with the left hand.
4. With your right hand, rest the foot stop (2) at right-angles to the soles of the baby's feet.

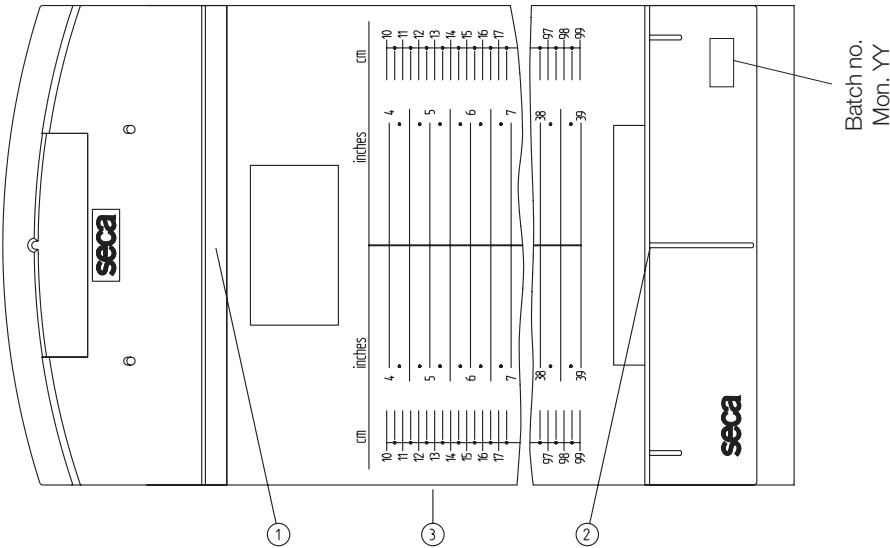

5. Read off body length at the marking on the measuring mat (3).

If the above steps are followed, accuracy greater than ± 5 mm is achieved.

Hygiene treatment

⚠ WARNING!

Risk of infection

- ▶ Subject the device to a hygiene treatment at regular intervals.

⚠ NOTICE!

Damage to device

- ▶ Use alcohol-based disinfectant (e.g. 70 % ethanol) exclusively.
- ▶ Follow the instructions for use of the disinfectant.
- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Clean the device:
 - If required, moisten a soft cloth with a mild soapy solution and wipe the device over with it.
- ▶ Disinfect the device:
 - Moisten a soft cloth with disinfectant and wipe the device over with it.

Before and after each measurement	As required
• Surface of measuring mat • Head stop • Foot stop	All other parts of measuring mat

Technical data

Measurement range: 100 - 990 mm /
4 - 39 inches

Graduations: 5 mm / 1/4 inch
Accuracy: greater than ± 5 mm

Dimensions
(W x H x D): 300 mm x 140 mm x
1340 mm

Weight: approx. 500 g

Temperature range: +10 °C to +40 °C

Medical device in
accordance with

Regulation

(EU) 2017/745: Class I with
measuring function

Disposal

This device consists of high-quality raw materials which can be recycled. Dispose of the device in accordance with the local laws and regulations applicable in your country.

Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under

warranty can be honoured only if the complete original packaging was used for any transport and the merchandise secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so. We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

Declaration of conformity CE

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: www.seca.com.

Medical Measuring Systems and Scales since 1840

seca gmbh & co. kg
Hammer Steindamm 3–25
22089 Hamburg
Germany

T +49 40 20 00 00 0
F +49 40 20 00 00 50
E info@seca.com

seca operates worldwide
with headquarters in
Germany and branches in:
seca france
seca united kingdom
seca north america
seca schweiz
seca zhong guo
seca nihon
seca mexico

seca austria
seca polska
seca middle east
seca suomi
seca américa latina
seca asia pacific
seca danmark
seca benelux
seca lietuva

and with exclusive
partners in more than
110 countries.

All contact data at
seca.com