GIMA

PROFESSIONAL MEDICAL PRODUCTS

PLICOMETRO FAT-1

SKINFOLD CALIPER FAT-1

PLICOMÈTRE FAT-1

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ADIPÔMETRO FAT-1

HAUTFALTENDICKENMESSER FAT-1

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Fig. 4

TECHNICAL CHARACTERISTICS

The skinfold caliper is an instrument created by Tanner and Whitehouse in 1955 which permits to assess the nutritional state of a subject thanks to the measurement of the thickness of the skinfolds. It must therefore meet two precise and now universally adopted characteristics, which are:

- 1) the two arms of the instrument must exercise on the skinfold a pressure of approximately 10 g./mm²
- 2) the grasp on the skinfold must correspond to an area of approximately 6 x 11 millimetres.

This instrument, in compliance with the above technical characteristics, thanks to the reading scale in mm on the upper edge and indicated by the black arrow, permits to measure with an approximation of +/- 2 mm the skinfold.

MODE OF USE

In order to make the measurement it is necessary to lift a skinfold between the fingers and measure its thickness with the skinfold caliper.

For the tricep measurement (Fig. 1) lift the skinfold in the back area of the arm, exactly halfway between the acromion and the proximity end of the olecranon. For the subscapula measurement (Fig. 2) lift the skinfold in the region immediately under one of the two scapulas. For the bicep measurement (Fig. 3) lift the skinfold at mid-length of the bicep. For the suprailiac measurement (Fig. 4) lift the skinfold found on the side.

Once completed this operation, proceed with the measurement using the instrument and immediately you can read the dimension in mm of the skinfold thickness.

The following page shows some values of skinfolds divided by sex, adults, athletes and children.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ADULTI - ADULTS - ADULTES - ERWACHSENE - ADULTOS - ADULTOS - ΕΝΗΛΙΚΕΣ - ADULTI - DOROŚLI LUDZIE - VUXNA - DOSPELÍ - FELNÖTTEK - ODRASLI - VOLWASSENEN - VOKSNE - ВЪЗРАСТНИ - PIEAUGUŠIE - اشخاص بالغين
 Pliche sottocutanea* sottoscapolare e tricipitale per maschi e femmine - Subscapula and tricep skinfold* for males and females - Plis sous-cutané* sous-scapulaire et tricipital pour hommes et femmes - Subkutane*, subscapulare und tricipitale Hautfalte bei Männern und Frauen - Pliegues subcutáneos* subescapular y tricipital para hombres y mujeres - Pregas de gordura subcutânea* (tricipital e sub-escapular) para homens e mulheres - Υποδόριες* δερματοπτυχές υποπλάτιου και τρικέφαλου για άνδρες και γυναίκες - Pliuri subcutanate* subscapulare și triceps pentru bărbați și femei - Faldy podskórne* podłopatkowe i trójglowe u mężczyzn i kobiet - Subkutana veck* subscapular och triceps för män och kvinnor - Podkožné záhyby* podlopatkové a tricepsové pre mužov a ženy - Subcutan redők* lapocka alatti és tricepsz férfiak és nők számára - Subkutani nabori* subscapular i triceps za muškarce i žene - Subcutane plooiën* subscapulier en triceps voor mannen en vrouwen - Subkutane folder* subscapular og triceps for mænd og kvinder - Подкожни гънки* субскапуларни и трицепс за мъже и жени - Zemādas krokas* zemlāpstiņas un tricepss vīriešiem un sievietēm - طبقات الكتف تحت الجلد وعضلة الذراع الثلاثية للذكور والإناث

PLICHE - SKINFOLD PLICHE - HAUTFALTE PLIEGUES - PREGAS ΔΕΡΜΑΤΟΠΤΥΧΕΣ - PLE- ACHES - PRZYJEMNIE - PLEACHER - PLEACHY - PLÁCSOK - PLICES - PLEACHEN - PLEACHER - ПЛЕЧИ - PLEACHES - طيّات (mm)	MASCHI - MALES - HOMMES - MÄNNERN HOMBRES - HOMENS - ΑΝΔΡΕΣ - MASCULIN - MEŹCZYŻNA - MANLIG - MUŽ - FÉRFI - MUŠKI - MANNELIJK - HAN - МЪЖКИ - VRIETIS - ذكر							FEMMINE - FEMALES - FEMMES - FRAUEN MUJERES - MULHERES - ΓΥΝΑΙΚΕΣ - FEMEIE - KOBIETA - KVINNA - ŽENA - NÖI - ŽENA - VROUWELIJK - KVINDE - ЖЕНСКИ ПОЛ - SIEVIETE - أنثى						
	Età in anni - Ages in years - Âge en années - Alter in Jahren - Edad en años - Idade (anos) - Ηλικία σε έτη - Vårsta în ani - Wiek w latach - Alder i år - Életkor években - Dob u godinama - Leef tijd in jaren - Alder i år - Възраст в години - Vecums gados - العمر بالسنتين													
	20-29	30-39	40-49	50-59	60-69	70-79	>80	20-29	30-39	40-49	50-59	60-69	70-79	>80
Sotto-scapolare - Subscapula Sous-cutané - Subscapulare Subescapular - Sub-escapular Υποπλάτιου - Subscapular Podłopatkowe - Subscapularis - Subscapularis - Subscapu- laris - Subscapularis - Sub- scapularis - Subscapularis - Subscapularis - Subscapularis تحت الكتف	15,7	18,5	20,2	19,9	20,5	18,9	16,4	16,3	20,2	22,7	22,6	21,8	19,3	14,4

Tricipitale - Tricep - Tricipital Tricipitale - Tricipital - Tricipital Τρικέφαλου - Triceps - Triceps - Triceps - Triceps - Triceps - Triceps - Triceps - Triceps - Трицепс - Triceps عضلة الذراع الثلاثية	12,7	13,6	13,9	14,1	14,9	14,0	12,7	21,1	23,9	25,6	25,9	25,4	22,9	19,0
---	------	------	------	------	------	------	------	------	------	------	------	------	------	------

BAMBINI - CHILDREN - ENFANTS - KINDER - NIÑOS - CRIANÇAS - ΠΑΙΔΙΑ - COPII - DZIECI - BARN - DETI - GYERMEKEK - DJECA - KINDEREN - BØRN - ДЕЦА - BÈRNI - أطفال

Pliche sottocutanee sottoscapolare, tricipitale e bicipitale per maschi e femmine - Subscapula, tricep and bicep skinfold** for males and females - Plis sous-cutané** sous-scapulaire, tricipital et bicipital pour hommes et femmes - Subkutane**, subscapulare, tricipitale and bicipital Hautfalte bei Männern und Frauen - Pliegues subcutáneas** subescapular, tricipital y bicipital para hombres y mujeres - Pregas de gordura** subcutânea (tricipital, bicipital e sub-escapular) para homens e mulheres - Υποδόριες** δερματοπτυχές υποπλάτιου, τρικέφαλου και δικέφαλος για άνδρες και γυναίκες -**

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