



GIMA

PROFESSIONAL MEDICAL PRODUCTS

SPONDA PER LETTO A FORMA DI N N-SHAPED BEDRAIL BARRIÈRE DE LIT EN FORME DE N BARANDILLA DE CAMA EN FORMA DE N GRADE DE CAMA DE FORMA N N-GEFORMTES SEITENTEIL FÜR BETTEN ȘINĂ PENTRU PAT ÎN FORMĂ DE N N-FORMAT SÄNGRÄCKE BARIERKA ŁÓŻKOWA W KSZTAŁCIE LITERY N ΚΑΓΚΕΛΑ ΚΡΕΒΑΤΙΟΥ ΣΧΗΜΑΤΟΣ-N

È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Il est nécessaire de signaler tout accident grave survenu et lié au dispositif médical que nous avons livré au fabricant et à l'autorité compétente de l'état membre où on a le siège social.

Es necesario informar al fabricante y a la autoridad competente del Estado miembro en el que se encuentra la sede sobre cualquier incidente grave que haya ocurrido en relación con el producto sanitario que le hemos suministrado.

É necessário notificar ao fabricante e às autoridades competentes do Estado-membro onde ele está sediado qualquer acidente grave verificado em relação ao dispositivo médico fornecido por nós.

Jeder schwere Unfall im Zusammenhang mit dem von uns gelieferten medizinischen Gerät muss unbedingt dem Hersteller und der zuständigen Behörde des Mitgliedsstaats, in dem das Gerät verwendet wird, gemeldet werden.

Orice accident grav produs, privitor la dispozitivul medical fabricat de firma noastră, trebuie semnalat producătorului și autorității competente în statul membru pe teritoriul căruia își are sediul utilizatorul.

Det är nödvändigt att meddela tillverkaren och de behöriga myndigheterna i den berörda medlemsstaten, om alla allvarliga olyckor som inträffat i samband med den medicintekniska utrustning som levererats av oss.

Należy poinformować producenta i kompetentne władze danego Kraju członkowskiego o każdym poważnym wypadku związanych z wyrobem medycznym naszej produkcji.

Σε περίπτωση που διαπιστώσετε οποιοδήποτε σοβαρό περιστατικό σε σχέση με την ιατρική συσκευή που σας παρέχουμε θα πρέπει να το αναφέρετε στον κατασκευαστή και στην αρμόδια αρχή του κράτους μέλους στο οποίο βρίσκεστε.

GIMA 27697



Mingguang Longway Technology Co., Ltd.
No. 59 Lingji Rd., Industrial Park, Mingguang, Anhui, China
Made in China



LW10206



Lotus NL B.V. Koningin Julianaplein 10, 1 e Verd, 2595AA, The Hague, Netherlands



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com





Weight Capacity: 75kg

INTENDED USE:

The bedside rail is an accessory designed to prevent an accidental fall of a person from any bed for hospital or home care.

Suitable for most metal beds & with easy control system, so it does not have to be completely removed for transporting the patient to and from the bed. The bedside rails are designed to fit in most beds thanks to its simple frame, that can be placed on the headboard and the footboard of the bed.

CAUTION (!):

- In case of damage or malfunction of your product **DOT NOT** use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- LONGWAY Medical declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- LONGWAY Medical reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$ Deviation for the Dimensions of the Products.

GENERAL SAFETY WARNINGS:

- ⚠ • If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- ⚠ • Keep away the product from heat sources.
- ⚠ • Save this document for future reference.
- ⚠ • Be careful when children are nearby and do not allow the children to play with the product.
- ⚠ • Do not exceed the maximum payload.
- ⚠ • Special care is always required where there are moving parts that can cause limb entrapment and injury.
- ⚠ • Do not attempt to lift the product by any removable parts.
- ⚠ • Do not use the product seat for a different purpose from the intended one.
- ⚠ • The bedside rails are not intended to carry the full weight of an individual.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

BEFORE USE:

- Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- Check, if the product is properly assembled and all screws are secure and well screwed on.
- Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

MAINTENANCE, CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use.

The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:


1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. **DO NOT** leave the product wet.
6. Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product
7. If you need to disinfect the product, use a common, mild disinfectant
8. Protect the item from scratches, cuts and punctures.
9. Lubrication: the product has designed for a minimal maintenance; however, check every six months the lubrication on the moving & mechanical parts, this would ensure an optimal duration.

ASSEMBLING:

NOTE: The illustrations for the assembly are indicative to make the assembly easier. May will be differences from reality.

To assemble the Rails on the Bed:

1. Remove the screws with the fixing knob at the base of the clamps, adjust the remaining fixing screws on the terminals according to the structure of the bed, insert the sideboard with both clamps on the bed frame, replace the screws with the fixing knobs previously removed, adjust the position of the side rail with respect to the bed and equally tighten the fixing knobs on the terminals (as shown in the figure).

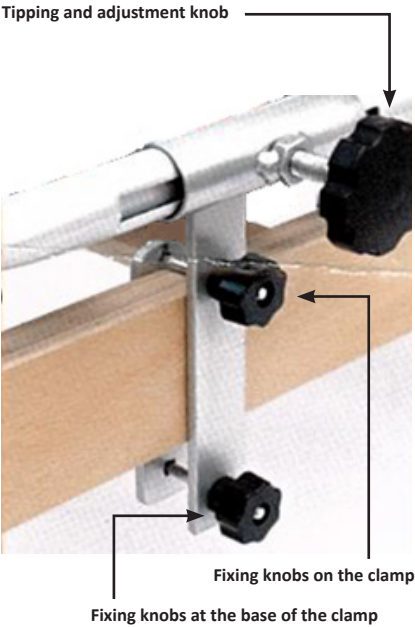

2. The tilting and adjustment knobs allow, by loosening them, the movement of the side rail towards the patient's feet or head for the range allowed by the slots. At the end of the adjustment the knobs must be tightened again.

3. If the tilting and adjustment knobs are loosened until the slots are completely free, the side can be folded 180 °, allowing any type of maneuver on the bed or on the patient. At the end of the maneuvers the side must be put back in position and the knobs must be tightened again.

WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

SYMBOL LEGEND

	Manufacturer
	Date of manufacture
	Product code
	Lot number
	Authorized representative in the European community
	Caution: read instructions (warnings) carefully
	Consult instructions for use
	Medical Device compliant with Regulation (EU) 2017/745
	Medical Device
	Unique device identifier
	Imported by
	Keep in a cool, dry place
	Keep away from sunlight

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.