

ENGLISH

HEEL CUPS AND TRAVEL CUSHION IN SILICONE HOLLOW FIBER

WARNINGS

Before performing any operation, visually check the integrity of the device. If the product shows any anomalies, immediately contact the point of sale where the purchase was made. Erregi sas assumes no responsibility for improper use of the device. In the event of a serious incident related to the use of the device, please report the incident to the manufacturer and the competent authority of the member state where the user and/or patient is located.

DESCRIPTION

Medical device intended for the prevention of pressure injuries, made of cotton fabric with siliconized hollow fiber filling. Selected models feature adjustable Velcro closures for optimal positioning. The combination of high elastic recovery capacity and optimal breathability ensures effective prophylactic action against pressure-related skin lesions.

INTENDED USE

Pressure ulcer prevention for adult patients aimed at reducing pressure peaks, minimizing shear and friction forces, and promoting air circulation.

INDICATIONS AND PRECAUTIONS FOR USE

Medical device indicated for the prevention of pressure ulcers, to be used following clinical evaluation by a specialist physician. Place the device on the affected body area and secure it with Velcro fastenings where present. It is recommended to regularly monitor the affected area, perform frequent patient repositioning, and maintain proper hygiene. For this purpose, consult protocols and guidelines for pressure ulcer prevention. Always use the device while wearing a protective garment. This device is not fire-resistant.

CONTRAINDICATIONS

No contraindications have been detected, except in cases of proven sensitivity to the component materials.

COMPOSITION

COVER: Cotton

FILLING: Siliconized hollow fiber

WASHING INSTRUCTIONS

			
WASH AT MAX 40°C	DO NOT BLEACH	DO NOT IRON	DO NOT TUMBLEDRY
	USE ANY SOLVENT EXCEPT TETRACHLOROETHYLENE		

The manufacturer is not responsible for any changes in the technical specifications of the device in case of non-compliance with the washing instructions.

DISINFECTION

Use products with medical-surgical use.

STORAGE

Store in a place protected from light, heat, and moisture.

DISPOSAL

Dispose of the device in accordance with the applicable environmental protection regulations and separate waste collection.

	Product code		Medical Device
	Lot number		Manufacturer
	Medical Device compliant with Regulation (EU) 2017/745		Consult instructions for use
	Keep away from sunlight		Keep in a cool, dry place
	Date of manufacture		Not made with natural rubber latex
	Authorized Representative in Switzerland		Distributed by
	Unique Device Identifier		

DO NOT remove the label from the Medical Device, as it ensures traceability of the supplied product (Manufacturer, UDI, batch number, product code).

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Gima 28555 - 28556 - 28557



Erregi S.a.s.

Via Adua 40
21040 Vedano Olona (Va)
commerciale@erregiausilimedici.it
www.erregiausilimedici.it
MADE IN ITALY



08/13 - 13/11 - 13/04



Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
Made in Italy



Elite Med Group Sagi

Via Luigi Lavizzari 4
6850 Mendrisio
Switzerland



M28555-Rev.7-07.26
