FASCIA BIADESIVA FOAM
BIADHESIVE FOAM STRIP
BANDE ADHÉSIVE DOUBLE-FACE EN MOUSSE
TIRA ADHESIVA AMBAS CARAS DE ESPUMA
FITA BIADESIVA DE ESPUMA
ZWEISEITIGE SCHAUMKLEBESTREIFEN
ΑΥΤΟΚΟΛΛΗΤΗ ΤΑΙΝΙΑ ΑΦΡΟΥ ΔΙΠΛΗΣ ΟΨΗΣ
TAŚMA PIANKOWA DWUSTRONNA
OBOUSTRANNÝ PĚNOVÝ LEPICÍ PROUŽEK
DUBBELHÄFTANDE SKUMREMSA
KAKSIPUOLINEN VAAHTOMUOVILIIMANAUIHA
BANDĂ DIN SPUMĂ BIADEZIVĂ
DUBBELZIJDIGE SCHUIMSTRIP
KÉTOLDALÚ HAB RAGASZTÓSZALAG
ДВОЙНОЗАЛЕПВАЩА ЛЕПЕНКА ОТ ПЯНА

Gima 28715

REF 19921503

Mizra Medical Ltd
21 Hasadna Street, Industrial Park, Holon, 5881411, Israel
Made in Israel

EC REP AF Pharma Service Europe SL
Muntaner 281, Barcelona, 08021, Spain

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

catheters.

Please follow the securing instruction before using the product.

Biadhesive strips are accessories of the Male External Catheter (MEC) and compliant with Regulation (EU) 2017/745


Soft, easy to apply, secure the MEC, Non-sterile, non-sterilizable product.

How To Apply:

1. Clean the Area before applying the Biadhesive strip.
2. The hair around the penis shaft may be shaved before application.
3. Peel off one of the two paper cover strips.
4. Wind the sticky side of the strip around the penis shaft in a spiral motion starting from the back of the base and working upwards.
5. Do not make the bi-adhesive strip too tight to prevent circulation problems.
6. The strip must overlap.
7. Before applying the MEC remove the other paper protection strip.
8. Apply the MEC leaving about 2cm between the top of the glans and the end of the MEC.

Warnings

- It is recommended that the MEC and the strip will be replaced every 12-24 hours.
- Do not over-tighten the double-sided adhesive strip to avoid circulatory problems.
- Non-sterile product, not sterilizable.
- Disposable product, do not re-use, do not recycle. After use, all medical devices that have been in contact with a patient can contain contaminants, including pathogenic microorganisms, that are difficult to eliminate.
- Store in a cool and dry place, away from sources of heat.
- After use, do not dispose to the environment

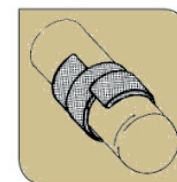Spiral overlapping of the tape stripe

Apply pressure on the condom

Composition:

An adhesive strip made from an elastic rubber foam that adapts to the body and is easily removable. Glue protection: siliconized paper MEC

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ENGLISH

INSTRUCTIONS

Biadhesive strips provide secure and gentle fixation for external

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos - índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Symboli-indeksi - Indeks simbolov - Index symbolov - Index de simbol - Symbool index - Indeks simbola - Szimbólum index - Symbolindeks - Индекс на символа - Simbolių rodyklė - Simbolu rādītājs - Sümboleindeks

	IT - Data di fabbricazione GB - Date of manufacture FR - Date de fabrication ES - Fecha de fabricación PT - Data de fabrico DE - Herstellungsdatum GR - Ημερομηνία παραγωγής PL - Data produkcji CZ - Datum výroby SE - Tillverkningsdatum FI - Valmistuspäivämäärä RO - Data fabricației NL - Productiedatum HU - Gyártás dátuma BG - Fabrikationsdato
	IT - Fabbricante GB - Manufacturer FR - Fabricant ES - Fabricante PT - Fabricante DE - Hersteller GR - Παραγωγός PL - Producent CZ - Výrobce SE - Tillverkare FI - Valmistaja RO - Producător NL - Fabrikant HU - Gyártó BG - Fabrikant
	IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - À conserver à l'abri de la lumière du soleil ES - Conservar al amparo de la luz solar PT - Guardar ao abrigo da luz solar DE - Vor Sonneneinstrahlung geschützt lagern GR - Κρατήστε το μακριά από ηλιακή ακτινοβολία PL - Przechowywać z dala od światła słonecznego CZ - Skladujte mimo slunečního světla SE - Skyddas från solljus FI - Säilytä auringonvalolta suojassa RO - A se păstra ferit de razele soarelui NL - Afgeschermd van zonlicht opslaan HU - Napfénytől védve tárolandó BG - Мă ikke udsættes for sollys

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