PROFESSIONAL MEDICAL PRODUCTS

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GIMA 34075 - 34102

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.

No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu
Province, CHINA
Made in China

YDC-6A3, YDC-6A1

SUNGO Europe B.V., Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, The Netherlands

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

ENGLISH

INSTRUCTIONS FOR USE

Name

Vacuum Mattress Stretcher

Model

YDC-6A1, YDC-6A3

Intended Use

Vacuum Mattress Stretcher is used to transferring the wounded in hospitals, gymnasium, ambulance and station. This product is used for emergency rescue. This is a reuse device, provided non-sterile.

Storage

Store this stretcher in the place of damp proof and non-corrosion environment.

Storage, transportation, as shown by the box stacking.

Component

Vacuum Mattress Stretcher is composed of fabric, stuffing, safety belt and special air pump.

Material

Vacuum Mattress Stretcher is made of TPU and Nylon.

Specification

The dimension is as below:

Model	Specification	Load-bearing	Picture
YDC-6A1	204x104x7cm	160kg	
YDC-6A3	204x100x5cm	160kg	

Shelf Life

5 years

Duration

Less 4 hours

Cleaning and maintenance

Keep clean as routine. Wipe with 75% alcohol solution.

Check the connecting and rotating parts of the stretcher.

Check the safety belts regularly, if they are worn or have cracks, replace them immediately.

How to use the device

- 1 Open the vacuum mattress stretcher flat and keep the particles in uniform distribution inside of stretchers.
- 2 When patient lie on the stretcher, the first-aid person need to connect the pump trachea with the stretcher valve together, and then open the valve as counterclockwise direction, exhaust the air from the stretcher according to the demand can be adjust the stretcher to be soft or hard. Closed the valve by hand as clockwise direction if feel the stretcher is comfortable.
- 3 Using the safety belt restraint the patient, first-aid person can hold the handles transporting patients.

Warning

1. Use safety belt when necessary.
2. Keep clean as routine.
3. Store this stretcher in the place of damp proof and non-corrosion environment.
4. Be transferred by common transporting tools.
5. Packed in cartons.
6. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Disposal

Please dispose the product after use to comply with local regulation.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos - índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Symboli-indeksi - Indeks simbolov - Index symbolov - Index de simbol - Symbol index - Indeks simbola - Szimbólum index - Symbolindex - Индекс на символа - Simbolių rodyklė - Simbolu rādītājs - Sümboleindeks

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