



GIMA

PROFESSIONAL MEDICAL PRODUCTS

LACCI EMOSTATICI MONOUSO
DISPOSABLE TOURNIQUETS
GARROTS JETABLES
TORNIQUETES DESECHABLES
GARROTES DESCARTÁVEIS
EINWEG-STAUBINDE
JEDNORAZOWE OPASKI UCISKOWE
STASSLANGAR FÖR ENGÅNGSBRUK
ΤΟΥΡΝΙΚΕ ΜΙΑΣ ΧΡΗΣΗΣ
EGYSZER HASZNÁLATOS ÉRSZORÍTÓK
ÜHEKORDSED ŽGUTID
TOURNIQUETS VOOR EENMALIG GEBRUIK
ÜHEKORDSED ŽGUTID

عاصبات للاستعمال الواحد

È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede - All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located - Il est nécessaire de signaler tout accident grave survenu et lié au dispositif médical que nous avons livré au fabricant et à l'autorité compétente de l'état membre où on a le siège social - Es necesario informar al fabricante y a la autoridad competente del Estado miembro en el que se encuentra la sede sobre cualquier incidente grave que haya ocurrido en relación con el producto sanitario que le hemos suministrado - É necessário notificar ao fabricante e às autoridades competentes do Estado-membro onde ele está sediado qualquer acidente grave verificado em relação ao dispositivo médico fornecido por nós - Jeder schwere Unfall im Zusammenhang mit dem von uns gelieferten medizinischen Gerät muss unbedingt dem Hersteller und der zuständigen Behörde des Mitgliedstaats, in dem das Gerät verwendet wird, gemeldet werden- Należy poinformować producenta i kompetentne władze danego Kraju członkowskiego o każdym poważnym wypadku związanym z wyrobem medycznym naszej produkcji - Det är nödvändigt att meddela tillverkaren och de behöriga myndigheterna i den berörda medlemsstaten, om alla allvarliga olyckor som inträffat i samband med den medicintekniska utrustning som levererats av oss- Σε περίπτωση που διαπιστώσετε οποιοδήποτε σοβαρό περιστατικό σε σχέση με την ιατρική συσκευή που σας παρέχουμε θα πρέπει να το αναφέρετε στον κατασκευαστή και στην αρμόδια αρχή του κράτους μέλους στο οποίο βρίσκεστε- Potrebno je prijaviti svaku ozbiljnu nezgodu koja se dogodila u vezi s isporučenim medicinskim proizvodima i nadležnom tijelu države članice u kojoj se nalazi- Peate teatama kõigist meie tarnitud meditsiiniseadmega seotud tõsistest vahetuhtumistest tootjale ja teie asukohaliikmesriigi pädevale asutusele- Alle ernstige ongelukken die zich in verband met het door ons geleverde medische hulpmiddel voordoen, moeten gemeld worden aan de fabrikant en aan de bevoegde instantie van de lidstaat waar u gevestigd bent- Orice accident grav produs, privitor la dispozitivul medical fabricat de firma noastră, trebuie semnalat producătorului și autorității competente în statul membru pe teritoriul căruia își are sediul utilizatorul - يجب الإبلاغ فوراً عن أي حادث خطير وقع فيما يتعلق بالجهاز الطبي الذي زدونا به إلى الجهة الصناعية والسلطة المختصة في الدولة العضو التي يقع فيها -



Ningbo Jenius Polymer Sci-Tech Co. Ltd
N° 2112, Ningheng South Road, Hengxi Town, Yinzhou District,
Ningbo, Zhejiang, China (Mainland)
Made In China

GIMA 37510-37511-37512



REF JS-TQ001



MedPath GmbH
Mies-van-der-Rohe-Strasse 8, 80807
Munich, Germany



SPICA CONSULTING SERVICE LT
41 DEVONSHIRE STREET, GROUND FLOOR OFFICE 1,
LONDON, UNITED KINGDOM W1G 7AJ



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



Model JS-TQ001, size 450mm*25mm*0.635mm

This product is made of medical polymer thermo-plastic elastomer, milky white or color, long flat type,

The new design point is used to pack the box, which is flexible and can be extracted continuously.

This product is suitable for medical institutions in routine treatment and treatment of infusion, blood transfusion, blood transfusion, hemostatic one-time use;

Emergency hemostasis in case of body bleeding or bleeding from field snake and insect bite.

Contraindications, matters needing attention, warnings and reminders:

1. This product is disposable and shall be destroyed after use;
2. If the package is damaged, it is strictly prohibited to use.
3. After use, please cover the suction cover to avoid dust and bacterial contamination;
4. The production date is shown on the side of the packaging box or the certificate of qualification;

Warnings and reminder:

A tourniquet can block the flow of blood, and tying it too long can seriously damage tissue, even lead to limb necrosis.

A tourniquet should only be used to bind the limbs, not the head, neck, or trunk.

Don't cover it with anything else, and don't cover the tourniquet tied to the limb.

If you have to leave the patient alone in the camp, over his head, with a pen or lipstick, write down the timing of the bundle and the location of the kit.

Check blood circulation:

After the tourniquet is bandaged, the toe or finger should be checked frequently to see if there is a dark purple spot at the end of the tourniquet, and whether the body temperature at the end of the tourniquet has dropped.

If there is such a phenomenon, should loosen the cloth band, otherwise for a long time, will cause tissue necrosis.

For the same reason, a tourniquet should not be used for long periods of time to tie a limb, unless it is necessary to do so during an arterial ligation. In general, direct pressure should be tried first, followed by pressure points until direct pressure is maintained on the wound.

If the wound is at the end of the limb, elevate it.

Storage conditions:

the packed tourniquet should be stored in the relative humidity of no more than 80%, without corrosive gas and well ventilated

the packaged tourniquet shall be valid for 3 years from the date of production under the conditions of storage, transportation and storage

Symbols:

	Keep in a cool, dry place
	Manufacturer
	Lot number
	Medical Device compliant with Regulation (EU) 2017/745
	Medical Device
	Keep away from sunlight
	Date of manufacture
	Product code
	Consult instructions for use
	Authorized representative in the European community
	Disposable device, do not re-use
	Expiration date
	Imported by
	Unique identifier
	Authorized Representative in the UK

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies