

READ ALL INSTRUCTIONS PRIOR TO USE. CONSULT PHYSICIAN BEFORE USE.

CONTRAINDICATIONS: Not intended for persons over 242 lbs (110 kg) - Not intended for icy or slippery conditions. - Not intended for persons with latex allergies.

INSTRUCTION GUIDE: To unfold your cane hold on to the handle and let the foldable shaft segments unfold towards the floor. To prevent falls, you should always check to make sure that the individual shaft segments (Fig. 1) are securely locked into place.


WARNING: DO NOT disassemble the body of the Trusty Cane TM. The cane contains an elastic cord inside. Uncontrolled release of the elastic cord inside can cause severe injury to unprotected body parts, particularly eyes, if the elastic cord is cut, frayed or damaged discontinue use. DO NOT stretch the cord more than is needed to disassemble or assemble the segments.

To adjust the height of your Trusty Cane™: Loosen the twist lock (Fig. 2) by rotating it clockwise several turns. Depress the locking button (Fig. 3) and pull up on the handle to lengthen, or push down on the handle to shorten. Remember to retighten the twist lock after adjusting the Trusty Cane™ to your desired height.

WARNING: To prevent falls, always ensure individual shaft segments are securely locked into place and locking button is fully engaged.

To collapse the Trusty Cane™: Your Trusty Cane™ is easily foldable in to a convenient travel size by separating the 4 individual shaft segments from each other. Hold each set of shaft segments near the joint and pull them apart (Fig. 4). The individual shaft segments will neatly fold together (Fig. 5)

To use the lights on your Trusty Cane™: The button on the top of the Trusty Cane™ handle (Fig. 6) activates the lights. Pressing the button once will activate the lights, and pressing it again will turn them off. The lights will go off automatically after 20 minutes.

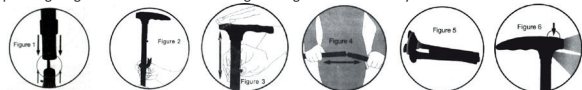

Maintaining your Trusty Cane™: To replace batteries, use a Phillips head screwdriver to unscrew the battery compartment under the Trusty Cane™ handle. Remove any dead batteries and properly dispose of them. Using 3 AAA batteries, align + and - as indicated, otherwise lights will not operate. Replace and tighten the screw on the battery door. Regularly inspect cane for signs of wear, including any fraying to the elastic cord or wear to the rubber feet. If you see signs of wear, discontinue use immediately.

WARNING: Do not mix old and new batteries. Do not mix alkaline, standard (carbon zinc) or rechargeable (nickel-cadmium) batteries.

GIMA WARRANTY TERMS: The Gima 12-month standard B2B warranty applies.

Gima 43044 - 43045 REF GB-801 (SV) GB-801 (BK)

Fabricato da/Manufactured by/Fabrique par/Fabricado por/Hergestellt von/
Fabricado por/Κατασκευάστηκε από/Произведено от/Vyrobce/Fremstillet af/
Tootja/Vaalistaja/Proizvodat/Gyártó/Ražots/Gamintojas/Produsert av/
Getfabriceerd door/Wyprodukowane przez/Fabricat de/Vyrába/Proizvajalec/
Tilberladd av/تصنيعها بواسطة
Ningbo Gaobo Sports Co., Ltd
No.1, Xiajin, Dali, Shenzhen Town, Ninghai County
Ningbo 315514, China
Made In China

Importato da/imported by/importé par/importado por/importiert von/
Importado por/Εισαγυγής από/Dovezeno od strane/Dovezeno uživatelem/importeret
af/Importēja/ Tuojas/Uvezeno od strane/importálta/importēja/importavo/
Importert av/Geimporteerd door/importowane przez/importat de/Dovážal/
Uvozi/Importat av/توزيع بواسطة
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



