



PROFESSIONAL MEDICAL PRODUCTS

DEAMBULATORE STILISH STLISH ROLLATOR DÉAMBULATEUR ÉLÉGANT ANDADOR STLISH

ATTENZIONE: È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede.

ATTENTION: All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

AVIS: Il est nécessaire de signaler tout accident grave survenu et lié au dispositif médical que nous avons livré au fabricant et à l'autorité compétente de l'état membre où on a le siège social.

ATENCIÓN: Es necesario informar al fabricante y a la autoridad competente del Estado miembro en el que se encuentra la sede sobre cualquier incidente grave que haya ocurrido en relación con el producto sanitario que le hemos suministrado.

GIMA 43164



Foshan HCT Medical Equipment Co., Ltd
No.501 of 19 Building,Daxing Liandong U Valley,
No. 4, Xiangshun Road, Danzao Town, Nanhai
District, Foshan, Guangdong, China



SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, The
Netherlands



HCT-9226



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
Made in China



GENERAL SAFETY WARNING

Please ensure these instructions are fully read and implemented. Failure to do so may result in injury to the user. Retain in a safe place for future reference

Intended Use

The Lightweight 4 wheel Rollator is designed to provide support and assistance to those unable to, or who lack the confidence, to walk without some level of support. This rollator is intended for indoor and outdoor use, and folds away for easy storage or transportation

Safety Precautions - Do's and Don'ts

- Do not exceed the maximum user weight limit of 136Kg.
- Do not use the rollator on stairs or an escalator.
- Do not attempt to push the rollator while someone is sitting in the seat.
- Do not use the seat to transport people or objects.
- Do not use the seat without the backrest in place.
- Do not sit on the seat when the rollator is on an incline.
- Only use the rollator as a walking support.
- Only use the specified accessories and spare parts for this rollator
- Loads should only be transported using the bag. Do not hang heavy items from the handles as this may affect the stability of the Rollator.

Safety Precautions - Pre Use Checks

- Ensure Rollator is locked open before use.
- Check the brakes before use.
- Make sure that all parts are secure and that moving parts are in good working order.

Safety Precautions - During use

- Do not self-propel the rollator while seated. Serious injury to the user and/or damage to the rollator frame or wheels may result from improper use.
- Do not use the rollator to walk backwards.
- Caution must be taken when negotiating curbs and other obstacles, or when using on sloping, uneven or slippery surfaces. Failure to do so may result in a serious risk of a fall or injury.
- Do not perform any adjustments to the rollator while it is in use.
- Exercise caution when there is a heavy load in the bag (max. load 5Kg)
- All wheels must be in contact with the floor at all times.
- Brakes must be in the locked position before using the seat.
- When using the rollator in a stationary position, the brakes must be locked.

Content list

- A) 1 Rollator Frame
- B) 2 Removable Front wheels
- C) 1 Bag

Unpacking

- Check all parts carefully before assembling the product.
- Do not assemble if any of the parts are damaged.
- Do not assemble if any of the parts in the Contents list are missing.

Assembly


A Open the frame by lowering the rear legs until their housing and secure them. Double check it is locked into place.

B Open the rollator, push down on either side of the seat. **IMPORTANT:** Both sides must "click" into position.

C Insert the wheels into the front wheel slot. Be sure you hear a click indicating the wheel is securely in place.

D Proceed to the height adjustment of the handles. Press the locking pin and adjust the position of the handlebars to your desired height. Place the bag.

Folding the Rollator (E)

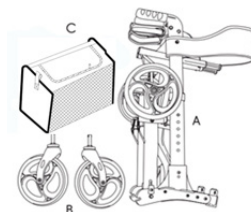

Make sure you stand behind the rollator. Pull the strap upwards to fold the rollator

Carrying the rollator(F)

When the rollator is folded you can carry the rollator by using the lifting handles

Folding Rear wheels (G)

Slide down the red tab to unlock the rear wheel. Fold upwards until locked into place

Detaching Front wheels (H)

To detach the front wheels, simply press the red button and pull down the wheels

Sitting

When using the rollator as a seat, ensure the safety precautions detailed on page one are adhered to, in particular;

- Ensure the parking brakes are in the locked position.
- Do not use as a seat without the backrest being in position
- Do not use as a seat on sloping or uneven ground
- Do not exceed the maximum user weight
- Do not use the rollator as a wheelchair

Checking and Operating the Brakes (I)


Before using the rollator always check the brakes are working correctly.

To slow down or brake, pull the brake lever. Release the lever to resume the march

To engage the parking brake, push the brake lever down into the locked position

Maintenance and Cleaning


Always remove the dirt that forms after each use, using the no aggressive solutions and taking care to dry. Do not use abrasive sponges. Periodically check the brake functionality as well as the state of the sealing of the clamping elements and welds

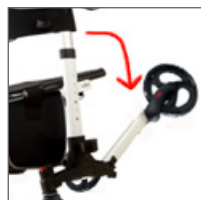

A



B

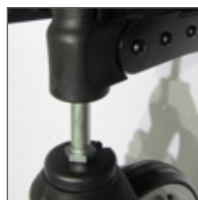

C




D




E




F

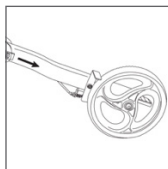

G

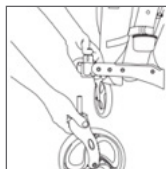

H

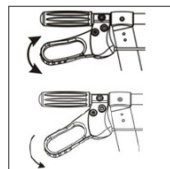

I

	Product code		Manufacturer
	Lot number		Caution: read instructions (warnings) carefully
	Keep in a cool, dry place		Caution: read instructions (warnings) carefully
	Keep away from sunlight		Medical Device
	Consult instructions for use		Date of manufacture
	Imported by		Authorized representative in the European Community
	Non-Sterile		Unique Device Identifier

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.