

ENGLISH

POSTURAL BELTS

WARNINGS

Before performing any operation, visually check the integrity of the device. If the product shows any anomalies, immediately contact the point of sale where the purchase was made. Erregi sas assumes no responsibility for the improper use of the device. In the event of a serious incident related to the use of the device, please report the incident to the manufacturer and the competent authority of the member state where the user and/or patient is located.

DESCRIPTION

Postural belts made of water-repellent, Class 1 fire-retardant fabric, with Class 1IM fire-retardant expanded polyurethane padding and polypropylene straps, available in different versions: with simple closure, safety closure, or eyeleted straps.

INTENDED USE

Postural belts for maintaining the posture of non-cooperative adult patients in wheelchairs, commode chairs, and for fall prevention.

INDICATIONS AND PRECAUTIONS FOR USE

The device must be positioned with adequate tension to ensure proper functioning. When adjusting the straps, a flat hand should be able to pass between the patient and the device. Do not use belts with eyeleted straps on very agitated patients. It is recommended to regularly monitor both the patient and the area in contact with the belt.

Always use the device over clothing to protect the person.

CONTRAINDICATIONS

No contraindications have been detected, except in cases of proven sensitivity to the component materials.

COMPOSITION

OUTER FABRIC: Polyamide + F.R. polyurethane resin

Class 1 fire-retardant

FILLING: Expanded polyurethane, Class 1 IM fire-retardant

WASHING INSTRUCTIONS

			
WASH AT MAX 40°C	DO NOT BLEACH	DO NOT IRON	DO NOT TUMBLEDRY
	USE ANY SOLVENT EXCEPT TETRACHLORETHYLENE		

Attention: where present, remove keys and blocks before washing, as contact with liquids may compromise their functionality.

The manufacturer is not responsible for any changes in the technical specifications of the device in case of non-compliance with the washing instructions.

DISINFECTION

Use products for medical-surgical use.

STORAGE

Store in a place protected from light, heat, and moisture.

DISPOSAL

Dispose of the device in accordance with the applicable environmental protection regulations and separate waste collection.

	Product code		Medical Device
	Lot number		Manufacturer
	Medical Device compliant with Regulation (EU) 2017/745		Consult instructions for use
	Keep away from sunlight		Keep in a cool, dry place
	Date of manufacture		Not made with natural rubber latex
	Authorized Representative in Switzerland		Distributed by
	Unique Device Identifier		

DO NOT remove the label from the Medical Device, as it ensures traceability of the supplied product (Manufacturer, UDI, batch number, product code).

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

GIMA 43175 - 43176 - 43177



Erregi S.a.s.

Via Adua 40
21040 Vedano Olona (Va)
commerciale@erregiausilimedici.it
www.erregiausilimedici.it
MADE IN ITALY



21/26/S - 20/26/S- 19/26/S



Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
Made in Italy



Élite Med Group Sagi

Via Luigi Lavizzari 4
6850 Mendrisio
Switzerland

M43175-Rev.6-07.26

