

CARRELLO PORTABIANCHERIA A 3 RIPIANI

LAUNDRY TROLLEY 3

CHARIOT À LINGE 3 RAYONS

CARRO DE LAVANDERÍA 3 ESTANTES

CARRINHO DE LAVANDARIA 3 PRATELEIRAS

WÄSCHWEGABE MIT 3 ABLAGEN

ΤΡΟΧΗΛΑΤΟ ΙΜΑΤΙΣΜΟΥ 3 ΡΑΦΙΩΝ

**WÓZEK DO TRANSPORTU BRUDNEJ
BIELIZNY Z 3 PÓŁKAMI**

CĂRUCIOR PENTRU RUFE 3 RAFTURI

VOZÍK NA PRÁDLO SE 3 POLICEMI

TVÄTTVAGN 3 HYLLOR

WASGOEDWAGEN 3 SCHAPPEN

KOLICA ZA VEŠ 3 POLICE

SZENNYES KOCSI 3 POLCOS

КОЛИЧКА ЗА ПРАHE 3 НИВА

PESUVAUNU 3 HYLLYÄ

**SKALBINIŲ VEŽIMĖLIS SU 3
LENTYNOMIS**

MANUALE D'USO – USER MANUAL – MODE D'EMPLOI – MANUAL DE USO - MANUAL DO USUÁRIO - BENUTZERHANDBUCH
- ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΤΗ - INSTRUKCJA OBSŁUGI - MANUAL DE UTILIZARE - UŽIVATELSKÝ MANUÁL - ANVÄNDARMANUAL
- HANDLEIDING - KORISNIČKI PRIRUČNIK - HASZNÁLATI UTASÍTÁS - РЪКОВОДСТВО ЗА УПОТРЕБА - ОНЈЕКИРЈА -
VARTOTOJO VADOVAS



Gima 45913

Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante / Hersteller / Κατασκευαστής / Producent / Producent / Výrobce / Tillverkare / Fabrikant / Proizvođači / Gyártó /
производитель / Valmistaja / Gamintojas:
Jiangsu Saikang Medical Equipment Co.,Ltd
No. 35 Lehong Road, Modern Agriculture Demonstration
Park, Zhangjiagang City, Jiangsu Province, China
export@saikangmedical.com
Made in China

Importato da / Imported by / Importé par / Importado por / Importado por / Importiert von / Εισάγεται από / Importowane przez /
Importat de / Importováno / Importerad av / Geïmporteerd door /
Uvezao / Importálja / Внесен от / Maahantuoja / Importavo / :
Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

MANUAL INTRODUCTION

Note: Please carefully read this manual and strictly follow the steps before installing and using. We require all users to have the knowledge of user's manual and all the cautions.




- This manual is applicable to SKH027 laundry trolley. It contains the installation method, operating instructions and maintenance check points.
- The graphics in this manual are for users' reference only. In actual use, please refer to the real object.
- This user manual includes the description, instruction, maintenance and technical parameters of the product.
- Any questions, please contact the after-sale service center or distributor.
- Anyone using or operating this product should read and comply with it.
- Except for some unexpected situations, users are required to be familiar with the usage and precautions

1. INSTALLING GUIDE

1.1 List of standard accessories

After opening the packing case, please check whether the following parts are complete in the packing case:

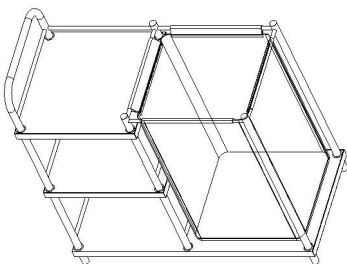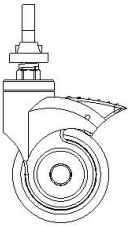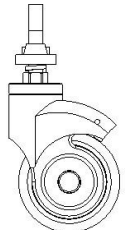
Table a:



N.	Name	Picture	Specification	QTY	Unit
1	Main body		L900 x L450 x A930mm	1	pc
2	Caster with brake		3inch caster with brake	2	pc
3	Caster without brake		3inch caster without brake	2	pc

1.2 Casters installation

Lay a protective pad and put the laundry trolley body upside down;

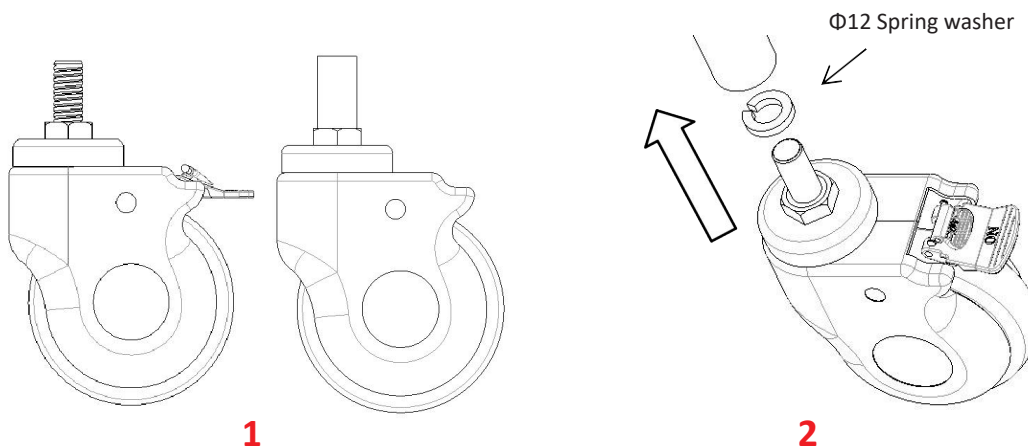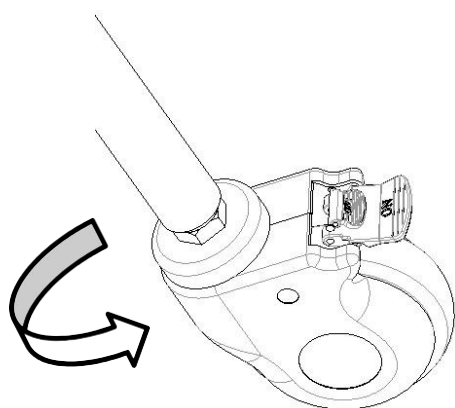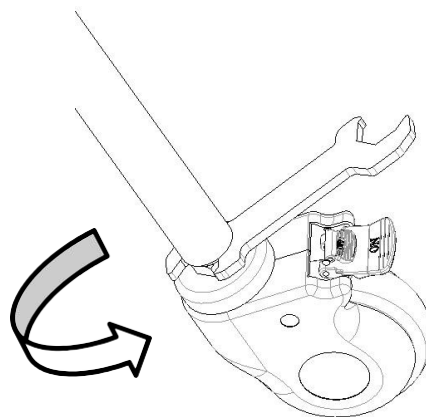Take out the $\Phi 12$ spring washer from the screw pack, insert the caster into the tube and tighten it clockwise, and then tighten it with an hex wrench.



**3****4****2. USE METHOD****1**

1-Main body

2


2-Casters

Laundry trolley size	Length	mm	900
	Width		450
	The height of the trolley above the ground		1.050
Weight	Safe load bearing	Kg	50

2.1 Use method of casters


2.1.1 Lock the caster:

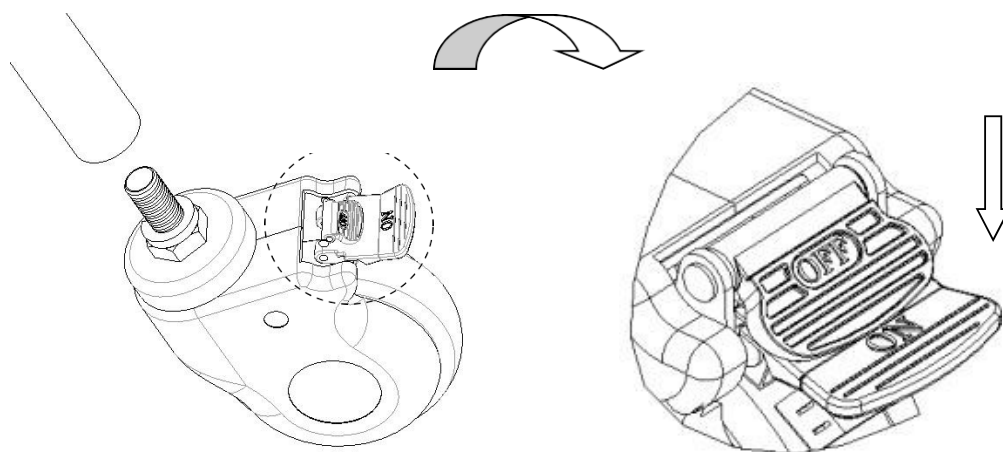After pushing the trolley to the desired position, press the button and lock the caster when you hear a “click”, as shown in the figure below.



2.1.2 Unlock the caster:

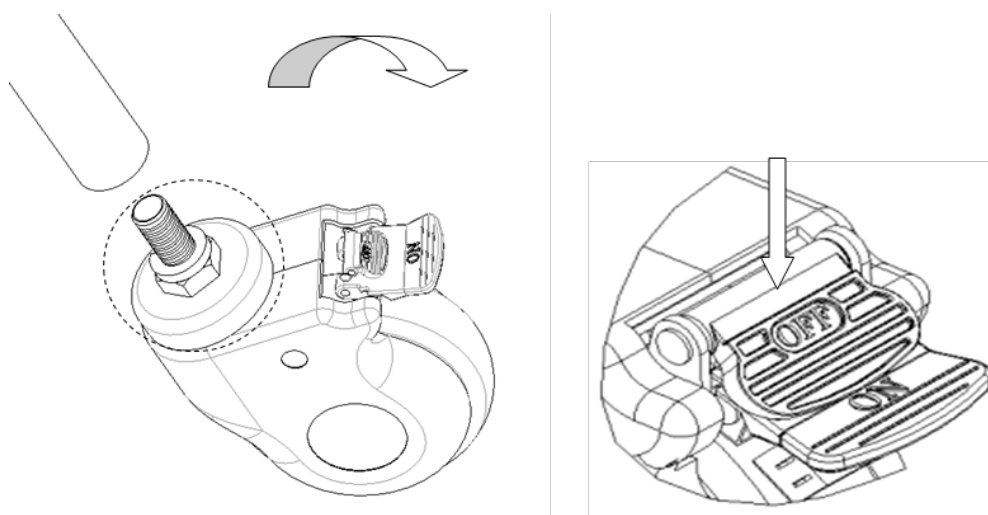Lift the button with your foot and unlock the castor when you hear a “click”.



3. FAULTS EXCLUSION

Common failures	Probable reasons	Excluding methods
Difficult to move the whole trolley	1. The caster is locked without releasing the brake 2. The caster is damaged	1. Release the brake 2. Replace the caster

4. LAUNDRY TROLLEY-MAINTENANCE

- In order to use the laundry trolley safely, it is necessary to perform regular safety inspections on the laundry trolley. It is recommended to check it every six months to ensure that the connections are not loose and all operations are normal.
- When the laundry trolley frame ages and it reaches a certain servicing time, which the metal part of the laundry trolley and plastic parts of the stainless steel basin frame can be recycle.
- Avoid scratching the panel with sharp-angle appliances or knives during use, and clean it frequently to keep it clean and dry.
- If the panel is accidentally stained with stains, it is recommended to clean it. Do not use alkaline or corrosive chemicals to clean the laundry trolley, which will cause the stainless steel surface to rust.
- Regularly check the casters to avoid hitting or overloading, which will cause damage to the casters.
- It is forbidden to force the casters to brake when braking, which will cause damage to the brake system.
- The wear of the tire surface of the wheel can be detected by visual inspection. Some thread spun yarn and other accumulations may be wrapped around the wheel, remove the bolts and nuts of the wheel, clean up the debris, and check whether the bearing of the wheel is damaged. If the parts are not damaged, can be reassembled and used.
- Judging whether the caster is replaced according to the time used, wear and tear, you can replace the wheels and make sure that the axles bolt and nuts are well put together and connected. If possible, use a lock washers or locknuts to help you with this process.
- For the casters equipped with brakes, it is necessary to regularly check whether the brakes are working properly. Check the brakes once a day or before each use. For a laundry trolley equipped with multiple brake casters, only one brake caster can be locked at a time which will allow you to try and push the laundry trolley also check whether one of the brake performance of each caster is good. If the function of the brake fails due to wear or damage to the wheels, immediately replace the wheels and then retest the brake system.
- If the mechanism of the brake system of the caster is damaged, and the brake needs to be repaired or replaced, please contact our after-sales

service department or authorized dealer provide by our company. Every time the brakes are replaced, the brake performance of the casters must be retested

5. PACKAGING, TRANSPORT, STORAGE

1. The packaging of laundry trolley is carried out according to the contract or product standard.
2. Avoid rushing out, violent vibration, and protection from sunlight and rain during the transportation of the laundry trolley.
3. The laundry trolley should be stored:
 - a) Ambient temperature: -40°C~ 70°C.
 - b) Relative humidity: ≤ 95%.
 - c) Atmospheric pressure: 500hPa ~ 1060 hPa. The max load is 50Kg

6. AFTER-SALES SERVICE

1. Please keep the files that comes with the machine and invoices of this product properly, and you need to present these files when the company performs warranty and maintenance for the product.
2. If there is any problem in the process of using, please contact our company in time, so that our company can provide you with accurate and fast technical support and maintenance services in a timely manner.
3. From the date of sale, if the product is broken or damaged due to the correct installation and use according to the specification, the product will enjoy one-year free warranty and lifelong maintenance service with the "certificate" or invoice.
4. From the date of purchase, if it is indeed damaged or does not work normally due to quality problems within one year, the company will provide to the users a free repairing of the product.
5. Lifetime service from manufacturer: JIANGSU SAIKANG MEDICAL EQUIPMENT CO LTD

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies