GIMA

PROFESSIONAL MEDICAL PRODUCTS

**TAVOLA SPINALE
PE STRETCHER
PLANCHE DORSALE
TABLA ESPINAL
PRANCHA DORSAL
WIRBELSÄULENBRETT
ΣΑΝΙΔΑ ΑΚΙΝΗΤΟΠΟΙΗΣΗΣ
DESKA ORTOPEDYCZNA
PÁTEŘNÍ DESKA
RYGGRADSBRÄDA
RANKALAUTA
SPINALNA NOSILA
CHRBTCOVÁ DOSKA
TARGĂ TIP PLACĂ SPINALĂ
IMMOBILISATIEPLANK
SPINALNA NOSILA
GERINCTÁBLA
SPINAL-BÅRE
ΓΡΨΒΗΑΥΗΑ ΟΠΟΡΑ
STUBURO LENTA
MUGURKAULA DĒĻIS
RYGGBÅRE**

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GIMA 56040 - 56044 - 56047 - 56528

JIANGSU RIXIN MEDICAL EQUIPMENT CO.,LTD.
No.427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu
Province, CHINA
Made in China

YDC-7A1 - YDC-7B1 - YDC-7A4 - CB-01

SUNGO Europe B.V., Fascinatio Boulevard 522, Unit 1.7,
2909VA Capelle aan den IJssel, The Netherlands

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

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Model: YDC-7A1, YDC-7A4, YDC-7B1, CB-01

Intended Use

PE Stretcher is used for transport patients and wounded person with cervical and lumbar injuries. Fix the safety belts on the stretcher, and it can be used with head immobilize at the same time. This is a reuse device, provided non-sterile.

Components

PE Stretcher is composed of spine board and safety belts.

Material

PE Stretcher is made of polyethylene plastic and Nylon, which is durable and easy to use.

Specification:

The dimension is as below:

Model	Specification	Load-bearing	Picture
YDC-7A1	184x45x6cm	181kg	
YDC-7A4	136x35x5cm	91kg	
YDC-7B1	185x46x5cm	227kg	
CB-01	60x41.5x8cm	159kg	

Storage

Store this stretcher in the place of damp proof and non-corrosion environment. Storage, transportation, as shown by the box stacking.

Cleaning and maintenance

Keep clean as routine. Wipe with 75% alcohol solution. Check the connecting and rotating parts of the stretcher. Check the safety belts regularly, if they are worn or have cracks, replace them immediately.

Shelf Life

8 years

Duration

Less 4 hours

Use method

1.The PE Stretcher can transport the patients directly. 2.The patient lying on the stretcher and wear the safe belt.

Warning

When transporting sick people, make sure that the safety belt is fastened to ensure safety. All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Disposal

Please dispose the product after use to comply with local regulation.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos- índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Symboli-indeksi - Indeks simbolov - Index symbolov - Index de simbol - Symbool index - Indeks simbola - Szimbólum index - Symbolindeks - Индекс на символа - Simbolių rodyklė - Simbolu rādītājs - Sūmbolite indeks

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