GIMA

PROFESSIONAL MEDICAL PRODUCTS

OROLOGIO DIGITALE PER INFERMIERI

DIGITAL NURSE WATCH

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Functions

- 6 digits display: Month, Date, Week day, Hour, Minute, Second
- Stop watch
- 12/24 hours
- Alarm
- EL background light

Instructions

Set Time & Date, week day

3 x Press B, second flashes, set time by pressing D,C, by pressing C to confirm & return normal display. Please set time by following steps.

3xPress B (Example: **13:02:23**)

Second (23) flashes, press D to increase the seconds, press C back to 00, press C to confirm.

Minute (02) flashes, press D to increase the numbers , press C to confirm.

Hour (13) flashes, press D to increase the numbers, press C to confirm.

Date flashes, press D to increase the numbers, press C to confirm

Month flashes, press D to increase the numbers, press C to confirm

Week flashes, press D move the flash to the correct week day, then press B to confirm all setups, and back to normal display.

Stop watch

Under normal display, 1 x press B
 enter stopwatch mode,
 1 x press D Start/stop to counting time
 1 x press C clear data.
 1 x press B, back to normal display.

12/24 Hours

Under normal display , 3 x press B,
 enter time setup mode,
 2 x press C enter 12/24 switch mode,
 press D until 12/24 displays.
 1 x press B back to normal display.

EL background light

1 x press A, EL light on 3 seconds.

Set Alarm

Under normal display.
 2 x press B, enter alarm mode, hour flashes, press D to adjust the hour, then press C,
 minute flashes, press D to adjust minute, press B back to normal display.

Alarm On/Off

Press C(don't remove), press D at the same time to set ON/OFF of alarm. Alarm
 keeps on 1 minute.

Hourly bell

Press C(don't remove),press B to set hourly bell on/off. When hourly bell is on, all
 week day display in on.
 Otherwise means hourly bell is off.

Battery replacement

On the back of the watch, there is a notch to open the housing and get to the battery.
 Lift the rear with a sharp and thin instrument. Remove the metal cover and the battery
 by sliding it on the side.

Insert the new battery complying with the correct polarity with the “+” facing up.
 Replace the metal cover and close the case.

Weight: 45g

Battery: 3V Lithium Battery CR2025

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

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