GIMA

PROFESSIONAL MEDICAL PRODUCTS

STERILE TONGUE DEPRESSOR
ABBASSALINGUA STERILE
ABAISSÉ-LANGUE STÉRILE
STERILER ZUNGENSEPTEL
DEPRESOR LINGUAL ESTÉRIL
ABAIXA-LÍNGUA ESTÉRIL
ΑΠΟΣΤΕΙΡΩΜΕΝΟ ΓΛΩΣΣΟΠΙΕΣΤΡΟ
СТЕРИЛЕН ДЕПРЕСОР ЗА ЕЗИК
STERILNÍ JAZYKOVÝ DEPRESOR
STERIL TUNGESPATEL
STERIILNE KEELESPAATEL
STERIILI KIELILASTA
STERILNI DEPRESOR ZA JEZIK
STERIL NYELVNYOMÓ
STERILUS LIEŽUVIO SPAUSTUKAS
STERILS MĒLES SPIEDĒJS
STERIL MUNNESPATEL
STERIELE TONGSPATEL
SZPATUŁKA STERYLNA
DEPRESOR DE LIMBĂ STERIL
STERILNÝ STLAČOVAČ JAZYKA
STERILNÝ STLAČOVAČ JAZYKA
STERIL TUNGESPATEL

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Gima 25511 - 25512

M25511-M-Rev.3.05.26

Jiangsu Suyun Medical Materials Co., Ltd.
No.18 Jin Qiao Road Dapu Industrial Park 222002
Lianyungang, Jiangsu province
PEOPLE'S REPUBLIC OF CHINA
Made in China

72522

Shanghai International Holding Corp.,
GmbH (Europe) - Eiffeustraße, 80,
20537 Hamburg, GERMANY
Tel.0049-40-2513175

0123

Medicare AG,
Hauptstr. 51 5024 Küttigen,
Schweiz CHRN-AR-20002506

Gima S.p.A
Via Marconi, 1 - 20060 Gessate (Mi) - Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

INSTRUCTIONS FOR USE

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Product Name: Sterile Tongue Depressor

Indications: It is used for people who need throat examination or treatment.

Intend purpose: Sterile tongue depressor is intended to inspect oral diseases for those patients.

Contraindications: None.

Caution:

- A. The product should be used by professional staff
- B. If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.
- C. Sterilized by EO gas. Product is sterile.
- D. For single use. Discard after use.
- E. Please checking the individual package before using it.
- F. Don't use if package is opened or damaged.
- G. Sterile and validity for 5 year. Don't use if overdue.
- H. Discard after use. After use, the products shall be collected, transported, stored and disposed to ensure that the products are harmless to human health and the environment. Or the product can be disposed in accordance with the local medical regulations for waste disposal.

Patient group: Sterile Tongue Depressor is suitable for patients who need throat examination or treatment.

Intended users: It can be used by trained doctors, nursing staff.

Clinical benefits: Sterile Tongue Depressor can be convenient for doctors to check for oral disease

Any residual risk and any undesirable side-effects: none

Performance characteristics: The product is provided in a sterile state and sterilized with ethylene oxide. Single-use

Physical performance:

Appearance: The surface of the tongue depressor shall be flat and smooth, and the edge shall be free of burrs and burrs.

The tongue depressor shall be free from cracks and disconnection.

The surface of the tongue depressor shall be clean. Oil stains, pollution and mildew are not allowed.

Dimension: The dimension of tongue depressor shall comply with the requirements.

Biological performance: The product should be sterile.

Structural composition: The product is composed of tongue depressor body.

Method:

- a. Before use, check whether the product is perfect and whether it can operate normally.
- b. Open the small package and take the product out.
- c. According to the clinical routine operation, inspect oral diseases

Storage: Storage condition: In relative humidity not higher than 80%. No corrosive gas, cool, dry, well ventilated, clean environment. Do not store with disinfectant that contain chlorine compound.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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