ITALIANO

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ENGLISH

Instructions/Instructions for Use for Reusable NON-SURGICAL INSTRUMENTS & Accessories

Introduction: These Instructions apply to Reusable NON-SURGICAL instruments and accessories supplied by S. Jee Enterprises
WARNINGS
Wear Gloves while using the instruments. Be careful of the sharp point and of the moving parts of the instruments, if any. The instruments must be stored in dry place and must be handled with care to prevent damage. Use only for its intended purpose to ensure maximum effectiveness. Steam (moist heat) is the only recommended method. Personal Protective Equipment (PPE) should be worn while using the instruments. Saline and cleaning/disinfection agents containing aldehyde, chloride, active chlorine, bromine, bromide, iodine or iodide are corrosive and should not be used. Do not allow biologic soil to dry on contaminated devices. Metal brushes and scouring pads must not be used during manual cleaning, because they will damage the surface and finish of the instruments. Use only soft bristle nylon brushes with different shapes, lengths and sizes to aid with manual cleaning. Do not process instruments with polymer components at temperatures equal to or greater than 140°C/285°F. Oils or silicone lubricants should not be used on instruments. Dry instruments with a clean, absorbent non-shedding lint free cloth. Clean, filtered compressed air may be used to remove moisture from lumens, holes, cannulas and difficult to access areas. After cleaning, all devices should be thoroughly inspected for residue biologic soil or detergent. If contamination is still present repeat the cleaning process. Only qualified person should use the healthcare devices. Although there is no chance of any serious incident due to these medical devices, anyway if you find any serious incident that has occurred in relation to these devices inform us (manufacturer) and relevant competent/regulatory authority.

Cleaning Steps

Place instruments in a suitable validated washer/disinfector. Follow the washer/disinfector manufacturer's instructions for loading the instruments for maximum cleaning exposure; e.g. place concave instruments on their side or upside down, place heavy devices on the bottom of trays and baskets. If the washer/disinfector is equipped with special racks use them according to the manufacturer's instructions.

Recommended parameters are:

Sterilization

Always consult and follow the sterilizer manufacturer instructions for load configuration and equipment operation. Sterilizing equipment should have demonstrated efficacy (e.g. FDA clearance, EN 13060 or EN 285 compliance). Additionally the manufacturer's recommendations for installation, validation, and maintenance should be followed. Validated exposure times and temperatures to achieve a 10 sterility assurance level (SAL) are listed in the following table.

Cycle	Description	Notes:
1	Pre-wash -- Cold Softened Tap Water -- 2 Minutes	- The washer/disinfector manufacturer's instructions should be followed.
2	Enzyme Spray & Soak -- Hot Softened Tap Water -- 1 Minute	- A washer/disinfector with demonstrated efficacy (e.g. FDA approval, validated to ISO 15883) should be used.
3	Rinse -- Cold Softened Tap Water	- Dry time is shown as a range because it is dependent upon the load size placed into the washer/disinfector.
4	Detergent Wash -- Hot Tap Water (64-66°C/146-150°F) -- 2 Minutes	
5	Rinse -- Hot Purified Water (64-66°C/146-150°F) -- 1 Minute	
6	Hot Air Dry (116°C / 240°C) -- 7 to 30 Minutes	

	Cycle Type	Minimum Temperature	Minimum Exposure Time	Pressure
For Europe recommended parameters	Pre-vacuum / Vacuum Pulse	134°C/273°F	3 Minutes	28 p.s.i.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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