GIMA

PROFESSIONAL MEDICAL PRODUCTS

SPONDA PER LETTO DELUXE - REGOLABILE CON 4 GANCI
DELUXE BEDRAIL - ADJUSTABLE WITH 4 HOOKS
BARRIÈRE DE LIT DELUXE - AJUSTABLE AVEC 4 CROCHETS
BARANDILLA DE CAMA DELUXE - AJUSTABLE CON 4 GANCHOS
GRADE DE CAMA DE LUXO - AJUSTÁVEL COM 4 GANCHOS
DELUXE SEITENTEIL - ZUM EINHÄNGEN MIT 4 HAKEN
ȘINĂ DELUXE PENTRU PAT - AJUSTABILĂ CU 4 SISTEME DE PRINDERE
DELUXE SÄNGRÄCKE - JUSTERAS MED 4 KROKAR
BARIERKA ŁÓŻKOWA DELUXE - REGULOWANA Z 4 HAKAMI
ΚΑΓΚΕΛΑ ΚΡΕΒΑΤΙΟΥ DELUXE - ΡΥΘΜΙΖΟΜΕΝΑ ΜΕ 4 ΑΓΚΙΣΤΡΑ

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Weight Capacity: 75kgs

INTENDED USE:

The bedside rail is an accessory designed to prevent an accidental fall of a person from any bed for hospital or home care. Suitable for most metal beds & with easy control system, so it does not have to be completely removed for transporting the patient to and from the bed. The bedside rails are designed to fit in most beds thanks to its simple frame, that can be placed on the headboard and the footboard of the bed.

CAUTION (!):

- In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- Do not use the product for a purpose not indicated in this manual
- LONGWAY Medical declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- LONGWAY Medical reserves the right to change the information contained in this document without previous notice
- There is a $\pm 3\%$ Deviation for the Dimensions of the Products.

GENERAL SAFETY WARNINGS:

- ⚠ If you don't read this user manual, it is preferable not to use this product or another available part. If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- ⚠ Keep away the product from heat sources.
- ⚠ Save this document for future reference.
- ⚠ Be careful when children are nearby and do not allow the children to play with the product.
- ⚠ Do not exceed the maximum payload.
- ⚠ Special care is always required where there are moving parts that can cause limb entrapment and injury.
- ⚠ Do not attempt to lift the product by any removable parts.
- ⚠ Do not use the product seat for a different purpose from the intended one.
- ⚠ The bedside rails are not intended to carry the full weight of an individual.

DECLARATION OF CONFORMITY:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

BEFORE USE:

- Check if the frame is damaged in order to guarantee a safe use of the product. (There are no cracks or fractures in the frame).
- In case of damage do not use the product and contact your dealer for further instructions.
- Check, if the product is properly assembled and all screws are secure and well screwed on.

- Always check the condition of wear for the mechanical parts to ensure the absolute safe use of the product for people and objects

ENVIRONMENTAL PROTECTION:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.

MAINTENANCE, CLEANING & DISINFECTION INFORMATION:

To guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

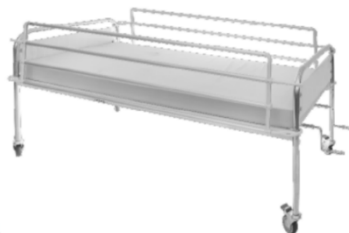
1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or mild detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.
5. DO NOT leave the product wet.
6. Do not use chemical cleaners to clean the frame. This could cause damage to the surface of the product
7. If you need to disinfect the product, use a common, mild disinfectant
8. Protect the item from scratches, cuts and punctures.
9. Lubrication: the product has designed for a minimal maintenance; however, check every six months the lubrication on the moving & mechanical parts, this would ensure an optimal duration.

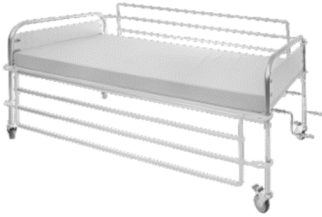
ASSEMBLING:

NOTE: The illustrations for the assembly are indicative to make the assembly easier. May will be differences from reality.

To assemble the Rails on the Bed:

1. Place the hooks on one side of the bed on the vertical part of the bed, e.g. on the headboard.
2. From the opposite end on the same side, e.g. on the footrest, extend the two hooks so can be hooked on the frame.

	Unique device identifier
	Imported by
	Store in a cool, dry place.
	Store away from sunlight.

WARRANTY:

The product is guaranteed for a period of one (1) year from the purchasing date.

The warranty apply on factory defects & DOES NOT apply on damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product’s operating instructions.

Also DOES NOT apply on the parts, that can wear out during use or over time. Damages or defects caused by natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty. No service or spare part is applied unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the end user- customer or by dealer.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

SYMBOL LEGEND

	Manufacturer
	Date of manufacture
	Product code
	Lot number
	Authorized representative in the European community
	Caution: read instructions (warnings) carefully
	Consult instructions for use
	Medical Device compliant with Regulation (EU) 2017/745
	Medical Device

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