GIMA

PROFESSIONAL MEDICAL PRODUCTS

BORSA GHIACCIO

ICE BAG

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EISTASCHE

BOLSA HIELO

BORSA DE GÊLO

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INTENDED USE

Before using it, ALWAYS ask for your doctor's opinion.

BEFORE EACH USE and from time to time apply to your doctor, who will show you possible collateral effects due to the specific use of the Device.

The present device aims at easing symptoms and/or dysfunctions generated on the human body by diseases; The ice to be used inside the bag can be produced using only drinkable water.

It can only be applied to external healthy parts of the body.

Other uses are considered improper and therefore FORBIDDEN:

- Do not use it in contact with injured parts or on parts inside the human body.
- Do not use it as invasive tool or together with invasive tools.
- Do not use together with or as part of other medical instruments, or together with drugs or substances considered improper.
- Do not modify or alter its components and/or features.
- Do not blow into the device and do not inflate it.

PRECAUTIONS AND WARNINGS

- STRICTLY OBSERVE THE INTENDED USE OF THE DEVICE.
- THE DEVICE CANNOT BE USED BY CHILDREN, DISABLE OR IMPAIRED PEOPLE WITHOUT SUPERVISION.
- IN CASE OF CONTEMPORARY USE OF SIMILAR DEVICES DO NOT INVERT DETACHABLE PARTS AND/OR ACCESSORIES.
- AFTER THE BAG IS FILLED, MAKE THE EXCEEDING AIR LEAK BEFORE PLUGGING IT. CLOSE THE BAG AND CHECK THE LOADING INLET TIGHTNESS.
- DO NOT USE OR LAY THE DEVICE ON/WITH: HOUSEHOLD ELECTRIC APPLIANCES, HOT OR POINTED SURFACES.
- BEFORE USING THE DEVICE ALWAYS MAKE SURE THAT ALL COMPONENTS ARE INTEGRAL AND NOT DAMAGED (FOR EXAMPLE CRACKING, DENTING, TEARING), THEIR COLOUR IS UNCHANGED AND THEY ARE FREE FROM SURFACE MOULDS, ETC

For a longer-lasting service life, it is advisable to store it in the original packing and in dry environment with temperatures ranging from +1°C to + 40°C, far from heat sources (sun or adverse weather conditions), sheltered from dust and dangerous materials and far from the children's reach.

MAINTENANCE AND DISPOSAL

Before using the device for the FIRST TIME and each time it is necessary, wash the interior of the bag with warm drinkable water and the external surface with warm water and neutral soap; do not use alcohol, solvents or acids. Do not use brushes, but only soft sponges so that not to scratch the rubber surface. After the use, completely empty the bag and make it drain/dry.

In case of replacement or breakages, dispose it according to the local legislation on the subject of waste disposal. No toxic substances are present.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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