**FASCIA BIADESIVA FOAM
BIADHESIVE FOAM STRIP
BANDE ADHÉSIVE DOUBLE-FACE EN MOUSSE
TIRA ADHESIVA AMBAS CARAS DE ESPUMA
FITA BIADESIVA DE ESPUMA
ZWEISEITIGE SCHAUMKLEBESTREIFEN
ΑΥΤΟΚΟΛΛΗΤΗ ΤΑΙΝΙΑ ΑΦΡΟΥ ΔΙΠΛΗΣ ΟΨΗΣ
TAŚMA PIANKOWA DWUSTRONNA
OBOUSTRANNÝ PĚNOVÝ LEPIČÍ PROUŽEK
DUBBELHÄFTANDE SKUMREMSA
KAKSIPUOLINEN VAAHTOMUOVILIIMANAUIHA
BANDĀ DIN SPUMĀ BIADEZIVĀ
DUBBELZIJDIGE SCHUIMSTRIP
KÉTOLDALÚ HAB RAGASZTÓSZALAG
ДВОЙНОЗАЛЕПВАЩА ЛЕПЕНКА ОТ ПЯНА**

Gima 28715

REF 19921503

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INSTRUCTIONS

Biadhesive strips provide secure and gentle fixation for external

catheters.

Please follow the securing instruction before using the product.

Biadhesive strips are accessories of the Male External Catheter (MEC) and compliant with Regulation (EU) 2017/745

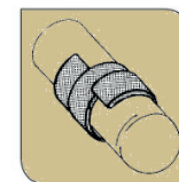
Soft, easy to apply, secure the MEC, Non-sterile, non-sterilizable product.

How To Apply:

1. Clean the Area before applying the Biadhesive strip.
2. The hair around the penis shaft may be shaved before application.
3. Peel off one of the two paper cover strips.
4. Wind the sticky side of the strip around the penis shaft in a spiral motion starting from the back of the base and working upwards.
5. Do not make the bi-adhesive strip too tight to prevent circulation problems.
6. The strip must overlap.
7. Before applying the MEC remove the other paper protection strip.
8. Apply the MEC leaving about 2cm between the top of the glans and the end of the MEC.

Warnings

- It is recommended that the MEC and the strip will be replaced every 12-24 hours.
- Do not over-tighten the double-sided adhesive strip to avoid circulatory problems.
- Non-sterile product, not sterilizable.
- Disposable product, do not re-use, do not recycle. After use, all medical devices that have been in contact with a patient can contain contaminants, including pathogenic microorganisms, that are difficult to eliminate.
- Store in a cool and dry place, away from sources of heat.
- After use, do not dispose to the environment

Spiral overlapping of the tape stripe

Apply pressure on the condom

Composition:

An adhesive strip made from an elastic rubber foam that adapts to the body and is easily removable. Glue protection: siliconized paper MEC

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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