ITALIANO

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ENGLISH

HYDROCOLLOID STRIPS INSTRUCTIONS FOR USE

Hydrocolloid strips provide secure and gentle fixation for external catheters.

Please follow the securing instruction before using the product.

Hydrocolloid strips are accessories MEC and compliant with Regulation (EU) 2017/745

Soft, easy to apply, secure the MEC, Non-sterile, non-sterilizable product.

How To Apply:

1. Clean the Area before applying the Hydrocolloid strip.
2. The hair around the penis shaft may be trimmed before application.
3. Peel off one of the two paper cover strips.
4. Wind the sticky side of the strip around the penis shaft in a spiral motion starting from the back of the base and working upwards.
5. Do not make the Hydrocolloid strip too tight to prevent circulation problems.
6. The strip must overlap.
7. Before applying the MEC remove the other paper protection strip.
8. Apply the MEC leaving about 2cm between the top of the glans and the end of the MEC.

Warnings

- It is recommended that the MEC and the strip will be replaced every 12-24 hours.
- Do not over-tighten the double-sided adhesive strip to avoid circulatory problems.
- Non-sterile product, not sterilizable.
- Disposable product, do not re-use, do not recycle. After use, all medical devices that have been in contact with a patient can contain contaminants, including pathogenic microorganisms, that are difficult to eliminate.
- Store in a cool and dry place, away from sources of heat.
- After use, do not dispose to the environment.

Composition: Hydrocolloid material

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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Mizra Medical Ltd
Gima S.p.A.