GIMA

PROFESSIONAL MEDICAL PRODUCTS

BARELLA CARICAMENTO AUTOMATICO AUTOMATIC LOADING STRETCHER BRANCARD À CHARGEMENT AUTOMATIQUE AUTOMATISCH VERLADENDE KRANKENTRAGE CAMILLA DE CARGA AUTOMÁTICA MACA COM SISTEMA DE CARREGAMENTO AUTOMÁTICO ΦΟΡΕΙΟ ΑΥΤΟΜΑΤΗΣ Φ'ΟΡΤΩΣΗΣ

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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Zhangjiagang Tengda Machinery Manufacturer Co., Ltd

No.999 Hongqi Road, Changyinsha
Modern Agricultural Demonstration Area,
Zhangjiagang City, Jiangsu, 215623, China
Made In China

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II. How to use

- The stretcher surface is made of soft sponge mattress, which make patient feel comfortable. One medical staff can push the trolley on to the ambulance or send to ward.
- When need low position, pull out the safety handle & then press it, the stretcher's height will automatically lower down.
- When destination arrived, pulling out the safety handle, the stretcher's legs will unfold automatically.
- There are the return elastic lock device at both side of the stretcher & the leg parts. Lock tightly when turn up for patient safety, turn down for patients out.
- Pressing the button under the stretcher, the head part will be raised or lowered.

III. Applied for

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, etc. to carry the wounded & patients. It also can be used in general operation.

IV. Cautions

- When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
- In general operation, be careful not to out torn the stretcher surface.

V. Maintenance

- Keep clean as routine (incl. sterilization).
- Often check if any parts loose or not.

VI. Storage & transportation

- Store this product in the place of damp proof & non corrosion.
- Common transporting vehicle can carry this product.
- Upside down is prohibited during transportation & storage. Stack the goods as per the instructions printed on the package.

	Product code		Manufacturer		Consult instructions for use
	Lot number		Medical Device compliant with Regulation (EU) 2017/745		Date of manufacture
	Keep in a cool, dry place		Caution: read instructions (warnings) carefully		Authorized representative in the European Union
	Keep away from sunlight		Medical Device		Unique identifier
	Imported by				

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

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