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EN OXYGEN THERAPY THERAPY MASKS INSTRUCTIONS FOR USE

INDICATIONS

Therapeutic administration of oxygen.

NB: Disposable product - do not reuse.

The oxygen source must be in accordance to the regulations in force.

The products must be used by qualified healthcare personnel.

PREPARATION AND CONNECTION

MEDIUM CONCENTRATION OXYGEN THERAPY MASK

Attach tubing connector to oxygen source. Attach other end to plastic output of mask. The mask has been designed to operate with a flow of between 5 and 10 LPM and to deliver oxygen flows of between 40% and 60%.

OXYGEN THERAPY MASK WITH VARIABLE CONCENTRATION

If using a model with fixed air-O2 dilutors (seven different colors), connect the selected dilutor to the corrugated tube; for models using a single air-O2 dilutor, select the concentration, make sure that the sliding part of the regulator is fully inserted in its seat, and connect the dilutor to the corrugated tube.

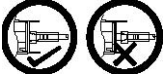
Connect cup to diluter. Attach sure flow tubing to diluter. Attach other end to plastic output of mask. The recommended flows and relative percentages of the oxygen delivered are printed on the diluter or on the label.

HIGH CONCENTRATION OXYGEN THERAPY MASK

Flatten plastic bag. Attach tubing connector to oxygen source. Attach other end to plastic output of mask. Adjust the capacity in order to prevent the bag from ever deflating more than half during inspiration. The oxygen flow delivered is contained between 90% and 100%.

OXYGEN THERAPY MASK FOR AEROSOL

Unscrew cap from base and add prescribed medication. Reassemble. Secure mask to top of nebulizer of opening provided. Attach one end of tubing to nebulizer output and the other end to oxygen or compressed air source. Verify that the connection is correct (as reported below).

For the duration of the administration follow the medical prescription. For an optimal nebulisation the recommended flow is between 6 and 9 LPM with a pressure of 3.44 bar (344 kPa, equivalent to 50 PSI). Efficient nebulisation is guaranteed up to a pressure of 1.73 bar (173 kPa, equivalent to 25 PSI).

ALL ABOVE MENTIONED MODELS: OXYGEN FLOW SETTING AND MASK DRESSING

Make sure tubing is safely fixed to oxygen source. Deliver oxygen and regulate the flow as prescribed by physician. Place mask on patient's face covering both mouth and nose. Pass elastic strap over patient's head and ears, on the neck. Adjust strap tension to hold mask in position by pulling. Model metal nose piece to patient's face.

OXYGEN THERAPY MASK FOR TRACHEOSTOMY OXYGEN THERAPY MASK FOR TRACHEOSTOMY WITH VARIABLE CONCENTRATION

Oxygen therapy mask for tracheostomy: connect aerosol tubing (not supplied with this package) between mask and gas source. Select correct liter flow at gas source and check for gas flow through mask device.

Oxygen therapy mask for tracheostomy with variable concentration: Connect prescribed diluter to corrugated tubing. Connect cup to diluter. Attach sure flow tubing to diluter. Attach other end to plastic output of mask. The recommended flows and relative percentages of the oxygen delivered are printed on the diluter.

Oxygen therapy mask for tracheostomy / Oxygen therapy mask for tracheostomy with variable concentration: position the elastic strap behind the neck and gently pull the end of the strap until the mask is secure. The mask inlet swivel is 360° to allow tubing to position itself for supine or upright patients. When using suction, loosen mask (by pulling strap in opposite direction or unsnapping one side of strap) and remove mask from suction area. Replace mask as previously noted. Warning: Be sure all connections are secure.

CONTRA-INDICATIONS

No contra-indications are described for oxygen therapy. For medicine administration refer to instructions supplied with medicine.

WARNINGS

If device is reused, contrarily to the present instructions, this may compromise following:

- Not cleanliness of the device and possible presence of biological residues that might cause cross-infections.
- Alteration of materials.
- Loss of initial functional features of product.

- Before connecting to oxygen source verify its functioning.
- The accidental crushing of kinking of the connection tubing do not allow oxygen to flow properly and may cause the disjunction of the connector. It is therefore recommended that the users control carefully.
- An oxygen flow of more than 10 litres per minute may cause a disturbing noise in the system. Also possibly liquid, coming from the humidifier, can filtrate into the mixture that is sent to the patient.
- All above models are disposable and cannot be used for a continuous period of more than 30 days.
- The concentrated oxygen can ignite combustible materials. To avoid risk of fire or explosion keep away from possible sources of ignition, and refer to the precautions for use of the equipment used for the delivery of oxygen.

WASTE DISPOSAL

Waste coming from health facilities must be disposed of according to the regulation in force.

GENERAL NOTES

If a serious incident occurs while using this device or as a result of its use, please report it to the Manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service..

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